**SWL: 300 kg/660 lbs**

## Table of contents

<b>English</b> .....	4-5
<b>Svenska</b> .....	6-7
<b>Norsk</b> .....	8-9
<b>Danske</b> .....	10-11
<b>Deutsch</b> .....	12-13
<b>Netherlands</b> .....	14-15
<b>Français</b> .....	16-17
<hr/>	
<b>Product information</b> .....	18

WendyLett is a slide sheet that facilitates positioning and turning in bed. It is bedded in the bed like a normal sheet.

WendyLett with elastic spring is bedded in the bed like a normal sheet and is fastened around the mattress with the elastic spring.

WendyLett can be used in combination with drawsheet, WendyLett2Way, or sliding drawsheet, WendyLett4Way.

WendyLett is available in several sizes and models.

SystemRoMedic is a series of different models of transfer assistive devices. The choice of assistive devices depends on the transfer situation and the patient's functional capability and needs.

## Functional inspection

### Visual inspection

Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.


## Always read the manual

Always read the manuals for all assistive devices used during a transfer.


Keep the manual where it is accessible to users of the product.

Do not leave the patient unattended during a transfer situation.


## Placement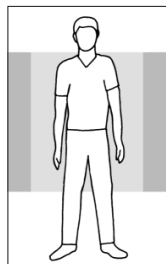


100x200  
100x220

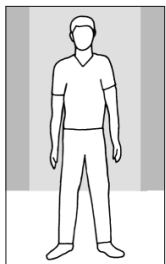140x200  
140x220

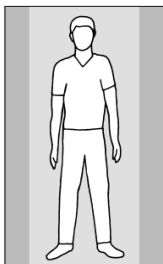200x200  
200x220

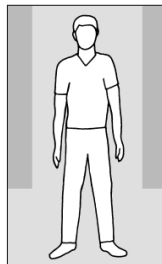
200x90, 200x105,  
200x120, 200x140

Place WendyLett with the smooth surface in the middle of the bed. It is bedded in the bed like a normal sheet.

Ensure that the mattress cover's rough surfaces or evenly distributed on both sides of the bed. These rough surfaces prevent the patient from sliding out of bed.

## Material

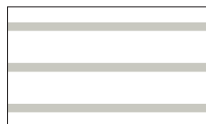
The WendyLett slide sheet has a pattern of white stripes on a grey background. Stripes indicate direction of glide.

The draw sheet, WendyLett 2Way, has a white fabric with grey stripes. Stripes indicate the direction in which the sheet is drawn.

WendyLett 4Way has a grey-checked pattern and has a sliding surface underneath. Stripes indicate the direction in which the sheet is drawn.


## Using WendyLett

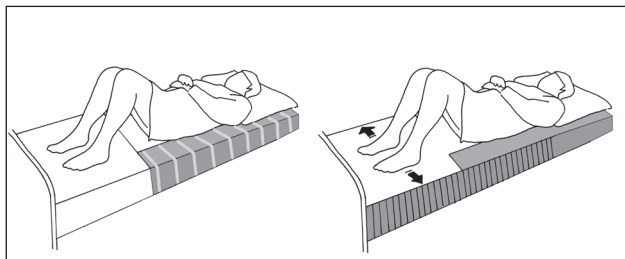WendyLett can be bedded in cases where the patient is able to turn himself/herself, but finds it easier with less friction.

The smooth surface of the sheet reaches over the foot of the bed, which makes it easier for the patient to shift his/her feet when sitting up at the edge of the bed.

## Accessories

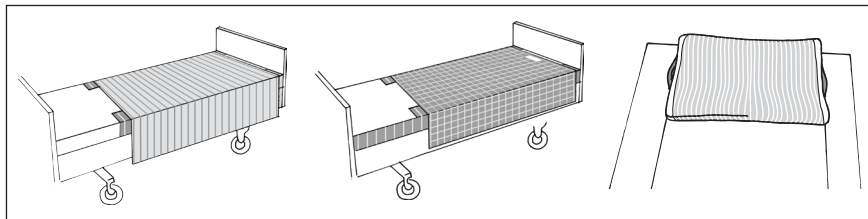With WendyLett2Way, the patient can turn himself/herself without sliding down in the bed when the head end is raised. If the patient needs some assistance or turning is heavy, WendyLett2Way can function as a draw sheet for the caregiver.

With WendyLett4Way, the patient can be helped to a position higher up in bed or closer to one side.

WendyLean is a pillow case with an anti-friction surface on the underside and handles on both sides. The anti-friction surface facilitates positioning of the pillow.

## Material:

Polyester, cotton, satin

## Care of the product:

Read the product label.

Do not use rinsing agent. To ensure maximum material life, hang dry or tumble dry on low heat.

**CE** Medical Device Class 1

The product meets the requirements of the Medical Devices Directive 93/42/EEC.

Contact your local distributor if you have any questions about the product and its use. See [www.handicare.com](http://www.handicare.com) for a complete list of distributors. Always make sure that you have the right version of the manual. The most recent editions of manuals are available for downloading from our website, [www.handicare.com](http://www.handicare.com).

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### WendyLett - sliding sheet

(Can be combined with WendyLett2Way drawsheet or WendyLett4Way sliding drawsheet)

1609	WendyLett, 60 cm/24" sliding surface ( <i>bed 80/90 cm - 32"/36"</i> )	100 x 200/39 x 79	White
1614	WendyLett, 60 cm/24" sliding surface ( <i>bed 80/90 cm - 32"/36"</i> )	140 x 200/55 x 79	White
1629	WendyLett, 60 cm/24" sliding surface ( <i>bed 80/90 cm - 32"/36"</i> )	100 x 200/39 x 79	grey/white striped
1634	WendyLett, 60 cm/24" sliding surface ( <i>bed 80/90 cm - 32"/36"</i> )	140 x 200/55 x 79	grey/white striped
1639	WendyLett, 70 cm/28" sliding surface ( <i>bed 105 cm/42"</i> )	200 x 200/79 x 79	grey/white striped
1667	WendyLett, 100 cm/40" sliding surface ( <i>bed 120 cm/48"</i> )	100 x 220/39 x 88	grey/white striped
1668	WendyLett, 100 cm/40" sliding surface ( <i>bed 120 cm/48"</i> )	140 x 220/55 x 88	grey/white striped
1669	WendyLett, 100 cm/40" sliding surface ( <i>bed 120 cm/48"</i> )	200 x 220/79 x 88	grey/white striped
1641	WendyLett, 70 cm/28" sliding surface ( <i>bed 90 cm/36"</i> ), tailored with elastic spring	200 x 90/79 x 36	grey/white striped
1651	WendyLett, 80 cm/32" sliding surface ( <i>bed 105 cm/42"</i> ), tailored with elastic spring	200 x 105/79 x 42	grey/white striped
1661	WendyLett, 95 cm/37" sliding surface ( <i>bed 120 cm/48"</i> ), tailored with elastic spring	200 x 120/79 x 47	grey/white striped
1671	WendyLett, 110 cm/43" sliding surface ( <i>bed 140 cm/56"</i> ), tailored with elastic spring	200 x 140/79 x 55	grey/white striped

### WendyLett2Way - drawsheet

(To be combined with WendyLett sliding sheet)

1620	WendyLett2Way	90 x 180/35 x 70	white
1631	WendyLett2Way	100 x 200/39 x 79	white/grey striped
1636	WendyLett2Way	140 x 200/55 x 79	white/grey striped

### WendyLett4Way - drawsheet

(To be combined with WendyLett sliding sheet)

1637	WendyLett4Way	140 x 200/55 x 79	grey/white checked
1647	WendyLett4Way	200 x 200/79 x 79	grey/white checked
1648	WendyLett4Way, inco	75 x 200/30 x 79	grey (inco) checked
1649	WendyLett4Way, inco with handles	140 x 120/55 x 47	grey (inco)
1657	WendyLett4Way, with handles	140 x 190/55 x 75	grey/white checked
1652	WendyLett4Way	140 x 260/55 x 102	grey/white checked

### WendyLean - pillowcase with handles

1680	WendyLean	50 x 60/20 x 24	grey/white striped
------	-----------	-----------------	--------------------

# SystemRoMedic™

For 25 years we have applied ourselves wholeheartedly to developing smart and easy-to-use assistive devices for easy transfers and to making life and work easier for both patients and personnel in the care sector.

Experience, innovation and training are the basis for SystemRoMedic, a total solution for every imaginable transfer situation.

**Transfer:** products for moving patients between locations.

**Positioning:** products for repositioning in the same location.

**Support:** products that provide mobility support.

**Lifting:** products adapted for lifting.

The philosophy behind SystemRoMedic is to prevent occupational injuries while improving the patient's sense of independence and dignity. Through a combination of training and a complete range of transfer-assistive devices, SystemRoMedic offers the means to improving both the work environment and the quality of care while enabling significant cost savings.

Our mission, to help people, has always been, and will continue to be, the driving force of innovation. We love easy transfers.

Contact your local distributor if you have any questions about the product and its use. See [www.handicare.com](http://www.handicare.com) for a complete list of distributors. Always make sure that you have the right version of the manual. The most recent editions of manuals are available for downloading from our website, [www.handicare.com](http://www.handicare.com)

handicare

Handicare AB  
Veddestav. 15, Box 640  
SE-175 27 Järfälla  
SWEDEN

Tel: +46 (0)8-557 62 200  
Fax: +46 (0)8-557 62 299  
E-mail: [info@handicare.se](mailto:info@handicare.se)

Internet: [www.handicare.com](http://www.handicare.com)