User manual - English

Bruksanvisning - Svenska

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**REF**

1609 1614  
1629 1634  
1639 1667  
1668 1669  
1641 1651  
1661 1671  
1681 1683  
1634CH  
1641CH  
1633

**SWL: 300 kg/660 lbs**

## Table of contents

<b>Product information</b> .....	<b>3</b>
<b>English</b> .....	<b>5</b>
<b>Svenska</b> .....	<b>7</b>
<b>Norsk</b> .....	<b>9</b>
<b>Dansk</b> .....	<b>11</b>
<b>Suomi</b> .....	<b>13</b>
<b>Deutsch</b> .....	<b>15</b>
<b>Nederlands</b> .....	<b>17</b>
<b>Français</b> .....	<b>19</b>
<b>Italiano</b> .....	<b>21</b>
<b>Español</b> .....	<b>23</b>

**Product information/ Produktinformation/ Produktinformasjon/ Produktinformation/  
Tuotetiedot/ Produktinformation/ Productinformatie/ Information produit/  
Informazioni sul prodotto/ Información del Producto**

Art. no.	Product/Produkt/ Tuote/ Produktname/ Product/ Produit/ Prodotto/ Producto	Size/ Storlek/ Størrelse/ Koko/ Größe/ Maat/ Taille/ Dimensione/ Tamaño cm/inch	Colour/Färg/Farge/ Farve/ Väri/ Farbe/ Kleur/ Couleur/ Colore/ Color
----------	--	--	---

**WendyLett - sliding sheet/ glidlakan/ glidelaken/ glidelagen/ liukuvan telttakankaan/ piirtää arkki/ gleitlaken/ glijlaken/ feuille de glissement/ telone scorrevole/ sábana de deslizamiento**

(Can be combined with WendyLett2Way drawsheet or WendyLett4Way sliding drawsheet)

1609	WendyLett, 60 cm/24" sliding surface ( <i>bed 80/90 cm - 32"/36"</i> )	100 x 200/39 x 79	white
1614	WendyLett, 60 cm/24" sliding surface ( <i>bed 80/90 cm - 32"/36"</i> )	140 x 200/55 x 79	white
1629	WendyLett, 60 cm/24" sliding surface ( <i>bed 80/90 cm - 32"/36"</i> )	100 x 200/39 x 79	grey/white striped
1634	WendyLett, 60 cm/24" sliding surface ( <i>bed 80/90 cm - 32"/36"</i> )	140 x 200/55 x 79	grey/white striped
1639	WendyLett, 70 cm/28" sliding surface ( <i>bed 105 cm/42"</i> )	200 x 200/79 x 79	grey/white striped
1667	WendyLett, 100 cm/40" sliding surface ( <i>bed 120 cm/48"</i> )	100 x 220/39 x 88	grey/white striped
1668	WendyLett, 100 cm/40" sliding surface ( <i>bed 120 cm/48"</i> )	140 x 220/55 x 88	grey/white striped
1669	WendyLett, 100 cm/40" sliding surface ( <i>bed 120 cm/48"</i> )	200 x 220/79 x 88	grey/white striped
1641	WendyLett, 70 cm/28" sliding surface ( <i>bed 90 cm/36"</i> ), tailored with elastic string	200 x 90/79 x 36	grey/white striped
1651	WendyLett, 80 cm/32" sliding surface ( <i>bed 105 cm/42"</i> ), tailored with elastic string	200 x 105/79 x 42	grey/white striped
1661	WendyLett, 95 cm/37" sliding surface ( <i>bed 120 cm/48"</i> ), tailored with elastic string	200 x 120/79 x 47	grey/white striped
1671	WendyLett, 110 cm/43" sliding surface ( <i>bed 140 cm/56"</i> ), tailored with elastic string	200 x 140/79 x 55	grey/white striped
1681	WendyLett, 70 cm/28" sliding surface	200 x 365/79 x 144	grey/white striped

**WendyLett2Way - drawsheet/ draglakan/ stikkklakenet/ træklagen/ piirtää arkki/ zuglaken/ treklaken/ dessiner feuille/ foglio di copertura/ sábana travesera**

(To be combined with WendyLett sliding sheet)

1620	WendyLett2Way	90 x 180/35 x 70	white
1631	WendyLett2Way	100 x 200/39 x 79	white/grey striped
1636	WendyLett2Way	140 x 200/55 x 79	white/grey striped

**WendyLett4Way - drawsheet/ draglakan/ stikkklakenet/ træklagen/ piirtää arkki/ zuglaken/ treklaken/ dessiner feuille/ foglio di copertura/ sábana travesera**

(To be combined with WendyLett sliding sheet)

1637	WendyLett4Way	140 x 200/55 x 79	grey/white checked
1647	WendyLett4Way	200 x 200/79 x 79	grey/white checked
1648	WendyLett4Way, inco	75 x 200/30 x 79	grey (inco) checked
1649	WendyLett4Way, inco with handles	140 x 120/55 x 47	grey (inco)
1657	WendyLett4Way, with handles	140 x 190/55 x 75	grey/white checked
1652	WendyLett4Way	140 x 260/55 x 102	grey/white checked

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WendyLett is a slide sheet that facilitates positioning and turning in bed. It is bedded in the bed like a normal sheet. WendyLett with elastic spring is bedded in the bed like a normal sheet and is fastened around the mattress with the elastic spring.

WendyLett can be used in combination with drawsheet, WendyLett2Way, or sliding drawsheet, WendyLett4Way.

WendyLett is available in several sizes and models.

## Check Safety

### Visual inspection

Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.

### Friction test

Low friction - Ensure that the surface is smooth and slides effortlessly. Compare with a new product.


## Always read the user manual


Always read the user manuals for all assistive devices used during a transfer. Keep the user manual where it is accessible to users of the product. Always make sure that you have the right version of the user manual.

The most recent editions of user manuals are available for downloading from our website, [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com).


## Placement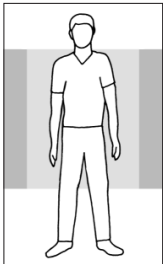


100x200  
100x220

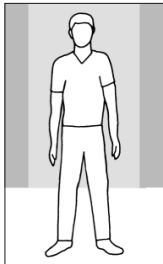140x200  
140x220

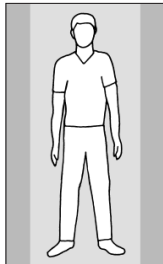
200x200  
200x220  
200x365

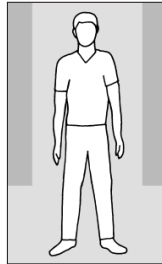200x90, 200x105,  
200x120, 200x140

Place WendyLett with the smooth surface in the middle of the bed. It is bedded in the bed like a normal sheet. Ensure that the mattress cover's rough surfaces or evenly distributed on both sides of the bed. These rough surfaces prevent the user from sliding out of bed.

## Material

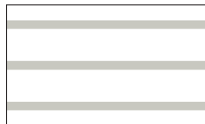
The WendyLett slide sheet has a pattern of white stripes on a grey background. Stripes indicate direction of glide.

The draw sheet, WendyLett2Way, has a white fabric with grey stripes. Stripes indicate the direction in which the sheet is drawn.

WendyLett4Way has a grey-checked pattern and has a sliding surface underneath. Stripes indicate the direction in which the sheet is drawn.


## Using WendyLett

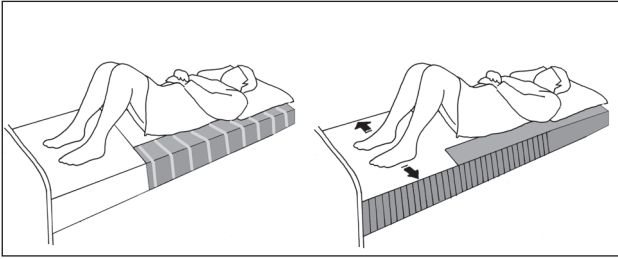WendyLett can be bedded in cases where the user is able to turn himself/herself, but finds it easier with less friction.

The smooth surface of the sheet reaches over the foot of the bed, which makes it easier for the user to shift his/her feet when sitting up at the edge of the bed.

## Accessories

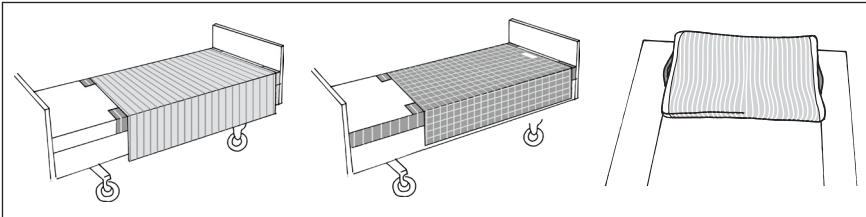With WendyLett2Way, the user can turn himself/herself without sliding down in the bed when the head end is raised. If the user needs some assistance or turning is heavy, WendyLett2Way can function as a draw sheet for the caregiver.

With WendyLett4Way, the user can be helped to a position higher up in bed or closer to one side.

WendyLean is a pillow case with an anti-friction surface on the underside and handles on both sides. The anti-friction surface facilitates positioning of the pillow.

## Material

Polyester, cotton, satin

## Care of the product

Read the product label.

Do not use rinsing agent. To ensure maximum material life, hang dry or tumble dry on low heat.

**CE** Medical Device Class I

Medical Device Class I. The product complies with the requirements of the Medical Devices Directive 93/42/EEC.

Contact your local distributor if you have any questions about the product and its use.

See [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com) for a complete list of distributors.

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## Simple solutions for great results

SystemRoMedic™ is the name of Direct Healthcare Group's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic™ is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

## Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic™ is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic™ offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

## Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading from our website; [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com).

## For questions about the product and its use

Please contact your local Direct Healthcare Group and SystemRoMedic™ representative. A complete list of all our partners with their contact details can be found on our website; [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com).

# Direct Healthcare Group

## Advancing Movement & Health

Direct Healthcare Group Sverige AB  
Torshamnsgatan 35,  
SE-164 40 Kista, Sweden

Tel: +46 (0)8-557 62 200  
[info@directhealthcaregroup.com](mailto:info@directhealthcaregroup.com)  
[www.directhealthcaregroup.com](http://www.directhealthcaregroup.com)