Installation instructions

Remove all packaging and unfold the frame, locate the keylock pin into keyhole and lock down into position on both sides of the frame, adjust to the desired height by pressing push button on height adjustable legs and sliding leg to the desired height position. Ensure the push button locates firmly back into the hole.

Using the product

Over a toilet, place the toilet frame over the toilet, without the bucket.

Beside a bed, place the toilet frame beside the bed, with bucket.

Shower chair, place the toilet frame in the shower, without the bucket.

Use the armrests on both sides during making the transfer in/out of the toilet frame.

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Maintenance

Clean regularly with normal household detergent. Do not use aggressive cleaners.

Technical data

Material, frame:	Aluminium
Material, screws:	Stainless steel
Material, seat:	Plastic
Material, bucket with lid:	Plastic
Material, feet (4 pcs):	Non-slip rubber
Max. user weight:	130 kg/285 lbs
Width, frame:	540 mm, 21.3"
Depth, frame:	500 mm, 19.7"
Seat height:	480-640 mm, 18.9"-25.2"
	(adjustment, 160 mm, 6.3", in 7 steps of 25 mm, 1")
Seat depth:	420 mm, 16.5"
Height, armrests (from seat):	160 mm, 6.3"
Distance between armrests:	475 mm, 18.7"
Height backrest:	860-1060 mm, 33.8"-41.7"
Height clearance:	420-580 mm, 16.5"-22.8"
Width clearance:	420 mm, 16.5"
Total weight:	3,8/4 kg, 8.4/8.8 lbs

Warning

- Risk of catching injury in the folding mechanism.
- Do not load the seat surface on one side only.
- Under no circumstance stand on the seat.

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Manual - Foldable toilet frame, Foldable toilet frame with backrest

Item No. 10499 & 10498
Max: 130 kg/285 lbs
Manual nr: 00748 Ver. 1 110921

Manufactured for: Valentine International Ltd., Taiwan R.O.C.

Contact your local distributor if you have any questions about the product and its use. See www.handicare.com for a complete list of distributors. Always make sure that you have the right version of the manual. The most recent editions of manuals are available for downloading from our website. www.handicare.com.

English

Functional inspection

Visual inspection
Inspect the product regularly. Check to ensure that material is free from damage.

Before use
Ensure that the product is correctly mounted/installed. Ensure that the frame is level and stable with all legs evenly placed on the floor.

Always read the manual

Always read the manual.
Keep the manual where it is accessible to users of the product.

Ready for life is a series of assistive devices that are designed to give users the best possible conditions for living an active life on a daily basis. The products have been specially developed to meet the conditions for homecare and institutional care settings. Ready for life offers the right quality, the right function and the right performance at the right price.

Distributed by:
Vedusav, 13, Box 640
SE-775 27 Bahala
SWEDEEN

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
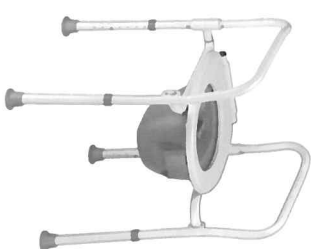
Item No.
10499

EN The foldable toilet frame is multi-purpose: as a toilet frame over a toilet, as a bedside commode when used with bucket and lid, as a shower chair and perfect for travelling and easy to store because of the compact and foldable frame.

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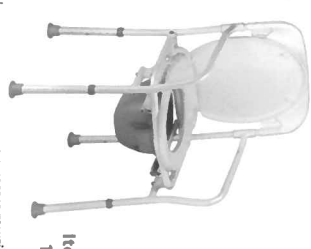Item No.
10498

EN The foldable toilet frame with backrest is multi-purpose: as a toilet frame over a toilet, as a bedside commode when used with bucket and lid, as a shower chair and perfect for travelling and easy to store because of the compact and foldable frame.

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