
Ready for life

Manual - Armrests with floorsupport

Item No. 10540
Max: 100 kg/220 lbs
Manual nr: 00752 Ver. 1 110921

Ready for life is a series of assistive devices that are designed to give users the best possible conditions for living an active life on a daily basis. The products have been specially developed to meet the conditions for homecare and institutional care settings. Ready for life offers the right quality, the right function and the right performance at the right price.

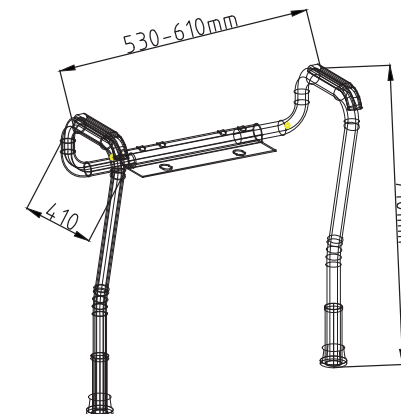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Tel: +46 (0)8-557 62 200
Fax: +46 (0)8-557 62 299

Internet: www.handicare.com
E-mail: info@handicare.se

Manufactured for: Valentine International Ltd., Taiwan R.O.C.

Contact your local distributor if you have any questions about the product and its use. See www.handicare.com for a complete list of distributors. Always make sure that you have the right version of the manual. The most recent editions of manuals are available for downloading from our website, www.handicare.com.

English

Svenska

Deutsch

Nederlands

Functional inspection

Visual inspection

Inspect the product regularly. Check to ensure that material is free from damage.

Before use:

Ensure that the product is correctly mounted/installed. Check tightness of toilet seat bolt regularly.

Always read the manual

Always read the manual.
Keep the manual where it is accessible to users of the product.

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Installation instructions

- Remove toilet seat from toilet and place bracket onto toilet base with frame assembly attached. Line up bracket holes with toilet seat screw bolt holes. Insert the toilet seat bolts through the bracket, then through the holes in the toilet. Before placing bolts through the holes, place washers between the bracket and the toilet. Tighten the toilet seat bolts well.
- Adjust leg height extension legs. Warning: when adjusting the height of the leg extension, make sure that each is set into using same height adjustment hole and that the frame is level. Push buttons must fully protrude through the adjustment hole of each leg extension to ensure a positive lock.
- Adjust the width of the armrests on both sides by pressing push buttons on both sides. Ensure the push button locates firmly back into the hole.

Using the product

The floorsupport must be standing on the floor. The load on the armrests should be in a downwards direction. The armrests can be lifted a bit for cleaning.

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English

Maintenance

Clean regularly with normal household detergent. Do not use aggressive cleaners.

Technical data

Material, frame and bracket:	Aluminum
Material, armrests:	Plastic
Total width:	530-610 mm, 20.9 - 24"
Width between armrests:	420-500mm, 16.5 - 19.7"
Armrests, length:	410 mm, 16.1"
Armrests, height:	710-810 mm, 27.9 - 31.9" (in 5 steps of 25 mm, 0.98")
For toilet heights:	400-500 mm, 15.7 - 19.7"
Max. user weight:	100 kg/220 lbs

Fits on toilets with 75 mm free depth for bracket.

Warning

- Please inspect all parts carefully for cracks prior to use.
- Do not load the armrests on one side only.
- The product is not suitable for sideward or lateral transfer.

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