English
Svenska
Norske
Danske
Deutsch
Français

Ready for life

Manual - ReadyMove

Manual nr: 00823 Ver. 3 110211

Ready for life is a series of assistive devices that are designed to give users the best possible conditions for living an active life on a daily basis. The products have been specially developed to meet the conditions for homecare and institutional care settings. Ready for life offers the right quality, the right function and the right performance at the right price.

Veddestav. 15, Box 640
SE-175 27 Järfälla
SWEDEN

Tel: +46 (0)8-557 62 200
Fax: +46 (0)8-557 62 299

Internet: www.handicare.com
E-mail: info@handicare.se

English

Svenska

With ReadyMove, using minimal effort and without lifting, a caregiver can give a user safe and secure transfer and mobility support.

Functional inspection

Visual inspection

Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.

Mechanical load

Test the ReadyMove regularly by stepping on one handle and pulling hard on one of the handles on the opposite side. Repeat the procedure until all of the handles have been tested.

Always read the instructions

Always read the manuals for all assistive devices used during a transfer. Keep the manual where it is accessible to users of the product.

Do not leave the patient unattended during a transfer situation.

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Using ReadyMove

Transfer:

For transfers between e.g., wheelchair and bed, a ReadyGlide/EasyGlide should also be used, in some cases, even a ReadyBelt/EasyBelt/FlexiBelt (1).

Raising:

For raising, ReadyMove can be used to move the patient's legs over the side of the bed (2) and then as support behind the back/seat (4). If the patient requires the assistance of two caregivers, one should help with the legs and the other with the upper body (3). During the actual raising operation, the caregivers stand on either side of the patient and choose the handles that enable the best working stance (5).

Higher up in bed:

To reposition the patient higher up in bed, ReadyMove is placed under the seat (5). It may also be advisable to use a friction-reducing device under the shoulders, for example, ReadySlide/EasySlide (6).

Material:

Polypropylene

Care of the product:

Read the product label.  Do not use rinsing agent. To prolong product life, avoid tumble-drying.

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