Ready for life

Manual - Rotating bathtub bar

Item No. 10879
Max: 130 kg/285 lbs
Manual nr: 00746 Ver. 1 110921

Ready for life is a series of assistive devices that are designed to give users the best possible conditions for living an active life on a daily basis. The products have been specially developed to meet the conditions for homecare and institutional care settings. Ready for life offers the right quality, the right function and the right performance at the right price.

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Manufactured for: Valentine International Ltd., Taiwan R.O.C.

Contact your local distributor if you have any questions about the product and its use. See www.handicare.com for a complete list of distributors. Always make sure that you have the right version of the manual. The most recent editions of manuals are available for downloading from our website, www.handicare.com.

Distributed by:
Veddestav, 15, Box 640
SE-175 27 Järfälla
SWEDEN

Tel: +46 (0)8-557 62 200
Fax: +46 (0)8-557 62 299

Internet: www.handicare.com
E-mail: info@handicare.se

English

Svenska

Deutsch

Nederlands

Functional inspection

Visual inspection

Inspect the product regularly. Check to ensure that material is free from damage.

Before use

Ensure that the product is correctly mounted/installed. Ensure that the bathtub bar is securely attached to the tub by trying to move the bathtub bar back and forth by applying some force to the handlebar. If loose, tighten clamp adjustment knob further.

Always read the manual

Always read the manual.
Keep the manual where it is accessible to users of the product.

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Installation instructions

- Position the bathtub bar onto the edge of the bathtub in an area that is flat on the inside and outside,. Make sure the clamp adjustment knob is facing the outside of the tub.
- Make sure both sides of the clamp are positioned flat against the inside and outside of the tub.
- Turn (clockwise) and tighten the clamp adjustment knob until the unit is securely clamped to the bathtub.

Using the product

Determine the appropriate handle position during a transfer in and out of the bathtub. Adjust to the desired handle position by pulling up on the latch pin, rotating the handle to the desired position, and ensuring that the latch pin drops back down to its locked position with an audible click. Hold the bathtub bar during stepping in and out the bathtub.

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Maintenance

Clean regularly with normal household detergent. Do not use aggressive cleaners.

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