Ready for life

## Manual - Toilet raiser with armrests and lid

Item No. 10659  
Max: 130 kg/285 lbs  
Manual nr: 00749 Ver. 1 110921

*Ready for life is a series of assistive devices that are designed to give users the best possible conditions for living an active life on a daily basis. The products have been specially developed to meet the conditions for homecare and institutional care settings. Ready for life offers the right quality, the right function and the right performance at the right price.*


**Distributed by:**  
Veddestav, 15, Box 640  
SE-175 27 Järfälla  
SWEDEN

Tel: +46 (0)8-557 62 200  
Fax: +46 (0)8-557 62 299

Internet: [www.handicare.com](http://www.handicare.com)  
E-mail: [info@handicare.se](mailto:info@handicare.se)

- EN** Toilet raiser with armrests and lid, raises toilet seat height 85 mm/3.35", 125 mm/4.92" or 165 mm/6.5" and offers support during sitting down and rising up.
- SV** Toalettförhöjare med armstöd och lock, höjer toalettsitsen 85 mm, 125 mm eller 165 mm och ger stöd vid uppstigning.
- DE** Toilettensitzhöhung mit Armlehnen und Deckel, erhöht den WC-Sitz um 85 mm, 125 mm oder 165 mm und bietet Stütze beim Aufstehen.
- NL** Toileetverhoger met armleuningen en deksel, verhoogt de zithoogte van het toilet met 85 mm/3.35", 125 mm/4.92" of 165 mm/6.5" en biedt steun tijdens het opstaan en gaan zitten.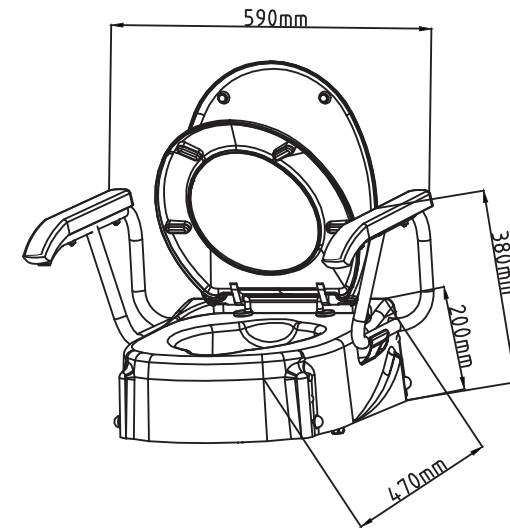

**Manufactured for:** Valentine International Ltd., Taiwan R.O.C.

Contact your local distributor if you have any questions about the product and its use. See [www.handicare.com](http://www.handicare.com) for a complete list of distributors. Always make sure that you have the right version of the manual. The most recent editions of manuals are available for downloading from our website, [www.handicare.com](http://www.handicare.com).

English

Svenska

Deutsch

Nederlands

### Functional inspection

#### Visual inspection

Inspect the product regularly. Check to ensure that material is free from damage.

#### Before use:

Ensure that the product is correctly mounted/installed. Periodical checking of all fixings for tightness is recommended.

### Always read the manual

Always read the manual.  
Keep the manual where it is accessible to users of the product.

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