Ready for life
Manual - Bathboard
Item No. 12180
Max: 150 kg/330 lbs
Manual nr: 00745 Ver. 1 110921

Ready for life is a series of assistive devices that are designed to give users the best possible conditions for living an active life on a daily basis. The products have been specially developed to meet the conditions for homecare and institutional care settings. Ready for life offers the right quality, the right function and the right performance at the right price.

Manufactured for: Valentine International Ltd., Taiwan R.O.C.

Contact your local distributor if you have any questions about the product and its use. See www.handicare.com for a complete list of distributors. Always make sure that you have the right version of the manual. The most recent editions of manuals are available for downloading from our website, www.handicare.com.

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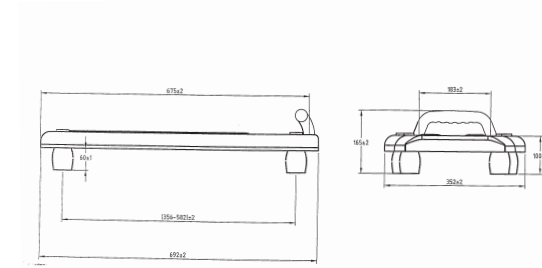
Functional inspection

Visual inspection

Inspect the product regularly. Check to ensure that material is free from damage.

Before use

Ensure that the product is correctly mounted/installed.

Always read the manual

Always read the manual.
Keep the manual where it is accessible to users of the product.

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Installation instructions

Handle:

1.To fit the blue handle, lay the bathboard with stops facing downwards and introduce the handle to the holes at the end of the board. The handle grip should face towards the centre of the board. From the underside of the bathboard, screw the large headed white plastic screws into the holes at the ends of the handles. Make sure not to cross the threads. Tighten using a large headed screwdriver or a small coin.

Bathboard:

2. Reverse the board and loosen the 4 supports, which hold the stops in place. Make sure the stops are free to move and that the non-slip white pads are facing the outside of the board.

3. Slide the 4 stops back towards the centre of the bathboard so that they will easily fit down into the bath.

4. Reverse the bathboard so that the stops hang down into the bath and the board is against the wall on the handle side. Handle should always be on the wall side.

5. Position the bathboard in the optimum position.

6. Slide the stops out so they touch the inside of the bath and tighten the wing screws finger tight. Do not over tighten.

7. Reverse the board again so that the stops are pointing upwards and adjust them outwards by approximately 2 mm on each stop. Now tighten the supports so that they do not move easily.

8. Turn the bathboard the right way up and press it gently down into the bath, without forcing it. If the board slides along the bath, the stops need to be adjusted further outwards. To do this repeat step 2 to 7.

Using the product

- Sit on the bathboard with the legs outside of the bath.
- Place one leg in the bathtub. The handle can be held for extra support.
- Place the other leg in the bathtub.
- Sit in the middle of the bathboard whilst washing.
- Move one leg outside the bathtub.
- Move the other leg outside the bathtub and place both feet on a non-slippery surface.

Maintenance

Clean regularly with normal household detergent. Do not use aggressive cleaners.

Technical data

Max. user weight:	150 kg/330 lbs
Weight:	2.5 kg/5.5 lbs
Length:	690 mm, 27.2"
Height:	100 mm, 3.9", including handle: 175 mm, 6.9"
Depth:	350 mm, 13.8" (side with handle), 300 mm, 11.8" (side without handle), adjustment: 310-530 mm, 12.2"-20.9"
Width, handle:	110 mm, 4.3"
Diameter, handle:	28 mm, 1.1"
Distance, handle-board:	50 mm, 1.97"
Material:	Plastic and rubber

Fits bathtubs up to 625 mm, 24.6" inner width, adjustable width: 410-625 mm, 16.1"-24.6"

Warning

- Check the board length in relation to the bathtub.
- Not suitable for straight sided baths.
- Under no circumstance stand on the board.

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