

Manual - Bed table

Item No. 300400310
Max: 20 kg/44 lbs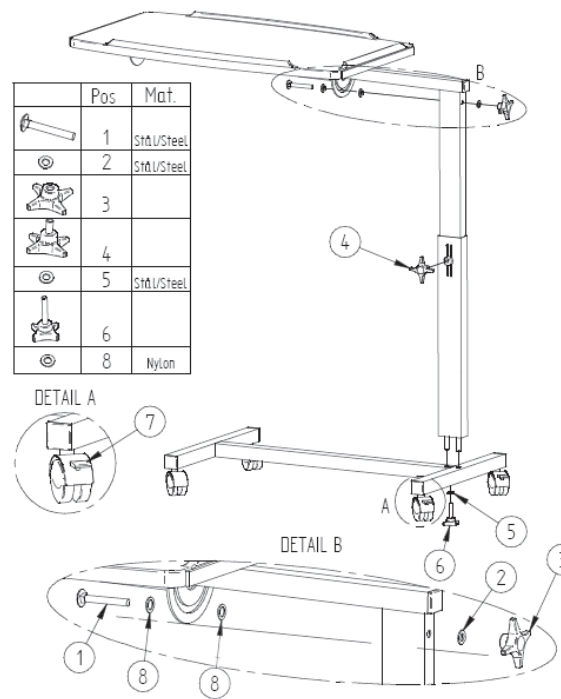

	Pos	Mat.
	1	Stål/Steel
	2	Stål/Steel
	3	
	4	
	5	Stål/Steel
	6	
	8	Nylon

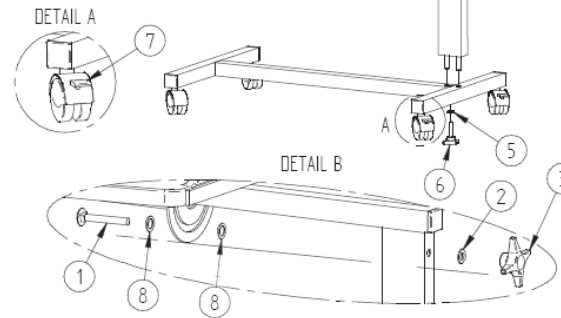Montage Nederlands

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Assembly English Montering Norske Asennus Suomi Montaje Español

Mount the upright on the base (pos.5 + 6) using the accompanying locking handle and washer. Tighten securely. Adjust the angled part of the frame to the desired height and tighten the locking handle (pos.4). The underside of the tabletop is equipped with two semi-circular fixtures. The angle of the tabletop is adjusted via the bolt (pos.1), which is inserted into the two semi-circular fixtures. When the tabletop is in place, mount bolt, washers and locking handle (pos.1 + 8 + 2 + 3), adjust to the desired angle and tighten.

Adjustments

The height of the bed table is regulated by raising and lowering the upright and locking handle (pos.4). Tighten the locking handle at the desired height position. The tabletop can be angled when the locking handle (pos.3) is loosened. Tighten the locking handle at the desired angle. The castors can be locked by pressing down on the brake pedals (pos.7).

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English

The bed table can be turned so that the legs fit under the bed and the tabletop is over the bed. The bed table must be used on a flat floor surface, not on a carpeted floor. The tabletop can be turned and angled to making reading and writing easier.

Max: 20 kg/44 lbs

Functional inspection

Visual inspection

Inspect the product regularly. Check to ensure that material is free from damage.

Before use:

Ensure that the product is correctly assembled.
Ensure that locking handles are securely tightened.

Always read the manual

Always read the manual.
Keep the manual where it is accessible to users of the product.

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
English

Important information

- Max. load is 20 kg.
- Ensure that locking handles are securely tightened.
- For optimal safety, we recommend that the castors are locked when the table is standing still.
- The bed table must not be used for support when standing.

Accessories

Bed table extension
Article number: 300500310

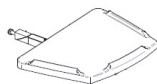Maintenance

The bed table can be cleaned with a non-abrasive, non-corrosive household cleaning agent. Solvents and chlorine should not be used. For optimal function, keep castors free of dirt and hair.

Recycling

Plastic components are sorted as combustible and steel parts as scrap metal.

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Direct Healthcare Group Sverige AB