
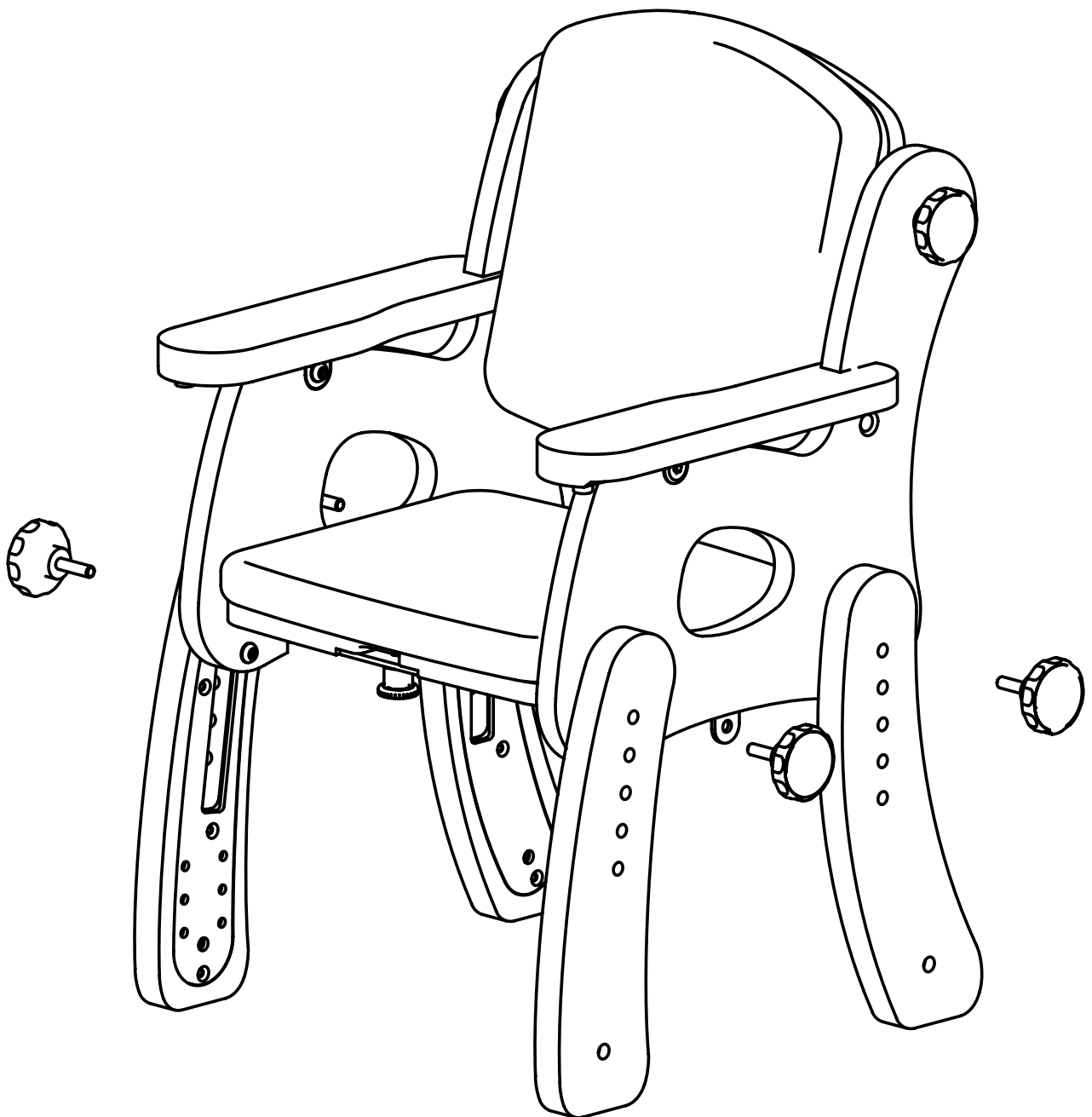
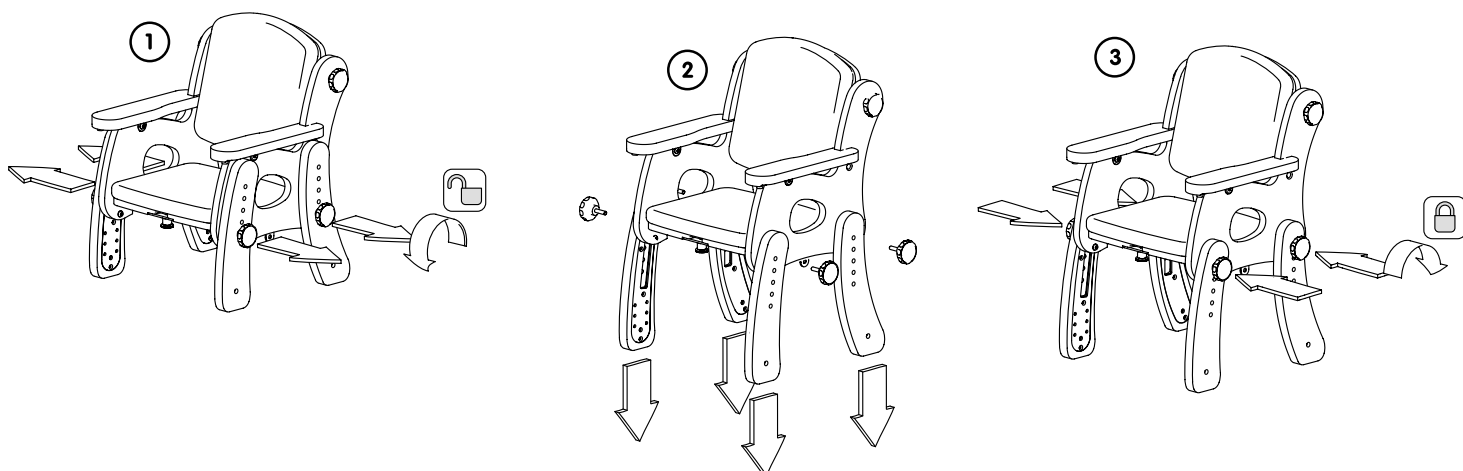
# LECKEY® Pal

User manual  
Bedienungsanleitung  
Guida per l'uso  
Mode d'emploi  
Manual del usuario


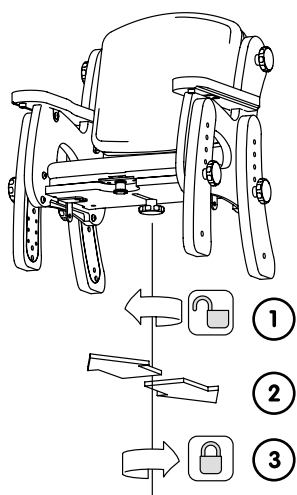
Seat depth  
Sitztiefe  
Profondità del sistema di seduta  
Profondeur d'assise  
Profundidad del asiento






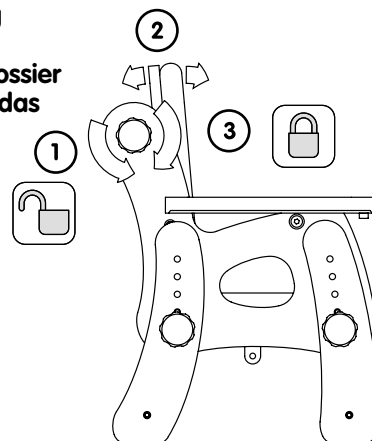
Backrest angle  
Rückenwinkelverstellung  
Angolo sedile schienale  
Angle d'inclinaison du dossier  
Angulo del reposa espaldas



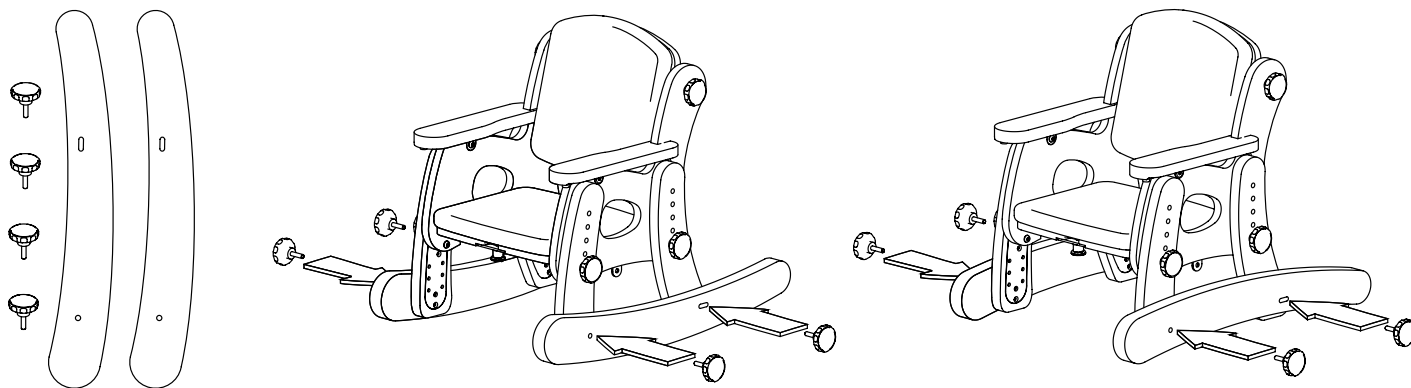
Rocker and stabiliser / Kufen zum Schaukeln oder Stabilisieren / Dondolo e stabilizzatore  
Système à bascule et stabilisateur / Mecedor y estabilizador

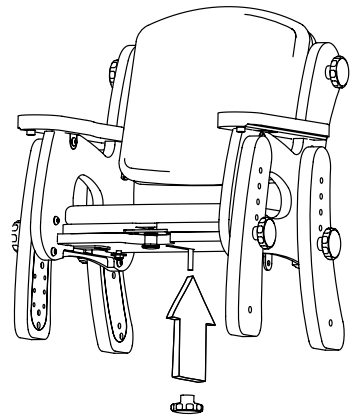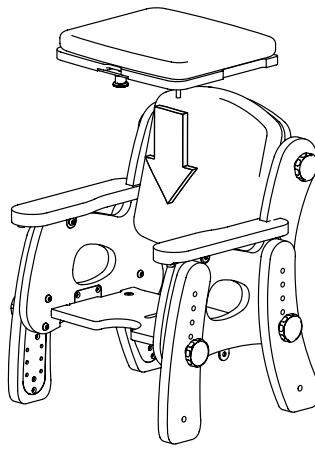
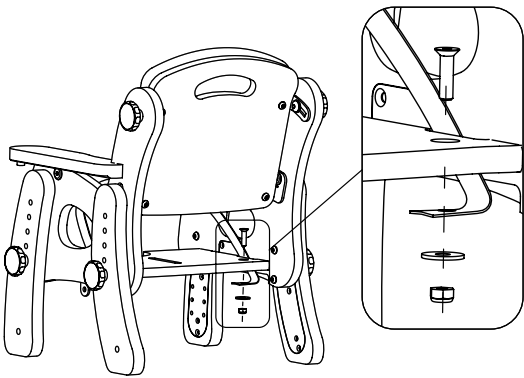
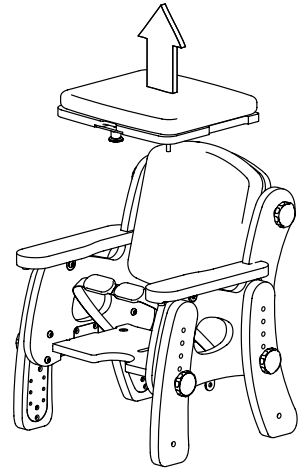
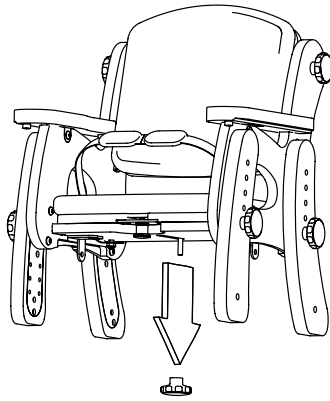
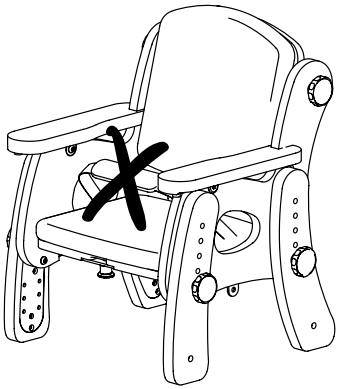
Removing the 2 point pelvic strap / Entfernen des 2-Punkt Beckengurts / Rimozione della cintura pelvica a 2 punti /  
Enlèvement de la sangle pelvienne à 2 points / Retirar la correa pélvica de 2 puntos

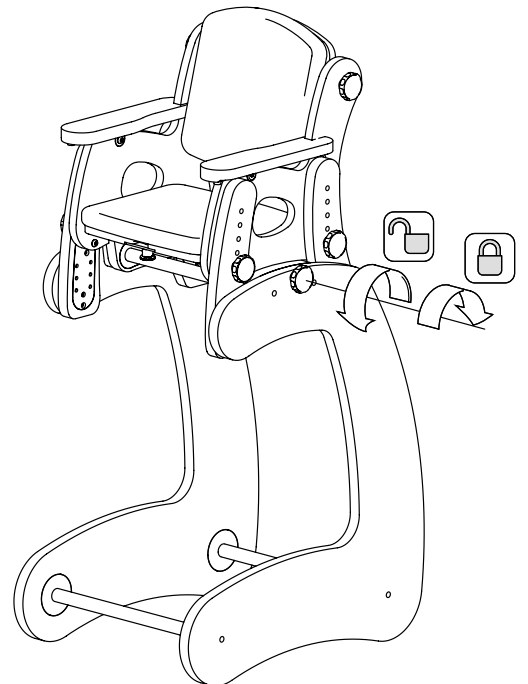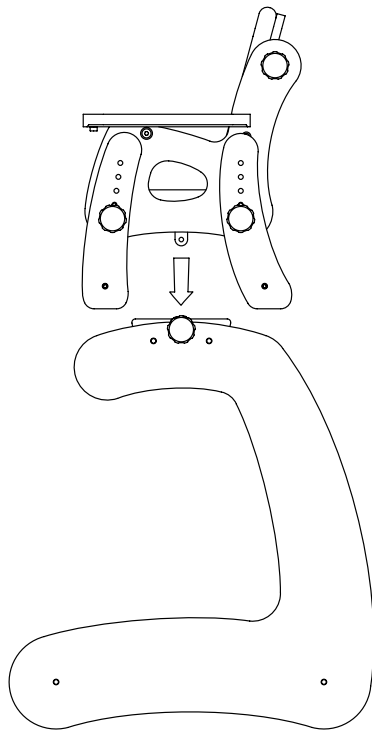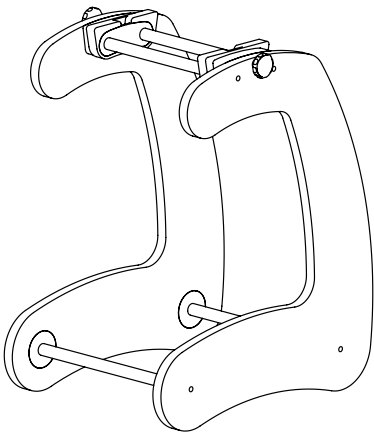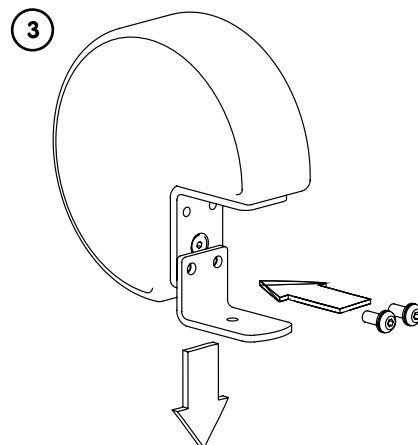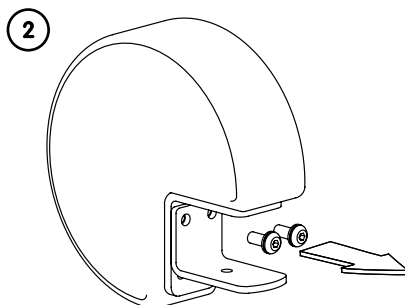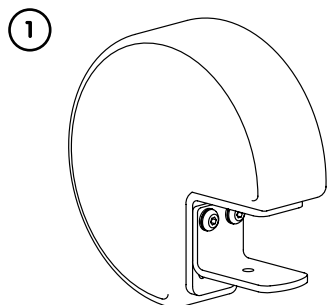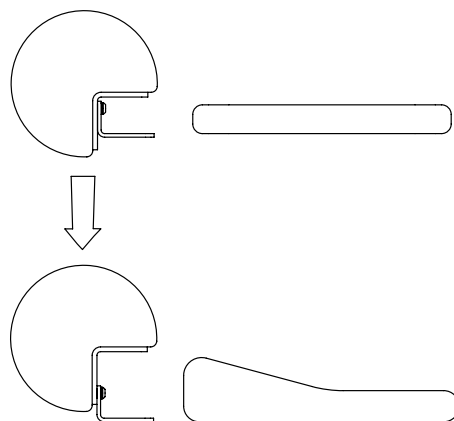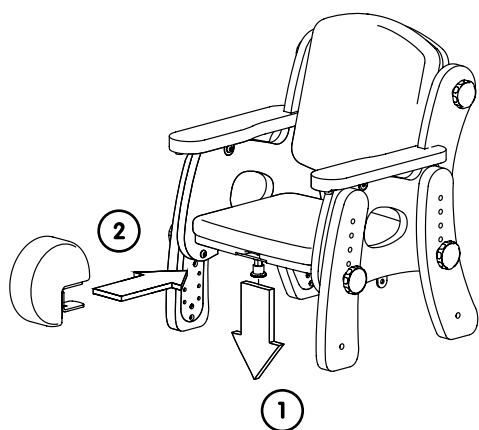
High legs  
Hochstuhlgestell  
Base d'appoggio alta  
Support rehaussant  
Patas altas




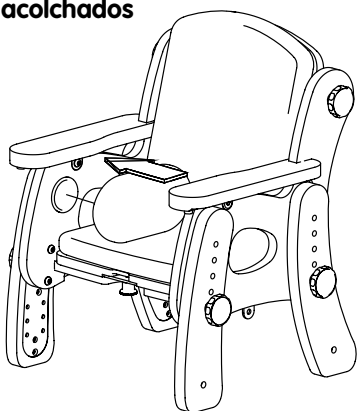
Side pads  
Beckenpolster zur Sitzbreitenreduzierung  
Imbottiture laterali  
Cales latérales  
Laterales acolchados

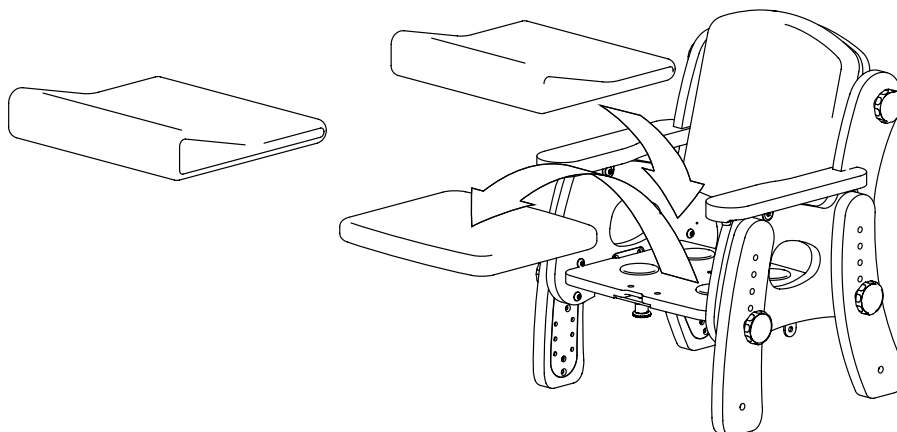Ramped cushion / Keilkissen / Cuscino a cuneo  
Coussin tampon / Cojín ergonómico

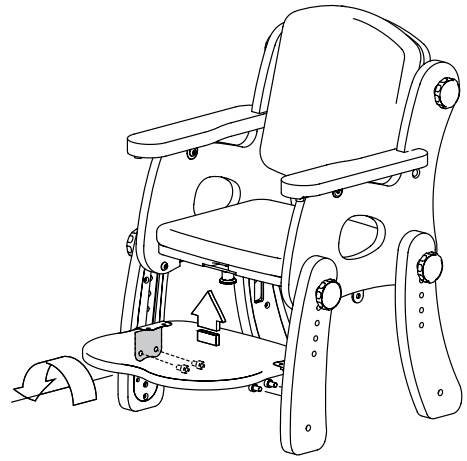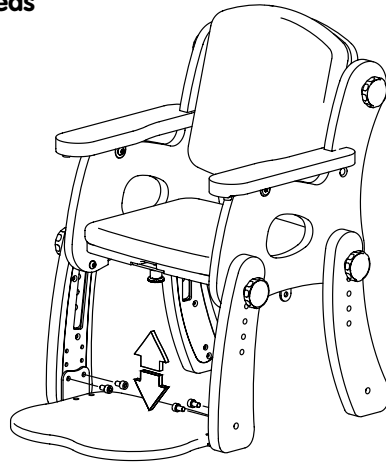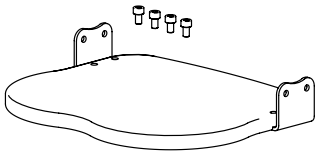**Footrest / Höhenverstellbares Fußbrett**  
**Pedana poggiapiedi / Repose-pieds**  
**Reposapiés**

**Tray and tray pad / Therapietisch und gepolsterte Tischauflage / Tavolino avvolgente e imbottitura per tavolino /**  
**Plateau et rebord de plateau / Bandeja y acolchado para bandeja**


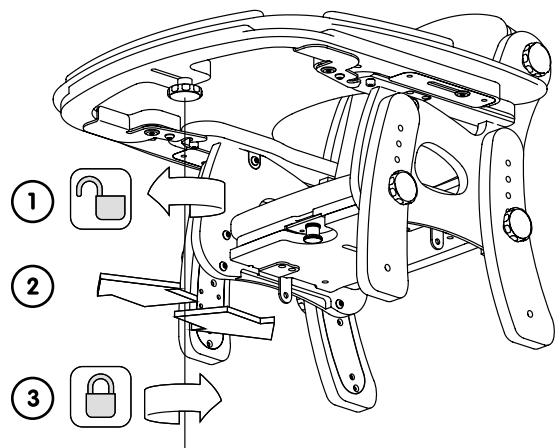
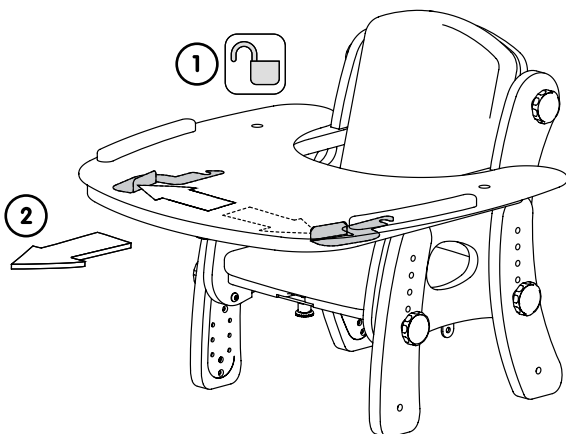
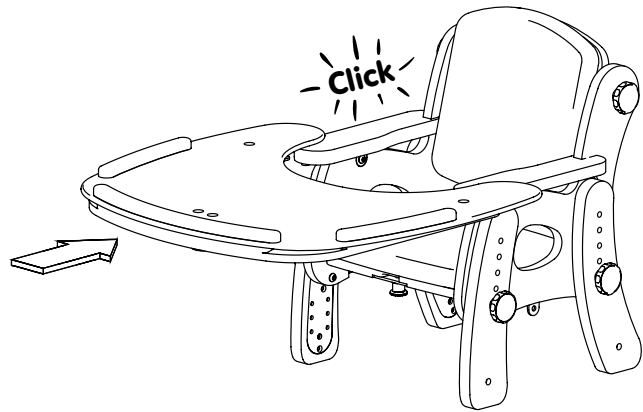
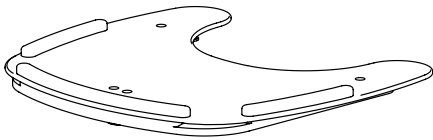
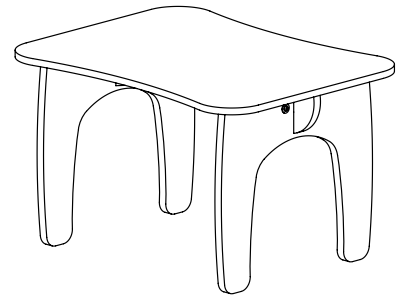
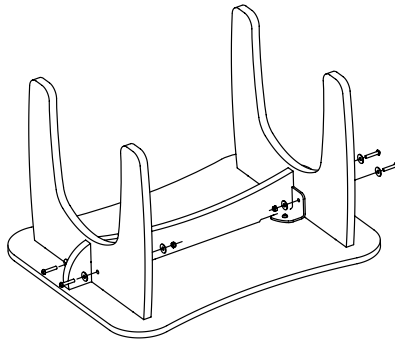
Table / Tisch / Tavolo / Table / Mesa

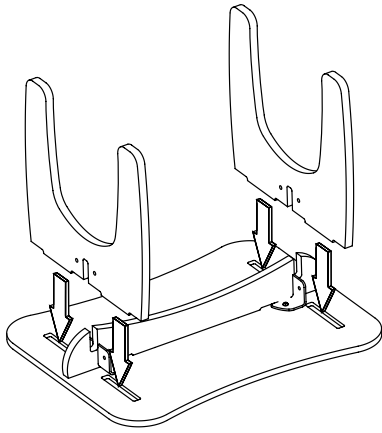The Pal Classroom seat has been designed for children with mild to moderate needs to help improve their posture and attention. This manual shows you how to adjust the seat and fix any accessories to it. It also contains instructions on safety and maintenance.

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#### **Contents**

1. Intended use
2. Declaration of conformity
3. Terms of warranty
4. Product history record
5. Product training record
6. Safety information
7. How to adjust seat and attach accessories (overleaf)
8. Cleaning & care information
9. Daily product inspection
10. Reissuing Leckey products

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## 1 Intended Use

The Leckey Pal Seat has been designed for use in the nursery or classroom by children aged 1-8 years old with mild to moderate postural needs. It is designed to give improved stability which helps reduce fatigue and allows the child longer periods of concentration and fine motor activity. It comes in 3 sizes, size one having a maximum user weight of 30kg (66lbs), size two 35kg (77lbs) and the size three 40kg (88lbs).

## 2 Declaration of Conformity

James Leckey Design Ltd. as manufacturer with sole responsibility declares that the Pal Classroom Seat conforms to the requirements of the 93/42/ EEC Guidelines and EN12182 Technical aids for disabled persons, general requirements and test methods.

## 3 Terms of Warranty

The warranty applies only when the product is used according to the specified conditions and for the intended purposes, following all manufacturer's recommendations (also see general terms of sales, delivery and payment). A two year warranty is provided on all Leckey manufactured products and components.

## 4 Product History Record

Your Leckey product is classified as a Class 1 Medical device and we recommend that a written record is maintained to provide details of all setups, reissue inspections and annual inspections of this product.

## 5 Product Training Record (Parents, Teachers & Carers)

Your Leckey product is a prescribed Class 1 Medical Device and as such Leckey recommend that parents, teachers and carers using the equipment should be made aware of the following sections of this user manual:

Section 6.

Safety information

Section 7.

How to adjust Seat and attach Accessories

Section 8.

Cleaning & care information

Section 9.

Daily product inspection and servicing

Leckey recommend that a written record is maintained of all those who have been trained in the correct use of this product.

## 6 Safety information

1. Always read instructions fully before use.

2. Users should not be left unattended at any time whilst using Leckey equipment.

3. Only use Leckey approved components with your product. Never modify the product in any way. Failure to follow instructions may put the user or carer at risk and will invalidate the warranty on the product.

4. If in any doubt to the continued safe use of your product or if any parts should fail, please cease using the product and contact our customer services department or local dealer as soon as possible.

5. Carry out all positional adjustments and ensure that they are securely fastened before you put the user into this product.

6. To attach some accessories may require the use of a tool which is provided with the relevant accessory. Keep all tools out of the reach of children.

7. When the product is stationary ensure that all castors are locked and facing away from the base as this will improve product stability.

8. Never leave the product on a sloping surface, greater than 5 degrees.

9. The product contains components which could present a choking hazard to small children. Always check that locking knobs and bolts within the child's reach are tightened and secure at all times.

10. Leckey products comply with fire safety regulations in accordance with EN12182. However the product contains wooden and plastic components and therefore should be kept away from all direct sources of heat including naked flames, cigarettes, electric and gas heaters.

11. Do not place hot items greater than 40 degrees Celsius (104 degrees Fahrenheit) on the tray.

12. Clean the product regularly. Do not use abrasive cleaners. Carry out maintenance checks on a regular basis to ensure your product is in good working condition.

13. The product is designed for indoor use and when not in use should be stored in a dry place that is not subjected to extremes of temperature. The safe operating temperature range of the product is +5 to +40 degrees Celsius (41 to 104 degrees Fahrenheit).

14. If a footplate is used, self transfer into a the seat is only recommended when the footplate is at its lowest setting, i.e. at floor level. If a footplate is used in any other setting we recommend a risk assessment is carried out (and reviewed as the child grows) and a carer holds the back of the seat while the child transfers in and out of the seat. For additional stability the anti-rocker skis can be attached to the legs.

15. If rocker skis are used, there is potential for the child to trap their foot under the skis or footplate (if attached) during self transfer. We recommend a risk assessment is carried out (and reviewed as the child grows) and a carer holds the back of the seat while the child transfers in and out of the seat.

16. If the seat is used on hard, slippery surfaces the seat may need to be stabilised by a carer while the child transfers in and out of the seat.

## 7. How to adjust seat and attach accessories (overleaf).

## 8. Cleaning & Care Information

Wipe clean with warm soapy water followed by a clean water rinse. Do not use strong detergents, abrasive products, organic solvents or dry cleaning fluids. Heavily ingrained soiling may be removed by scrubbing with a soft brush.

Where disinfecting is required use mild sanitizer wipes or disinfectant solution mix (1 part bleach to 10 parts water) from brands such as: GERMWARFARE ® & Choroclean ®.

Most stains, e.g. food and drinks or urine will not be absorbed by the vinyl and can be wiped off without any permanent damage, but difficult substances such as ball point pen ink should be cleaned off immediately to avoid permanent staining.

Check all wooden components regularly to ensure there are no cracks or splitters.

## 9. Daily Product Inspection & Servicing

We recommend that daily visual checks of the equipment are carried out to ensure the product is safe for use. The recommended daily checks are detailed below.

1. Ensure all adjustment knobs are in place and secure.

2. Check all upholstery for signs of wear and tear.

3. Check all castors are moving freely and locked securely.

4. Check all Velcro strips and brush off fluff to ensure components secure firmly.

If in any doubt to the continued safe use of your Leckey product or if any parts should fail, please cease using the product and contact our customer services department or your local dealer as soon as possible.

In the UK & ROI please contact the Leckey Service Centre on UK 0800 318265 or ROI 1800 626020 and our customer service department will be delighted to assist you with your servicing requirements.

All international service enquiries should be directed to the appropriate Leckey distributor who will be delighted to assist you. For further information on Leckey distributors please visit our website at [www.leckey.com](http://www.leckey.com).

## 10. Reissuing Leckey Products

Before reissuing a product we recommend that the therapist prescribing the product has carried out an equipment compatibility check for the new user and has ensured that the product being reissued contains no modifications or special attachments.

A detailed inspection should be carried on the product prior to reissuing as detailed under Section 9.

Ensure the product has been cleaned thoroughly in accordance with section 8 of this manual.

Ensure a copy of the user manual is supplied with the product. A copy can be downloaded from our website [www.leckey.com](http://www.leckey.com).

Leckey recommend that a written record is maintained of all product inspections carried out during the reissue of the product.

If in any doubt to the continued safe use of your Leckey product or if any parts should fail, please cease using the product and contact our customer service department or your local dealer as soon as possible.

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