



Instruction for use Shower Chair/Commode

1/  = 318kg

Maximum Safe Working load 318kg

2/ 

Do NOT use to transport patients

3/ For use with sanitary pail supplied. **Do not** use portable sanitary pans compliant to BS 2588-2:1992.

4/ Clean using detergents compatible with disinfectants

5/ Disinfect between patients or after storage with BS EN 1276:1997 & BS EN 13727:2003, compliant disinfectants

6/ Do not use if damaged or hygiene levels have not been maintained

Nightingale Care Beds Ltd
Unit 1 Ash Road South
Wrexham Industrial Estate
Wrexham
LL13 9UG

Tel: 01978 660810
Fax: 01978 660828

