




## Olympia Bariatric

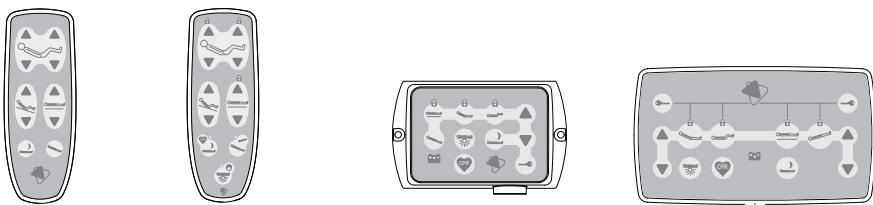

Manipulation	Hand switch	Hand switch HD	Mini-nurse box ACC	Nurse box ACO
BACKREST				
KNEEREST				
RELAX POSITION				
HIGH/LOW				
SLEEPING POSITION				
REVERSE-TREND.				
Emergency operations				
TREND.				
CPR				

Not active when a function has been locked.

Irrespective of any blocked functions.

Lock/Unlock			

Nightlight under the bed on/off




The available bed functionality is in line with the chosen and ordered configuration.






## Olympia Bariatric


Quick release

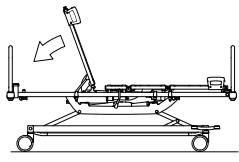

Lever

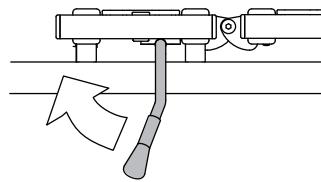


Bed extension

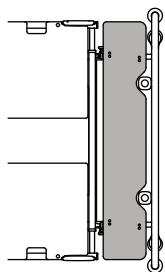

1. lift the footrest
2. pull out both pull knobs
3. pull out the bed extension
4. lock the buttons
5. place the removable foot extension

Mattress platform widening

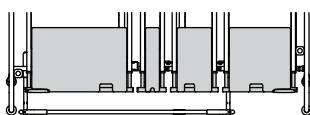




1. pull out both pull knobs of the sideguard
2. position the side rail and release the knobs
3. make sure the sideguard is locked
4. pull out the pull knobs of the mattress platform sections
5. position the telescopic sections and release the knobs
6. put the filling mattresses on both sides of the main mattress

Central brake mechanism

1 directional castor, fixed

4 castors unbraked

4 castors braked

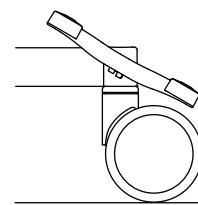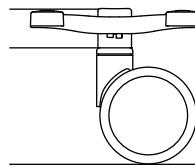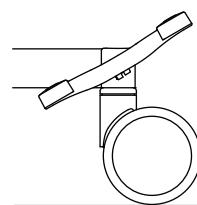

The available bed functionality is in line with the chosen and ordered configuration.

