# ReadyLett

System**RoMedic**

Instructions for use – English

Bruksanvisning – Svenska

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Návod na použitie – Slovensky

## REF

11629

11634

11639

11641

SWL: 300 kg/ 660 lbs

## Table of contents

English .....	3
Svenska .....	5
Norsk .....	7
Dansk .....	9
Deutsch .....	11
Français .....	13
Český .....	15
Slovensky .....	17
Symbols .....	18

ReadyLett is a slide sheet that facilitates positioning and turning in bed. It is bedded in the bed like a normal sheet. ReadyLett is available in several sizes and models.

## Functional Inspection

### Visual inspection

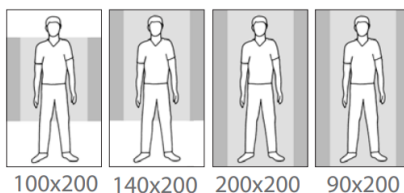
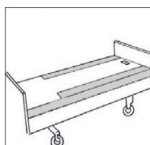
Check the condition and function of the assistive device regularly.  
Always inspect the product after laundering.  
Check to ensure that seams and material are free from damage.  
Check to ensure that fabric is not worn or faded.  
If there are signs of wear, the product must be discarded.

## Always read the user manual

Always read the manuals for all assistive devices used during a transfer.  
Keep the manual where it is accessible to users of the product.  
Do not leave the patient unattended during a transfer situation.

## Placement


Place ReadyLett with drawstring with the smooth surface in the middle of the bed. Ensure that the mattress cover's rough surfaces or evenly distributed on both sides of the bed. These rough surfaces prevent the patient from sliding out of bed.

## Use

ReadyLett can be bedded in cases where the patient is able to turn himself/herself, but finds it easier with less friction.

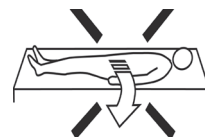**Note!** Where there is a risk of the patient sliding out of bed, use bed rails




## Accessories

### SystemRoMedic

The transfer assistive products can be used either independently or as a complement to products in the SystemRoMedic range.

WendyLett2Way and WendyLett4Way are two different draw sheets that can be used together with a slide sheet.

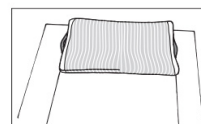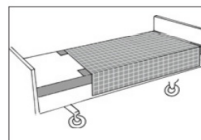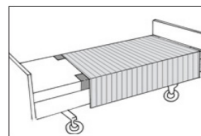WendyLett2Way is used for lateral repositioning and WendyLett4Way is used for both lateral repositioning and repositioning higher up or lower down in bed.

## Size

11629 ReadyLett, 60 cm/24" sliding surface (bed 80/90 cm - 32"/36") 100 x 200/40" x 80"

11634 ReadyLett, 60 cm/24" sliding surface (bed 80/90 cm - 32"/36") 140 x 200/56" x 80"

11639 ReadyLett, 70 cm/28" sliding surface (bed 105 cm/42") 200 x 200/80" x 80"

11641 ReadyLett, 70 cm/28" sliding surface (bed 90 cm/36"), tailored with drawstring 90 x 200 /36" x 80"

## Material

Polyester, Cotton, Satin

## Care of the product

Read the product label.

Do not use rinsing agent.

To prolong product life, avoid tumbledrying

Contact your local distributor if you have any questions about the product and its use.

See [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com) for a complete list of distributors.

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## Symbols

	This product complies with the requirements of the Medical Device Regulation 2017/745		Visual Inspection
	Medical Device		Read the manual
	Caution		Product Code
	Manufacturer information		Batch Code

## Simple solutions for great results

SystemRoMedic® is the name of Direct Healthcare Group's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic® is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

## Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic® is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic® offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

## Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website;  
[www.directhealthcaregroup.com](http://www.directhealthcaregroup.com)

## For questions about the product and its use

Please contact your local Direct Healthcare Group and SystemRoMedic® representative. A complete list of all our partners with their contact details can be found on our website; [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com).

Moving Health Forward

Medical Device Class I. The product complies with the requirements of the Medical Devices Regulation 2017/745.

Direct Healthcare Group Sverige AB  
Torshamnsgatan 35,  
SE-164 40 Kista, Sweden

Tel: +46 (0)8-557 62 200  
[info@directhealthcaregroup.com](mailto:info@directhealthcaregroup.com)  
[www.directhealthcaregroup.com](http://www.directhealthcaregroup.com)