# ReadySheet

System**RoMedic**

Instructions for use – English

Bruksanvisning – Svenska

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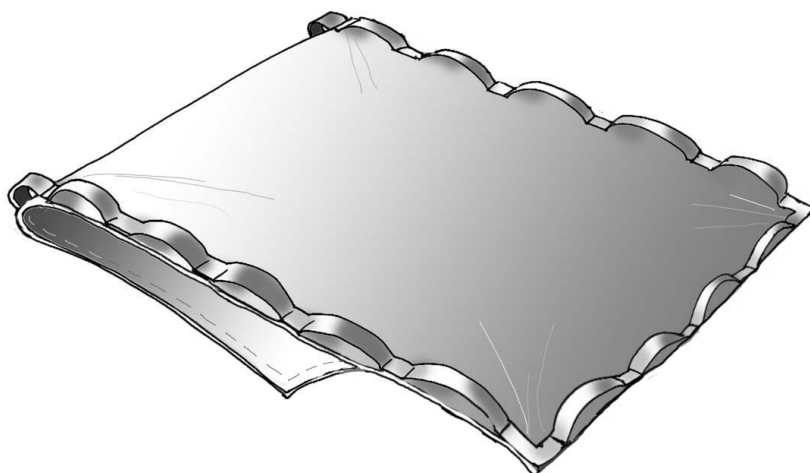
**REF**

11412

11430

**SWL: 180 kg/ 400 lbs**

## Table of contents

English .....	3
Svenska .....	5
Norsk .....	7
Dansk .....	9
Suomi .....	11
Deutsch .....	13
Nederlands .....	15
Français .....	17
Italiano .....	19
Español .....	21
Polski .....	23
Size .....	25
Symbols .....	25

ReadySheet is a simple and versatile assistive device for application of lifting slings and for repositioning in one location, for example, for turning or moving the user higher up in bed. ReadySheet is available in two variants; one open and one tubular, and in many sizes. All variants have handles for easier positioning.

ReadySheet is made of ULF (Ultra Low Friction), Direct Healthcare Group's own unique sliding material. By reducing the friction under the user, safe and easy repositioning, turning and repositioning higher up in bed can be performed with very little effort and without the need for lifting. It is comfortable for the user and it saves the caregiver's back and shoulders. ReadySheet is removed after use

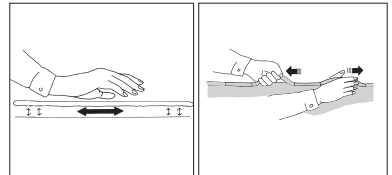
## Functional inspection

### Visual inspection

Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.

### Mechanical load

Test the friction of the material by pressing it against the underlying surface and rolling it back and forth (1). Note whether it still glides as easily as it did. Compare with a new product if possible.

Pull on the handles to check seams and strength (2).

## Always read the user manual

Always read the manuals for all assistive devices used during a transfer.

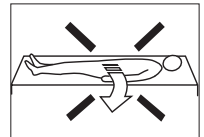
Keep the user manual where it is accessible to users of the product.

Always make sure that you have the right version of the user manual.

The most recent editions of user manuals are available for downloading from our website, [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com).

## Note


Always remove the product after a transfer is completed. ReadySheet has a very low-friction surface. Where there is a risk of the user sliding out of bed, use bed rails.

## Placement/removal

### Open

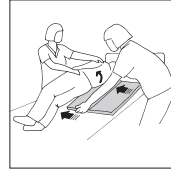
ReadySheet is placed under the user with the handles facing downward. Lean the user to one side, make use of the sheet, and place the folded product under the user (1). Lean the user in the opposite direction and pull the ReadySheet out on the other side. Remove the ReadySheet by pulling it upward and out from underneath (2).

### Tubular

The tubular version can be placed according to the above description, or it can be rolled up and then folded out under the user (3). With the user lying flat, place the roll under the knees. With one caregiver working on either side of the bed, unroll stepwise until the entire roll is fully rolled out under the user.

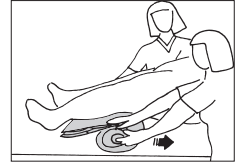
Remove the ReadySheet by pulling it out from underneath at a slight upward angle.

## Turning

### Open

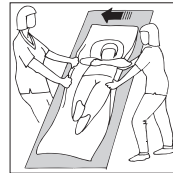For turning in bed, a ReadyLett is bedded around the mattress for even better glide. Place the ReadySheet under the user and push/pull the user laterally (1) before turning (2).

## Higher up in bed

### Open

For repositioning higher up in bed, a ReadyLett is bedded around the mattress. Place the ReadySheet under the user and pull the user higher up in bed.

### Tubular

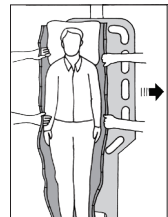
When repositioning a user higher up in bed using a tubular ReadySheet, place the sheet under the user as described earlier. Then, grasp the handles on either side and pull the user higher up in bed.

## Transfer between bed and gurney

### Open

For transfers between bed and gurney, use EasyGlide 170x59 as a bridge. Place the ReadySheet under the user in bed and pull/push the user over to the gurney. Remember to lock the wheels on both the bed and the gurney before starting the transfer.

## Material

Polyamid, polypropylene

## Care of the product

Read the product label. If disinfection is needed 70 % ethanol, 45% isopropanol or similar should be used.

Do not use rinsing agent.

To prolong product life, avoid tumble-drying.

Contact your local distributor if you have any questions about the product and its use.

See [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com) for a complete list of distributors.

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## Symbols

	This product complies with the requirements of the Medical Device Regulation 2017/745		Visual Inspection
	Medical Device		Read the manual
	Caution		Product Code
	Manufacturer information		Batch Code

# SystemRoMedic™

## Simple solutions for great results

SystemRoMedic® is the name of Direct Healthcare Group's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic® is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

## Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic® is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic® offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

## Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website;  
[www.directhealthcaregroup.com](http://www.directhealthcaregroup.com)

## For questions about the product and its use

Please contact your local Direct Healthcare Group and SystemRoMedic® representative. A complete list of all our partners with their contact details can be found on our website; [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com).

Moving Health Forward

Direct Healthcare Group Sverige AB  
Torshamnsgatan 35,  
SE-164 40 Kista, Sweden

Tel: +46 (0)8-557 62 200  
[info@directhealthcaregroup.com](mailto:info@directhealthcaregroup.com)  
[www.directhealthcaregroup.com](http://www.directhealthcaregroup.com)

Medical Device Class I. The product complies with the requirements of the Medical Devices Regulation 2017/745.