ReadySlide

SystemRoMedic™

Instructions for use – English

Bruksanvisning – Svenska

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

Návod na použitie - Slovenský

Large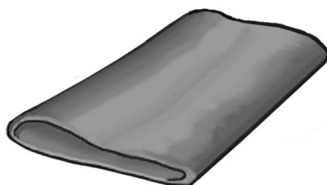


Medium

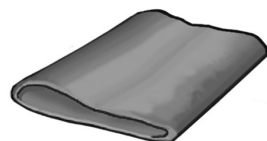Small

REF

11210
11215
11230
11235
11236
11240
11241
11250

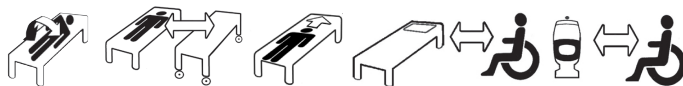Max. patient weight: 300 kg/ 660 lbs
Max. patient weight 11215: 150 kg/ 330 lbs

Table of contents

English	3
Svenska	6
Norsk	9
Dansk	12
Suomi	15
Deutsch	18
Français	21
Polski	24
Český	27
Slovenský	30

ReadySlide

SystemRoMedic™

Instructions for use - English

ReadySlide is made of ULF (Ultra Low Friction), Direct Healthcare Group's own unique sliding material. By reducing the friction under the user, safe and easy transfers can be performed with very little effort and without the need for lifting. It is comfortable for the user and it saves the caregiver's back and shoulders.

ReadySlide is available in different sizes. The larger sliding mats are used for transfers between bed and gurney or for turning in bed. The smaller mats can be used for turning in bed, for repositioning higher up in bed or for transfers between bed and wheelchair/toilet. The largest size is available both with and without handles at the sides.


Functional inspection

Visual inspection

Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.

Mechanical load

Test the friction of the material by pressing it against the underlying surface and rolling it back and forth (1). Note whether it still glides as easily as it did. Compare with a new product if possible.

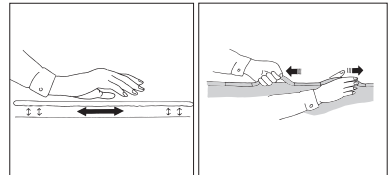Pull on the handles to check seams and strength (2).

Always read the manual

Always read the manuals for all assistive devices used during a transfer.

Keep the manual where it is accessible to users of the product.

Do not leave the patient unattended during a transfer situation.

Note


Remember to remove the product after use, as it has a very low-friction surface.

NOTE! Always lock the wheels on the bed, gurney and wheelchair.

Placement/removal

To place ReadySlide under the patient, lean the patient in the opposite direction. Make use of the sheet to lean the patient. Push ReadySlide in under the patient as far as possible (1). To remove, pull on the lower end of ReadySlide(2).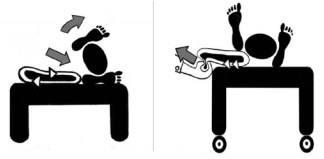


Turning

Position ReadySlide, and then turn the patient by rolling him/her over on one side or by pulling on the sheet. The patient slides easily on ReadySlide. Remove ReadySlide.

Bed to gurney

Position ReadySlide, move the gurney up beside the bed. Remember to lock the wheels. Push or pull the patient over to the gurney. Remove ReadySlide.

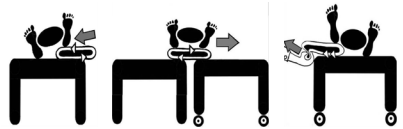Higher up in bed

Place ReadySlide under the patient's shoulders. Ask the patient to bend the knees, lift the seat and push up. Also works well with the assistance of a caregiver. Remove ReadySlide.

Transfer bed/wheelchair/toilet

Lock the wheelchair wheels and place ReadySlide under the patient's seat so that it is both on the bed and the wheelchair. Ask the patient to grasp the armrests and, using his or her own weight, slide over into the wheelchair. Raise the feet from the floor with the aid of a foot stool to reduce friction under the thighs. This transfer can also be performed by the caregiver with the aid of a care belt for added safety. Remove ReadySlide after completing the transfer.

For transfers between wheelchair and toilet, place ReadySlide as described above. By shifting his/her weight, the patients slides over to the toilet/wheelchair. Use a care belt if the patient requires the assistance of a caregiver. A foot stool also facilitates the transfer. Remove ReadySlide after completing the transfer.

Material

Polyamid

Care of the product

Read the product label. If disinfection is needed 70 % ethanol, 45% isopropanol or similar should be used.

Do not use rinsing agent.

To prolong product life, avoid tumble-drying.

Contact your local distributor if you have any questions about the product and its use.

See www.directhealthcaregroup.com for a complete list of distributors. Always make sure that you have the right version of the manual.

The most recent editions of manuals are available for downloading from our website, www.directhealthcaregroup.com.

Size

Art.no	Size cm/inch
11210	195 x 70 / 76,7x 27,5
11215	195 x 70 / 76,7x 27,5
11230	70 x 110 / 27,5 x 43,3
11235	110 x 90 / 43,3 x 35,4
11236	110 x 110 / 43,3 x 43,3
11240	60 x 50 / 23,6 x 19,7
11241	130 x 90 / 51,2 x 35,4
11250	70 x 70 / 27,5 x 27,5

Symbols

	This product complies with the requirements of the Medical Device Regulation 2017/745		Visual Inspection
	Medical Device		Read the manual
	Caution		Product Code
	Legal manufacturer Direct Healthcare Group Sweden AB		Batch Code

How to report a serious incident

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the MHRA or another competent authority of the country in which the user and/or patient is established.

UK

info@directhealthcaregroup.com

T: +44 (0) 800 043 0881

F: +44 (0) 845 459 9832

Other:

info.export@directhealthcaregroup.com

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T: +46 8-557 62 200

Simple solutions for great results

SystemRoMedic® is the name of Direct Healthcare Group's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic® is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic® is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic® offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website;
www.directhealthcaregroup.com

For questions about the product and its use

Please contact your local Direct Healthcare Group and SystemRoMedic® representative. A complete list of all our partners with their contact details can be found on our website; www.directhealthcaregroup.com.

Moving Health Forward

Direct Healthcare Group Sverige AB
Torshamnsgatan 35,
SE-164 40 Kista, Sweden

Tel: +46 (0)8-557 62 200
info@directhealthcaregroup.com
www.directhealthcaregroup.com

Medical Device Class I. The product complies with the requirements of the Medical Devices Regulation 2017/745.