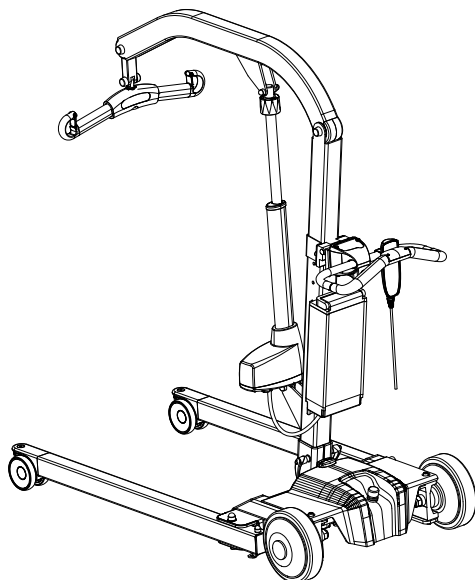
EvaDrive450/600

SystemRoMedic™

CE
Servicemanual - Svenska

SWL: 205 kg/450 lbs
270 kg/600 lbs

REF

60900002

60900003

Säkerhetskontroll

Visuell inspektion

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

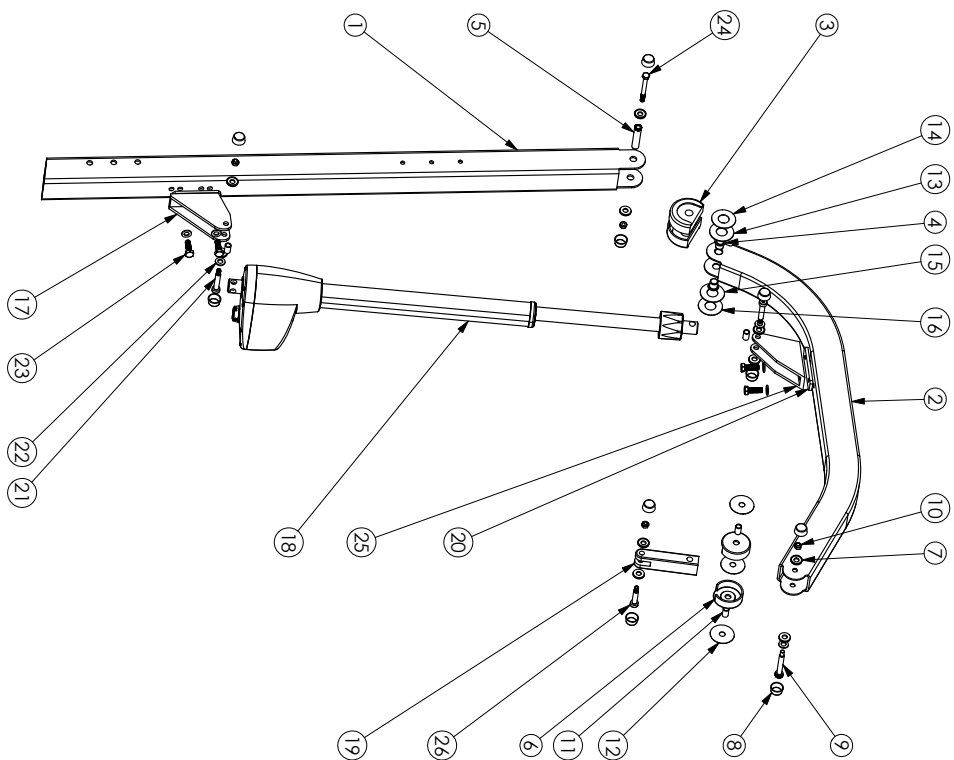
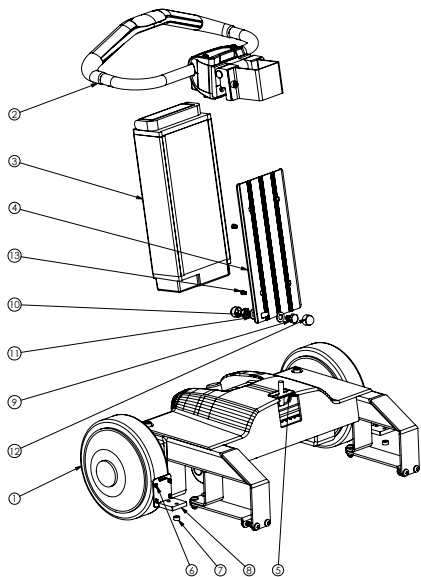
ITEM NO.	TITLE	QTY.
1	Eva Drive 450 base assy	1
2	EvaDrive Hinge assembly	1
3	Battery EvaDrive	1
4	Battery Plate EvaDrive	1
5	ED cables	1
6	Cover leg	2
7	Spacer 8x12x6	2
8	Converter Nut EvaDrive	2
9	Screw M6S DIN931 8.8 M10x16H fzb	1
10	Lock nut DIN 985 CL6 M10 FZB	1
11	Washer DIN125 10.5x20x2 fzb	2
12	Nut cover MSM10	2
13	Screw MLC6S DIN7984 8.8 M5x8	2

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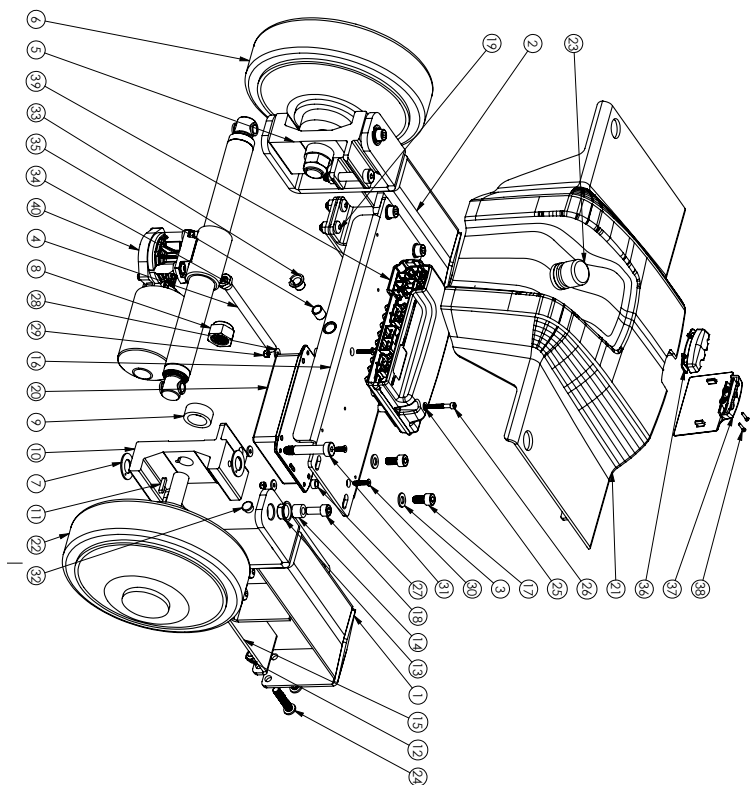REFNO	TITLE	QTY
1	End Drive leg right welded	1
2	End Drive leg left welded	1
3	Washer DIN125 8.4X16.1 1.0	4
4	End drive actuator 280777010	1
5	End drive wheel bracket left	1
6	End motor end wheel left	1
7	Sliding bearing COB810 20H16	4
8	Lock nut DIN934 M16 1.0	2
9	End drive wheel distance	2
10	End drive wheel bracket right	1
11	Frontal key DIN 6885 A KR19 4x20	2
12	Lock nut DIN934 M8 1.0	10
13	Sliding with tongue FITE COB101F 1000 10mm/rod	4
14	End drive sleeve	4
15	End drive tension bracket	2
16	End drive support welded	1
17	Screw M6x8 DIN913 2.0 8.0x16.1 1.0	4
18	Screw M6x8 DIN913 2.0 8.0x16.0 1.0	4
19	Screw M8x12 DIN913 1.0 8.0x16.0 1.0	6
20	Bracket lid enclosure - horizontal 109101010	1
21	End Drive cover	1
22	Main motor end wheel right	1
23	Emergency stop button CB111 08.11 (A 881)	1
24	Screw M6x12 DIN913 1.0 8.0x16.0 1.0	2
25	Washer DIN125 4.3x16.0 8.0 1.0	2
26	Screw M6x8 DIN913 2.0 8.0x16.0 1.0	2
27	Spacer 4.2x10x16.4x4.5 SH11V001 400 000 02	4
28	Washer DIN 2746 4.2x12.1 1.0 1.0	4
29	Lock nut DIN934 M4 1.0	4
30	Screw M6x5 DIN913 1.0 8.0x16.0 1.0	4
31	Shoulder bolt ISO1209 12.9 10x16x10	2
32	Ring Acuator part 1349 G	2
33	Flange bearing COB810 - 10 20 (16mm/rod)	1
34	Nut M16x1.5x16.0x1.0 DIN934 M16 1.0	1
35	Sliding bearing COB810 10 12	1
36	End Drive Cable entry top	1
37	End Drive Cable entry front	1
38	Screw for thermoplastic part 25x12x20 (RUF-48)	2
39	Control box VZ	1
40	Actuator LAD 2x9x 35mm	1

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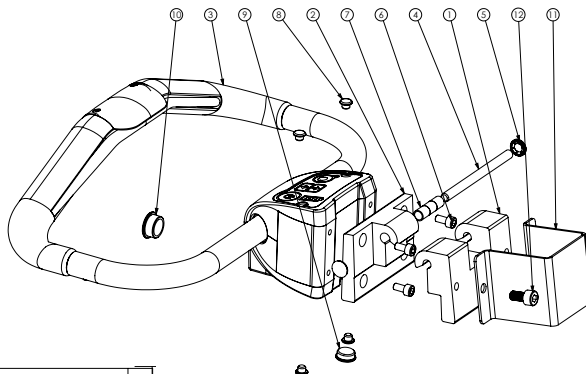ITEM NO.	TITLE	QTY.
1	Hinge outer	1
2	Hinge inner	1
3	Handle EvaDrive	1
4	Hinge axis	1
5	Washer starlock dome 8x5,7x16	2
6	MC6S DIN 912 8,8 M6x12 FZB	4
7	Hinge bushing 8x10x12	3
8	Cover 7mm grey	4
9	Cover 16mm grey	1
10	Cover 20mm grey	1
11	Handle Bracket Eva Drive	1
12	Screw MC6S DIN912 8,8 M8x16 fzb	2

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
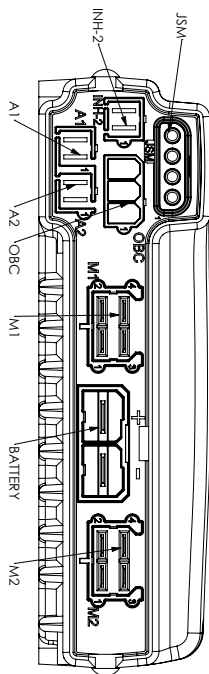
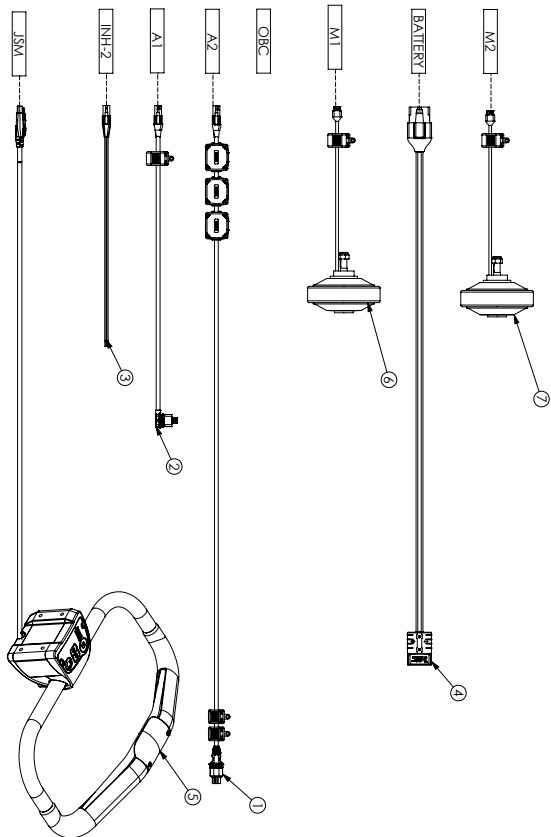
FIG DRIVES VR2 POWER MODULE CONNECTIONS

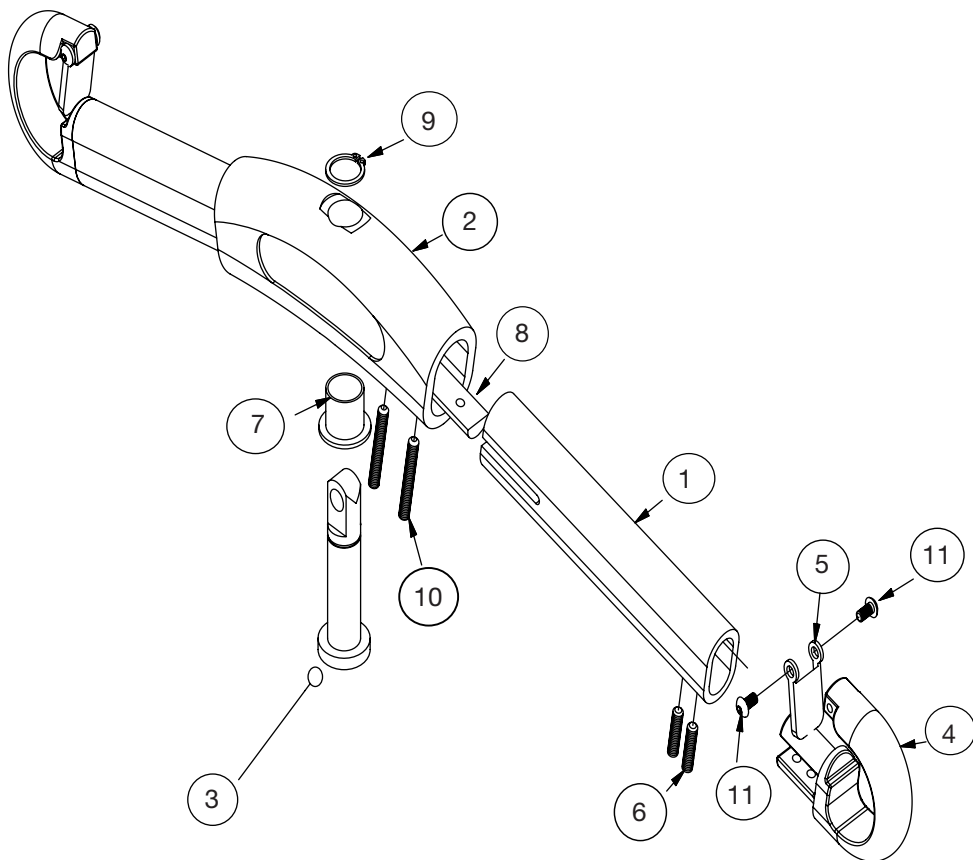ITEM NO.	TITLE	QTY.
1	ExDrive actuator cable long straight multi-tilt	1
2	ExDrive actuator cable short angled micro-fill	1
3	ExDrive Emergency/Stop Cable	1
4	ExDrive battery cable	1
5	ExDrive Handle	1
6	Hubmotor with connector right	1
7	Hubmotor with connector left	1

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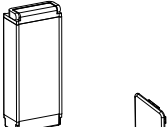
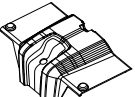
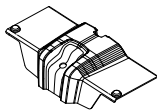
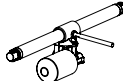
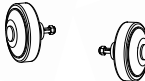
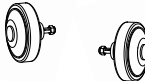
Appendix 1: Error codes

Trip Types and Their Possible Causes

Once the trip type has been established, refer to the relevant section below for further information.

No of battery LEDES flashing	ServiceLight flashing?	Type		Code	Description
4	Y	9		1505	Left Solenoid Brake Trip (1.9)
4		-		1320	Refer to section (1.16)
	Y	9		1506	Right Solenoid Brake Trip (1.9)
5	N	10		1600	High Battery Voltage (1.10)
		Charging		1000	Refer to section (1.6)
3	N	6		10000	Refer to section (1.13)
		Charging		100000	Refer to section (1.14)
0	Y	1		2C00	Low Battery Voltage (1.1)
		-		2C02	Low Battery Lockout (1.1)
		User		2F00	Refer to sections (1.7 & 1.11)
1	N	2		3B00	Left Motor Disconnected (1.2)
2	N	4		3C00	Right Motor Disconnected (1.4)
1	Y	3		3D00 -01	Left Motor Wiring Trip (1.3)
2	Y	5		3E00 -01	Right Motor Wiring Trip (1.5)
3	Y	7 + S		5400	Communications Trip (1.12)
4	N	8		4401	Control System Trip (1.8)
		A Only		7A03	Actuator Motor Wiring Trip (1.15)
3	Y	7		7100	Control handle Trip (1.7)
		User		7147	Dual Joystick Displaced Refer to section (1.11)
4	N	7 or 8		Other	Possible Control System Trip (1.7 & 1.8)
3	Y	8 or 8		Other	Possible Control System Trip (1.7 & 1.8)

1.1 Trip Type 1 - Low Battery Voltage

This occurs when the control system detects that the battery voltage has fallen below 16V. Check the condition of the batteries and the connections to the control system. If the trip is still present after the batteries and connections have been checked, then the Power Module may be defective. In the case of 2C02 the Control System is making a log of the times that the Low Battery Lockout has been initiated.

1.2 Trip Type 2 – Left Motor Disconnected

This occurs when the control system detects that the left motor has become disconnected. Check the left motor, motor connectors and wiring. If the trip is still present after the above checks have been made, then the Power Module may be defective. The VR2 control system may be programmed to exchange the left and right motor outputs. In this instance, this section will refer to the right motor. Consult Handicare for more details.

1.3 Trip Type 3 – Left Motor Wiring Trip

This occurs when the control system detects a fault in the wiring to the left motor, in particular if a motor connection has short-circuited to a battery connection. Check the left motor connectors and wiring. If the trip is still present after the above checks have been made, then the Power Module may be defective. The VR2 control system may be programmed to exchange the left and right motor outputs. In this instance, this section will refer to the right motor. Consult Handicare for more details.

1.4 Trip Type 4 – Right Motor Disconnected

This occurs when the control system detects that the right motor has become disconnected. Check the right motor, motor connectors and wiring. If the trip is still present after the above checks have been made, then the Power Module may be defective. The VR2 control system may be programmed to exchange the left and right motor outputs. In this instance, this section will refer to the left motor. Consult Handicare for more details.

1.5 Trip Type 5 - Right Motor Wiring Trip

This occurs when the control system detects a fault in the wiring to the right motor, in particular if a motor connection has short-circuited to a battery connection. Check the right motor connectors and wiring. If the trip is still present after the above checks have been made, then the Power Module may be defective. The VR2 control system may be programmed to exchange the left and right motor outputs. In this instance, this section will refer to the left motor. Consult Handicare for more details.

1.6 Trip Type 6 – Charger Connected

This occurs when the control system detects that an off-board charger is connected. Check that the battery charger is disconnected. If the trip is still present after the charger has been disconnected then the Control handle Module may be defective.

1.7 Trip Type 7 – Possible Control handle Trip

This occurs if the control system (PM) detects a problem within its own Force sensing control handle, or there is a communications error between the Control handle Module and Power Module. The control handle can only be replaced by a person authorized by Handicare.

7100 Loss of communications to the control handle, check the control handle cable.

7103 Internal trip, if you have authorization check the control handle internal connections and mating sockets. Ensure the cable is connected correctly to the PCB. If the trip is still present after the appropriate checks have been made then the Control handle Module may be defective.

1.8 Trip Type 8 - Possible Control System Trip

This occurs if the control system detects a problem within itself. The control system can only be repaired by an authorized person.

1.9 Trip Type 9 - Solenoid Brake Trip

This occurs when the control system detects a problem in the solenoid brakes or the connections to them.

1505 - Left Brake Trip

1506 - Right Brake Trip

Check these connections and the solenoid brakes. If the trip is still present after the above checks have been made, then the Power Module may be defective.

1.10 Trip Type 10 - High Battery Voltage

This occurs when the control system detects that the battery voltage has risen above 35V. The most common reasons for this are overcharging of the battery or bad connections between the control system and the batteries. Check the batteries and the connections to them. If the trip is still present after the batteries and connections have been checked, then the Power Module may be defective.

1.11 Control handlebar or DMS bar touched at Power-up

The most common cause of this trip is if the control handle is affected by a force when the control system is being switched on. When the control system is switched on, the battery gauge will blink for a short time. Check that the user is not touching the control handle before the blink finishes. If the problem persists, refer to section 1.7.

1.12 Communications Error

The most likely cause of a communications error is a defective cable between the Power Module and the Control handle Module. The cable should be checked for damage, and the entire handle replaced if found to have a fault. If the problem persists then the VR2 Power Module could be defective.

1.13 Inhibit 2 Active

This occurs when the Inhibit 2 input is active. The Inhibit 2 input is via the INH-2 way connector and is normally associated with speed limit or actuator functions. The operation of Inhibit 2 will depend upon the programmed settings and the vehicle on which it is being used. Check all wiring and switches connected to Inhibit 2. If these appear to be in working order, then the Power Module may be defective.

1.14 Inhibit 3 Active

This occurs when the Inhibit 3 input is active. The Inhibit 3 input is via the 3 way onboard charger (OBC) and is normally associated with this function. The operation of Inhibit 3 will depend upon the programmed settings and the vehicle on which it is being used. Check all wiring, switches and OBC (if fitted) connected to Inhibit 3. If these appear to be in working order, then the Power Module may be defective.

1.15 Actuator Motor Wiring Trip

This occurs when the control system detects a fault in the wiring to either actuator motor. Check the motor connectors and wiring. If the trip is still present after the above checks have been made, then the Power Module may be defective.

1.16 Current Limit Active

This occurs when the control system operates above the Current Limit Threshold for a period of time greater than the Current Limit Time. It has been designed to notify the Healthcare Technician that the control system has operated outside of its programmed range.

1.17 High Temperature

This occurs when the control system reaches its Temperature Threshold and thus becomes too hot. The controller will not allow drive until the controller has cooled down. An entry is made in the system log each time the controller gets too hot and goes out of drive.

SystemRoMedic™

Simple solutions for great results

SystemRoMedic™ is the name of Handicare's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic™ is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic™ is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic™ offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading from our website; www.handicare.com.

For questions about the product and its use

Please contact your local Handicare and SystemRoMedic™ representative. A complete list of all our partners with their contact details can be found on our website; www.handicare.com.

Handicare offers solutions and support to increase the independence of disabled or elderly people as well as to improve the convenience of those who are caring for them.

The Handicare Group is one of the leading healthcare companies in Europe with its own manufacturing organizations and sales companies in Norway, Sweden, Denmark, Germany, the Netherlands, Great Britain, France, China, Canada and the USA. Handicare's products are also distributed by partners in more than 40 countries worldwide. Our wide range of high-quality products includes a complete easy transfer system and other patient handling aids, stairlifts, car adaptations and bathing and toileting products.

Handicare AB
Torshamnsgatan 35
SE-164 40 Kista, SWEDEN
Tel: +46 (0)8-557 62 200
Fax: +46 (0)8-557 62 299
E-mail: info@handicare.se
www.handicare.com