SlingBar

System**RoMedic**

Instructions for use – English

Bruksanvisning – Svenska

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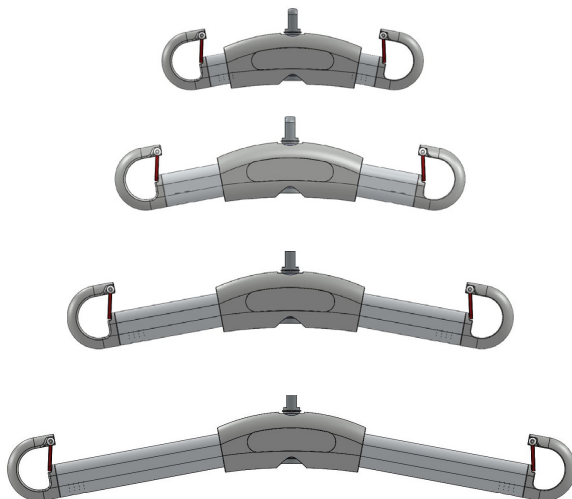
Návod na použitie – Slovensky

REF

70200001

70200002

70200003

70200071

SWL: 300 kg/ 660 lbs

Table of contents

English	3
Svenska	6
Norsk	9
Suomi	12
Deutsch	15
Netherlands	18
Français	21
Český	24
Slovensky	27
Symbols	29

SlingBar is a series of slingbars that can be used in combination with mobile or stationary lifts that are approved by Direct Healthcare Group. These slingbars have been designed for use with correctly fitted slings for lifting patients. They are available in four different sizes.

SystemRoMedic includes a wide range of functional, comfortable, high-quality slings that can be adapted for different types of lifting and for patients with different needs. The choice of model and material depends on the patient's needs and the transfer situation.

Functional inspection

Visual inspection

Check the condition and function of the product regularly. Check to ensure that material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.

Always read the manual

Always read the manuals for all assistive devices used during a transfer.

Keep the manual where it is accessible to users of the product.

Do not leave the patient unattended during a transfer situation.

To prevent discomfort and the risk of the patient sliding out of the sling, trial fit the sling carefully. First, apply the sling's back loops to the slingbar, and then apply the leg support loops. Use a slingbar that has been tested.

The lowest allowable safe working load always determines the safe working load of the assembled system. Always check the safe working loads for the lift and accessories before use.

Contact your dealer if you have any questions.

SlingBar Product Description

The SystemRoMedic SlingBar is made of aluminium.

SlingBars are used in combination with mobile or stationary lifts that are approved by Direct Healthcare Group.

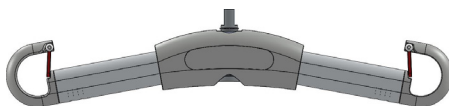
Lift hooks are equipped with safety latches.


SlingBar XS is 25cm wide and is appropriate for children, slim patients, and for gait training.

SlingBar S is 35cm wide and is appropriate for children, slim patients, and for gait training.

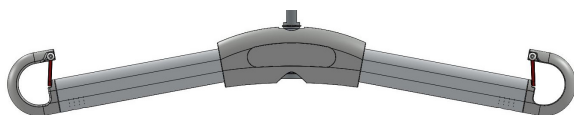
SlingBar M is 45cm wide and is appropriate for most types of transfers.

SlingBar L is 60cm wide and is appropriate for patients who require more space in the sling. These may include patients who are sensitive to pressure, broad-shouldered or obese.


Installation of SlingBar on mobile lifts:

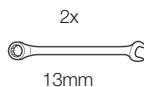- 1 x 90000038 (Screw M8x50)
- 1 x 90000026 (Nut M8)
- 1 x 90000039 (Sleeve 10/8x38)
- 2 x 90000124 (Washer for Cover cap M10)
- 2 x 90000014 (Cover cap M10)

Installation of SlingBar on stationary lift units:

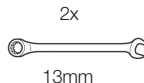- 1 x 90000038 (Screw M8x50)
- 1 x 90000026 (Nut M8)
- 1 x 90000039 (Sleeve 10/8x38)
- 2 x 90000124 (Washer for Cover cap M10)
- 2 x 90000014 (Cover cap M10)

Lifetime information & Periodic inspection

The product has an expected lifetime depending on how the product is used according to the manual instructions.

The service life of the product varies depending on usage frequency, washing, drying, etc. Please be sure to read the product label and follow the care instructions for this product on www.directhealthcaregroup.com

The periodic inspection of the non-rigid body support unit should be performed at the timestamps stated by the manufacturer, but at least every 6 months to be sure. More frequently inspections may be required if a non-rigid body support unit is used or cleaned more frequently than advised by the manufacturer. The periodic inspection instructions for this product are found on www.directhealthcaregroup.com

Safe working load

Different products on the same lift system (lift unit, slingbar, sling, scales and other lifting accessories) may have different allowable safe working loads. The lowest allowable safe working load always determines the safe working load of the assembled system. Always check the safe working loads for the lift and accessories before use. Contact your dealer if you have any questions.

Cleaning

When necessary, clean the slingbar using warm water or rubbing alcohol. Do not use cleaning agents containing phenol or chlorine, as this may damage the material.

Contact your local distributor if you have any questions about the product and its use.

See www.directhealthcaregroup.com for a complete list of distributors.

Always make sure that you have the right version of the manual.

The most recent editions of manuals are available for downloading from our website, www.directhealthcaregroup.com.

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Symbols

	This product complies with the requirements of the Medical Device Regulation 2017/745		Visual Inspection
	Medical Device		Read the manual
	Caution		Product Code
	Manufacturer information		Batch Code

Simple solutions for great results

SystemRoMedic® is the name of Direct Healthcare Group's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic® is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic® is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic® offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website;
www.directhealthcaregroup.com

For questions about the product and its use

Please contact your local Direct Healthcare Group and SystemRoMedic® representative. A complete list of all our partners with their contact details can be found on our website; www.directhealthcaregroup.com.

Moving Health Forward

Medical Device Class I. The product complies with the requirements of the Medical Devices Regulation 2017/745.

Direct Healthcare Group Sverige AB
Torshamnsgatan 35,
SE-164 40 Kista, Sweden

Tel: +46 (0)8-557 62 200
info@directhealthcaregroup.com
www.directhealthcaregroup.com