



Kirton Product Portfolio 2017



Specialist
Seating



Made in Britain

Customer Helpline: 0800 212709 | Email: info@kirtonhealthcare.co.uk | [@Kirton_Seating](https://twitter.com/Kirton_Seating) | www.kirton-healthcare.co.uk

Contents

Meet Ellie!	4	Fitform Riser Recliners	37
Training and Resources	5	Fitform 570 Vario	38
Are You Sitting Comfortably?	6	Fitform 570 Vario Hermes	40
Important Elements of Good Seating	8	Fitform 571 Vario Petite	41
Functionality Options of Dynamic Seating	10	Hallingbury	42
Working with Kirton	11		
Pressure Care Management	12	Upholstery Fabrics	
Choosing your Chair	14	Direct Upholstery	44
		Loose Covers	48
		Fitform Fabrics	50
Specialist Seating			
Duo	16	Warranty	51
Encora	18	Refurbishment Service	52
GE-II	20	Technical Specifications	54
Delta	22		
Florien Elite	24		
Florien Fife	26	Terms & Conditions	62
Florien II	28	Our Core Values	63
Omega	30		
Regent	33		
Stirling	34		
Sovereign	35		
Accessories	36		



Welcome to the **Kirton Portfolio** of **specialist seating**.

What we do at Kirton is simple; we make chairs to fit people. Posture and pressure care is our priority!

Since 1980 our products have been developed based on the needs and recommendations of both users and healthcare professionals, so you can rest assured that they are fit for purpose and provide the utmost comfort and support for each individual.

We produce all Kirton chairs at our factory in Suffolk so if there is something specific you need that you cannot see in our brochure, please just get in touch. We want as many people as possible to enjoy the benefits of good seating and will do all we can to help you achieve greater independence through sitting well!

Discover even more online.....

Visit www.kirton-healthcare.co.uk for further information on our products, expert advice from our OT Ambassador, Ellie Burcher, and the full Kirton Price List.

Beverly Lawrence
CEO

Meet Ellie!

Ellie Burcher is the Kirton OT Ambassador for Seating.

Having been an OT for many years, both with the NHS and as an independent practitioner, she has a wealth of experience across many clinical areas, but her true passion is for seating.

The complexities of seating have always fascinated me and nothing beats the feeling of getting it right for someone and seeing them sat correctly! From highly abnormal postures and challenging behaviours to seating clients that have been limited to 24-hour bed care, I've seen and assessed a wide range of conditions and postural needs. Now I want to share my knowledge and experience with you!

If you have any queries or would like to discuss something in more depth you can always visit the 'Ask Ellie' page at www.kirton-healthcare.co.uk and submit your questions directly to Ellie. Alternatively, see Ellie in action demonstrating the chairs on the individual product pages on our website.

Training and Resources

At Kirton we pride ourselves on our knowledge and experience of good seating and would love to share it with you! Our free training is offered nationwide

Training Options:

- 1) **Posture and Pressure Management Overview** - CPD accredited by the College of Occupational Therapists and delivered by either our fully trained team or our OT Ambassador, Ellie over a 1-2 hour time period.
- 2) **Seating Awareness Day** - Clinical CPD training from OT Ambassador, Ellie combined with a hands on product workshop, a fun-filled one day event.
- 3) **Tailor-made Training** - bespoke training for groups of healthcare professionals on the application of seating, posture and pressure management specific to your environment and client's needs.

The highly trained Kirton advisors are happy to demonstrate any product and collaborate with you on seating assessments too! Just call us on **0800 212709** or visit our website for more details.



Have you received your copy of our **Little Book of Seating** yet?

If not, contact us to request this pocket-sized guide from OT Ellie, crammed full of tips and advice to help make seating simple!



Are you sitting comfortably?


Problems associated with an incorrect sitting position

The majority of the population assume incorrect sitting positions and unfortunately it is a natural occurrence for the body to slump and change its position regularly. Poor sitting postures and unsuitable seating can develop and/or exacerbate both physical and medical conditions with far reaching consequences on the quality of life and overall well being of an individual. Sitting requires more effort than you think!

Difficulty in getting up

It is often difficult to get up as a result of reduced muscular strength and impaired functioning of the joints. This may also be hindered by the chair you are sitting in having:

- a seat height too low and/or too soft
- an inappropriate seat depth
- a large backrest angle
- armrests that are too short or absent altogether.

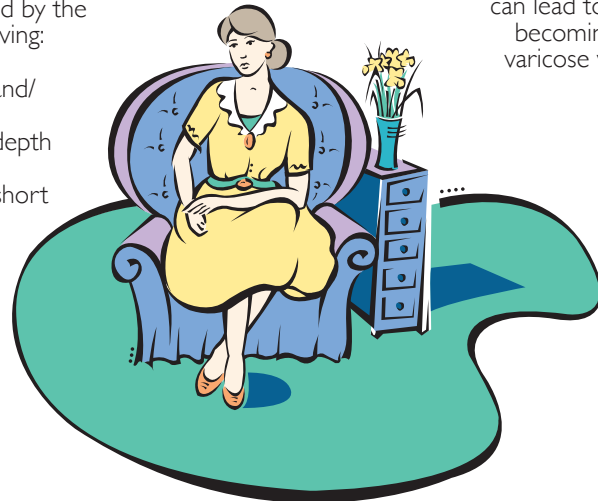

Blocking of veins and nerves

An excessively high sitting position with legs swinging can lead to veins and nerves becoming blocked causing varicose veins, sleeping legs and numbness.



Irritated tissue & back pain

Incorrect seat height can cause pressure on the bottom. An excessive slant on the back or too deep or too low a seat can lead to irritated tissue, back pain and cause difficulty getting up.



Back pain & poor posture

Inappropriate cushioning can result in sinking and slumping back into a convex shape, contributing to back pain and other joint and muscle related ailments.



Intestine trouble

An excessively low seat with an acute angle between the torso and upper legs can give rise to intestinal problems.

A well designed seat helps prevent fatigue & other complaints ensuring the body adopts the most natural posture.

Important Elements of Good Seating

To ensure the body assumes the correct natural anatomical position and optimal sitting posture, the chair should be tailored to each individual's exact measurements. In addition, the chair should combine the appropriate functionality and features to assist the seated person in having a healthy and comfortable posture.



Pressure Relief

Pressure relief characteristics within seating can considerably reduce the effects of friction, shearing and body weight upon skin and muscle tissue. Distributing body weight over a larger surface area will help reduce pressure. However, the use of special pressure relieving cushions and foams can help prevent the development of pressure ulcers. The inclusion of tilt-in-space is also excellent for redistributing and changing pressure points. Refer to page 12 for further information on pressure care management.

Upholstery

Upholstery has great significance on the properties of a chair. Body temperature, perspiration and tactile response are dependent on the finish and quality of the material used to upholster the chair; not forgetting other important factors including durability and wear.

Tilt-in-Space

Tilt-in-Space enables the occupant to be tilted forward or backwards in a chair whilst maintaining the position of the knees and hips to achieve a resting position. By keeping the angle between the chair seat and back at around 110°, the occupant will not slide forwards into an uncomfortable, slumped position. Tilting the body has the effect of distributing weight over a larger contact surface area providing a simple and effective way to combat the build up of pressure.

Back Angle Recline

Back angle recline has traditionally facilitated a resting or napping position and is suitable for those who are able to maintain their position independently. However, for those less able to sustain pelvic stability, back angle recline in conjunction with an angled or ramped seat or tilt facility, enables the benefit of a resting position to be realised.

Elevating or Arcing Legrest

An **elevating legrest** can assist with the lymphatic and blood circulation. Lifting and straightening the knees has the effect of allowing the blood to circulate more freely, reducing the effort required by the heart and arterial systems. Raising the legs can also help alleviate leg oedema, a build up of fluid in the legs.

An arcing or articulating legrest

extends forward as it elevates and is ideal for chairs with footboards as it maintains the positioning of the footboard and comfort for the user; in both the raised or lowered position. This is especially important where the occupant is able to independently adjust the legrest positioning and there is then no requirement for a carer to reposition the footboard.

Rise and Recline Facilities

Raise facilities on our Fitform range of riser recliner chairs gently lift individuals to a standing position without placing significant pressure on limbs and joints enabling independent transfers where possible. Stability and positioning is maintained by ensuring the feet remain close to the floor.

The **Recline facility** on our Fitform chairs enables independent repositioning back angle, allowing hips to open where required, and legrest elevation and/or tilt in space for ultimate comfort and positioning for the more ambulant and cognitively able.

Headrests

Many users do benefit from provision of headrest to maximise contact and support.

Seat Width

Achieving the correct **seat width**, the internal width of seat between the armrests, ensures arms sit comfortably on armrests and contributes to pelvic stability and security of the user. If too wide it can be difficult to hold good posture, if too narrow the person may not be able to sit right back in the chair.

Seat Depth

Achieving the correct **seat depth**, from the front of the seat cushion to the backrest of the chair is essential to provide comfort and postional support. If the seat is too deep the person will slide forward, losing posture and increasing localised pressure, and if it is not deep enough, pressure will be concentrated on a smaller area of the body and not as well distributed.

Seat Height Adjustment

Achieving the correct **seat height** for the user allows the feet to be positioned firmly on the floor or footboard, to ensure even weight distribution and assist the ambulant user to stand from the chair more readily. Too high or too low a seat can lead to pressure injuries and discomfort.

Mobility

For users needing to be moved when seated, the chair must incorporate push handles and braked castors. This allows the chair to be moved with ease and secured for moving and handling activities. When in transit it is essential that the user's feet are supported off the ground by an integral flip-up or sliding footrest to prevent accidental damage.

Height Adjustable Armrests

Appropriate armrest height provides comfort and support as well as an increased sense of security.

Lateral Supports

Users who have poor sitting balance, neurological conditions or postural changes tend to lack trunk stability or lean heavily to one side therefore the use of lateral supports within the chair can provide sufficient support to promote a more upright, midline posture.

Functionality Options for Dynamic Seating



Manual



Electric


Regulated
Motion

Tilt
-in-
Space

Back Angle
Recline

Elevating
Legrest

Seat Height
Adjustment

Raise
Facility

Please use this key to identify the features and benefits of each chair

Manual Adjustment

Manual adjustment enables fluid and precise movement of the tilt-in-space facility or back angle recline by the release of a manual mecklock lever situated on the push handle.

Manual tilt-in-space or back angle recline gently moves back and forth as directed by the carer with minimal effort, whilst the legrest can be elevated by a separate release mechanism at the front of the chair. The legrest gently raises or lowers accordingly, as guided by the carer, to attain the appropriate position.

Electric Adjustment

Electric adjustment enables the carer or user, where appropriate, to adjust the tilt-in-space or back recline facility and/or the legrest by the simple touch of a button, providing an electrically powered motion to these mechanisms. Using a simple handset, movement of the chair's electric adjustments is smooth and gentle, reducing the effort whilst increasing the positioning range and flexibility of the chair.

Regulated Motion

Regulated Motion has been incorporated into the GE-Il, Delta and Duo designs to create the ultimate pressure management chair as well as a source for proprioceptive and kinaesthetic input for those whose condition means they are largely immobile.

The combination of low interface pressures on the head, shoulders, arms, legs and ankles is achieved using Reflexion™ Foam alongside the appropriate pressure relieving seat cushion. A correctly dimensioned tilt-in-space chair, with arcing legrest elevation and the ability to "regulate the motion" or movement of the chair, provides the user with pressure management through dynamic movement.

How?

A six-buttoned hand controller is programmed to ensure the tilt-in-space has "Regulated Motion" or the ability to move over a set range automatically adjusting the tilt positioning. The pattern of movement is set to repeat to ensure motion over a period of time within a specified tilt range.

Why?


Regulated Motion ensures that pressure points on the body are in continuous flux. This enables the user, who may be sitting in the chair for long periods, to sleep with the secure knowledge that dynamic movement will be at work continuously changing the pressure hotspots. It will reduce the propensity for pressure related problems and improve the comfort and sitting experience. Also, carers will not be burdened with the expectation of 20 to 30 minute position changes.

Do you need further assistance?

We offer a free assessment service!

Our team of seating experts understand how important it is to make the right choice and can help you every step of the way. Finding a chair to suit your needs starts with a good assessment, so let us help you do it correctly!

Just call or visit us online at
www.kirton-healthcare.co.uk
and we'll be happy to help.

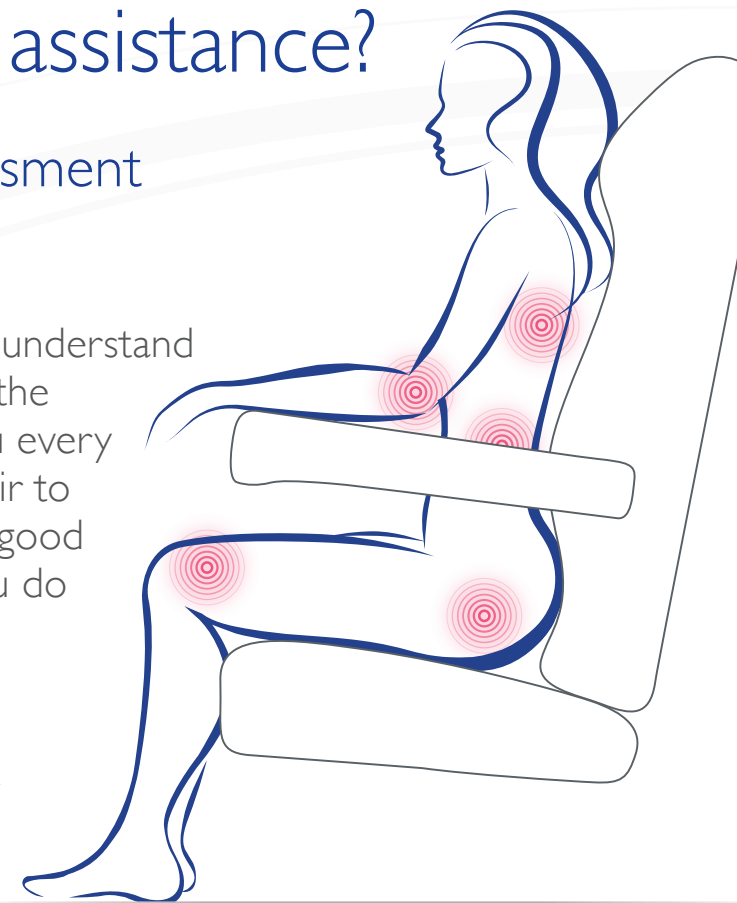



Pressure Care Management

Pressure care management is an important consideration for individuals that are seated for long periods of time or are limited in their ability to adjust their own posture.

What is a pressure injury?

A pressure injury is localised damage to the skin and the tissue underneath it. Sitting or lying in the same position will begin to cut off the flow of blood to that area, blocking oxygen and vital nutrients from maintaining healthy tissue.

When the tissue becomes starved for too long it begins to die and a pressure injury starts to form. Damage from a pressure injury will range from slight discoloration of the skin (stage 1) to open sores that go all the way to the bone (severe). Whilst there are many aetiological factors including microclimate nutrition, perfusion and the condition of the soft tissue that contribute to the development of pressure damage, the largest single cause is unrelieved pressure.

While all of our specialist seating products have pressure relieving properties that are integral to their design and functionality, additional pressure relief is available and can be built into the seat of your chair. The latest technology and an expert knowledge of pressure prevention, allow us to provide you with an unparalleled selection of pressure care solutions.

Choosing your pressure care is as easy as...

1 Reflexion™ Foam:
Low – Medium Risk



These cushions have an upper Vasco memory foam layer which is both energy absorbing and temperature sensitive. The cushion moulds itself to the user's shape and size helping reduce pressure points and spread body weight more evenly across the whole surface of the cushion. The supporting layer of melamine-free Reflex foam improves support and flexibility for the upper layer of Vasco.



**Need more information?
Ask Ellie!**

Our Little Book of Seating has tips and advice on pressure care from OT Ellie

2 Transflo Gel:
Medium – High Risk




The Transflo pressure cushion incorporates two fluid-filled, vapour-permeable, multi-stretch sacs that are encased within a supportive, heavy-duty foam surround. The fluid is able to move between the two compartments as the cushion contours to the body's shape and position, redistributing pressure over the widest surface area possible. The gel cushion's design encourages the user to sit in a neutral pelvic tilt with the majority of the weight being taken by the buttocks and thighs.



Intelli-Gel Evidence Pack
Our Clinical Evidence Pack underpins the efficacy of Intelli-Gel® Integral.

3 Intelli-Gel® Integral:
High – Very High Risk

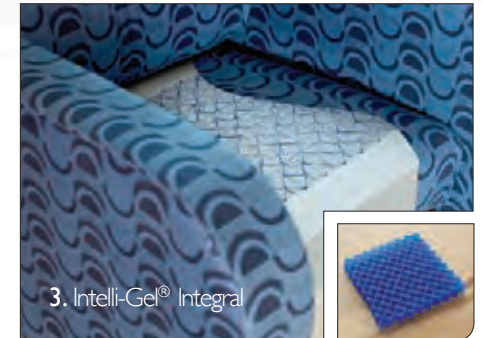

Intelli-Gel® has a soft elastomer, honeycomb structure that contours gently to large surface areas distributing body weight evenly. It is the honeycomb columns that when joined together provide the right support and pressure relief. The Intelli-Gel® cradles hotspots, providing superior pressure relief, support and stability when seated. Intelli-Gel® Integral can be built into the seat of your chair and is a suitable option for those at high to very high risk of pressure related problems. It can also be used with those who have pressure injuries up to Grade II (EPUAP).

More information on these pressure relief solutions is available on our website, where you will also find the results of our clinical assessments into the efficacy of Intelli-Gel® and Reflexion™ Foam.

Choosing your Chair

We would always recommend a full seating assessment in order to select the right chair; but to narrow down your options the chart below can help you to understand which features could be required.

Does your client...	Consider...	Is your client...	Consider...
<input type="checkbox"/> Lean left or right?	Lateral Supports	<input type="checkbox"/> Ambulant?	Rise Function Flip-up Footrest Negative Legrest Angle Forward Tilt
<input type="checkbox"/> Contract at the knee?	Negative Angle Legrest	<input type="checkbox"/> Non-Ambulant?	Good Hoist Access Mobile Frame Push Handle
<input type="checkbox"/> Droop their head?	Back Recline Tilt-in-Space	<input type="checkbox"/> Kyphotic?	Waterfall Backrest Articulating Backrest
<input type="checkbox"/> Have a propensity for pressure injuries?	Integral Pressure Relief Tilt-in-Space		
<input type="checkbox"/> Side transfer?	Drop Arm Removable Arm		
<input type="checkbox"/> Have poor balance?	Lateral Supports Tilt-in-Space		
<input type="checkbox"/> Slide forward in their chair?	Ramped Seat Module Tilt-in-Space		

For further details of what these product features are, or which chairs incorporate them, visit www.kirton-healthcare.co.uk, where you will also find a decision making tree to help you select the right product.



The following matrix can be used to help you find the right product from our dynamic specialist seating range based on which features you require:

	Duo	Encora	GE-II	Delta	Florien Elite	Florien II	Omega
Integral pressure relief	✓	✓	✓	✓	✓	✓	✓
Tilt-in-Space	✓	✓	✓	✓	✓	✓	-
Seat height adjustment	✓ ¹	✓	-	-	✓	-	-
Seat depth adjustment	✓	✓	✓	✓	✓	✓*	✓*
Seat width adjustment	✓	+	+	+	✓	+	+
Back height adjustment	✓	-	-	-	-	-	✓*
Footrest height adjustment	✓	✓	✓	✓	✓	✓	-
Legrest elevation	✓	✓	✓	✓	✓	✓	-
Manual Back Recline	✓	✓	-	-	✓	-	✓
Manual Tilt-in-Space	✓	✓	✓	✓	✓	✓	-
Electric Tilt-in-Space	✓	✓	✓	✓	-	-	-
Regulated Motion Tilt-in-Space	✓	-	✓	✓	-	-	-
Built in lateral support	OPTION	✓	✓	✓	OPTION	OPTION	✓
Waterfall back option	✓	✓	-	-	✓	✓	-

Remember, there are a few things that all dynamic Kirton chairs have in common...

- ✓ Modular design - refurbish and replace over time
- ✓ Postural support and pressure relief
- ✓ Versatility - suitable for domestic use, multi-user environments, clinical and care settings

- ✓ Ease of cleaning
- ✓ Simple to use
- ✓ British build

¹ Not with manual version

⁺ Different seat widths available

^{*} Specified at time of manufacture

DUO

For the non-ambulant user with profound posture and pressure care needs.

The **Duo** is our most **versatile seating system** designed for those who are unable to maintain their posture and need high levels of pressure relief.

See p.54 for
sizes and contact
us for your **free**
assessment

*DUO MAJOR with Regulated
Motion in Panvelle Stretch
Tenby Lilac*

Watch the
Duo in action
- scan here or
view the video
online!



Dynamic features provide the ultimate in **positioning** and **postural management** for individuals who are at risk of developing an **unconventional body shape**, unable to maintain a **midline position** and **require additional support**.

The adjustability and key features of the Duo range ensure the core principles of seating are met for each individual.

Features and Benefits

- Two chair sizes: Major and Minor
- Adjustable seat length, seat width, seat height*, back height, foot board and arm height
- Simple, tool-free adjustments
- Tilt-in-Space action - Manual, Electric or Regulated Motion
- Legrest elevation - Manual or Electric
- Arcing legrest action – does not compromise Flip Up Footboard positioning
- Split-wing shoulder support for adjustable support of the head and shoulders
- Adjustable headrest for precise positioning of the head
- Two back style options: Waterfall or Sculpted
- Multi-directional stretch, breathable Dartex material on all user contact surfaces
- Easy to use handset on Regulated Motion and Electric models
- Push handle
- Strong, durable 100mm single wheels, rear braked
- Modular construction enables easy refurbishment

Pressure Relieving Seat Options (integral)

- Intelli-Gel® Integral seat module
- Transflo Gel seat module
- Reflexion™ Foam seat module

Accessories

- Lateral supports
- Leg-Laterals
- Pommel
- Legrest bracket to achieve a 25° negative angle
- Flat seat board
- Foot Sandals
- Ankle Huggers
- Lap-Belts and Pelvic Positioners
- Universal Mobile Tray

Weight Limit

- 160kg / 25 stone

*Seat height adjustment not available on manual version

The Kirton Duo chairs, supported by W. Munro (Rehab) Ltd, have been developed for clinical needs in conjunction with Occupational Therapists, Physiotherapists and Postural Management Services.





Encora in Panvelle Stretch
Monaco Black/Chalk

ENCORA

For the semi and non-ambulant user with high posture and pressure care needs.

The ability to **raise** and **lower** the chair means the **Encora** will attain the correct **seat height** for each user. This helps to maintain an **even weight distribution** by ensuring feet are firmly positioned on the floor and assists the ambulant user to stand from the chair.

Watch the
Encora in
action - scan
here or view the
video online!



The Encora's **Advanced Comfort Backrest**, incorporating the highly acclaimed **Reflexion™ Foam**, moulds itself to the contours of the user, helping disperse **body weight** to provide **superior comfort** and **support**. Alternatively, a **Waterfall Back** is also available for more cushioned, immersive support.

Features and Benefits

- Tilt-in-Space action – Manual or Electric
- Legrest elevation - Manual or Electric
- Back angle recline - Manual or Electric
- Adjustable seat height – enables those who are still ambulant to place feet firmly on the floor
- Adjustable seat length
- Two back style options: Advanced Comfort Backrest with built in adjustable lateral support or Waterfall Backrest
- Pressure relieving Reflexion™ foam on user contact surfaces
- Multi-positioning, integral headrest
- Multi-directional stretch, breathable Dartex material on all user contact surfaces

Pressure Relieving Seat Options (integral)

- Intelli-Gel® Integral seat module
- Transflo Gel seat module
- Reflexion™ Foam seat module

Accessories

- Height adjustable flip up footrest
- Laminated tray with straps
- Cushioned or Universal Headrest
- Lap-Belts and Pelvic Positioners

Weight Limit

- 160kg / 25 stone

See p.54 for
available sizes and
contact us for an
assessment or
Price List

Encora 450 with Waterfall Back
and Electric adjustment in Panvelle
Stretch Monaco Mulberry



GE-II

For the semi and non-ambulant user with high posture and pressure care needs.

Underpinned by **scientific research** and **ergonomic design principles**, the **GE-II** is a groundbreaking model which achieves new levels of **comfort** and **immersion**.

See p.55 for sizes and contact us for your **free** assessment

Watch the GE-II in action - scan here or view the video online!



GE-II 450 Electric in Vinyl
Enduralex Prairie Duststorm.



Made in Britain

Offering excellent **postural support, positioning** and **pressure relief**, the **GE-II** is the perfect choice for those who want their chair to deliver on both **substance** and **style**.

This elegant chair has been designed to ensure that all the key adjustments are simple and effective, meeting the needs of both ambulant and non-ambulant users, as well as carers, in a wide variety of environments.

Features and Benefits


- Tilt-in-Space action – Manual, Electric or Regulated Motion
- Legrest elevation - Manual or Electric
- Arcing legrest action – does not compromise footboard positioning
- Adjustable footboard height and angle
- Built in, helical-motion lateral supports to stabilise the pelvis
- Evidence-based articulating, supportive backrest
- Immersive comfort in seat and back aiding pressure management
- Adjustable seat depth
- Multi-directional stretch, breathable Dartex material on all user contact surfaces
- Good hoist access
- Modular construction enables easy refurbishment
- Two wheel options: 100mm wheels, braked at rear and Large wheel version available with 315mm rear wheels with brake bar (400/450 seat widths only)
- Directional lock castors available on request.

Accessories

- Negative angle legrest
- Choice of Headrest (See page 36)
- Leg-Laterals
- Laminated Tray with Straps
- Headcap
- Lap-Belts and Pelvic Positioners

Weight Limit

- 180kg / 28 stone

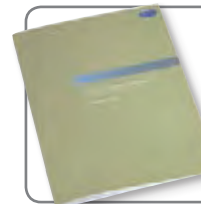

GE-II design underpinned by scientific research

- 1 Lateral Support 2 Articulating Backrest
3 Adjustable Seat Depth 4 Rear Braked Wheels



DELTA 450 with Manual adjustment and Universal Headrest

DELTA

For the semi and non-ambulant user with high posture and pressure care needs.

Easy to **adjust** and **manoeuvre**, the **Delta** is as **effective** for the **carer** as it is for the user, great for use in **care settings** or the **home**.

See **p.55** for dimensions and find the full Price List online



Key to the **Delta** is the incorporation of the remarkable **Reflexion™ Foam** on all user contact surface areas which, combined with the **vapour permeable, multi-stretch Dartex upholstery**, provides added **pressure relief** and **comfort**.

Features and Benefits

- Tilt-in-Space action – Manual, Electric or Regulated Motion
- Legrest elevation - Manual or Electric
- Arcing legrest action – does not compromise footboard positioning
- Two back style options: Reflexion™ Foam or Advanced Comfort Backrest
- Seat length adjustment
- Pressure relieving Reflexion™ Foam on user contact surfaces
- Multi-positioning, integral headrest
- Adjustable footboard height and angle
- Built-in lateral supports
- Multi-directional stretch, breathable Dartex material on all user contact surfaces
- Optional large button hand controller for Electric or Regulated Motion chairs
- Bumpers to reduce upholstery damage when in transit
- Two wheel options: 100mm wheels, braked at rear and Large wheel version available with 315mm rear wheels with brake bar (400/450 seat widths only)
- Directional lock castors available on request.

Pressure Relieving Seat Options (integral)


- Intelli-Gel® Integral seat module
- Transfo Gel seat module
- Reflexion™ Foam seat module

Accessories


- Leg-Laterals
- Negative angle legrest
- Laminated tray with straps
- Headcap
- Adjustable foam headrest
- Back height extension (by 100mm)
- Lateral support wedges
- Lap-Belts
- Pelvic Positioners

Weight Limit

- 180kg / 28 stone

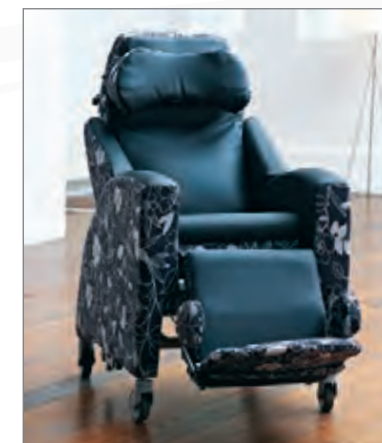

- 1 Bumpers for added protection
- 2 Adjustable footboard height and angle
- 3 Multi-positional integral headrest
- 4 Smooth arcing legrest motion

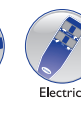

FLORIEN ELITE

Dynamic motion, exceptional comfort, postural support and pressure management

The **Florien Elite** has the ability to meet the **postural** and **pressure management** needs of **ambulant** as well as **semi** and **non-ambulant** service users seated for **long periods** whether receiving **treatment** or in **long term** care environments.

For full specification see p.56 and contact us for your free assessment!

With both **tilt-in-space** and **back angle recline** functionality the **Florien Elite** also has a choice of **two backrests**, **Waterfall** or **Contoured** and **three armrest** options to meet the needs of different **care** settings.

The unique dynamic upholstered arms float across the seat to achieve the appropriate seat width for each user. Moreover these arms have the ability to swivel backwards to effect a side transfer or easier hoist access.

Alternatively, the dynamic drop arm-rest is suitable for those that can side transfer and enables the chair to integrate into rehabilitation environments, similarly the basic cushioned armrest is ideal for general care and treatment facilities with the ability to apply a drip-stand or Phlebotomy arm support.

With a host of options and accessories to choose, the Florien Elite is a most versatile seating solution.

Features and Benefits

- Manual tilt-in-space with independent back angle recline
- Manual elevating legrest with negative retraction to aid standing transfers and accommodate contractures
- Removable, "swivel-back" armrests for side transfer and easier hoist access
- Multi-positioning integral headrest
- Adjustable seat width and seat length
- Adjustable seat height (specify at time of manufacture)
- Choice of a sculpted contoured or immersive waterfall backrests
- Choice of armrest style
 - Upholstered dynamic armrest (as standard)
 - Basic cushioned armrest (for portering or use with a drip-stand or phlebotomy armrest)
 - Dynamic drop-armrest
- Ergonomic push handle
- Simple replacement of modular components
- Reflexion™ foam upholstered in multi-stretch, breathable Dartex on all user contact surfaces
- No Velcro used
- 100mm Castors, braked at rear
- Directional lock castors available on request.

Pressure Relieving Seat Options (integral)

- Flat or ramped removable seat modules:
 - Intelli-Gel® Integral
 - Transflo™ Gel
 - Reflexion™ Foam

Accessories

- Lateral Supports
- Arcing / articulating legrest
- Height adjustable, "flip up" Footrest (90° foot positioning) or height adjustable articulating Footrest (Plantar flexion achievable)
- Drip Stand (includes fixings – only to be used with Basic Cushioned Armrest)
- Phlebotomy armrest (additional support - only to be used with Basic Cushioned Armrest)
- Cushioned or Universal Headrest
- Lap-Belt with D-Ring Straps
- Pelvic Positioners with D-Ring Straps
- Universal Tray

Weight Limit

- 160kg (353 lbs) / 25 stone



FLORIEN ELITE in
Panvelle Stretch Sonnet
Maive and Graphite
Dartex



Made in Britain

FLORIEN FIFE

For the semi and non-ambulant user with mid to high posture and pressure care needs requiring immersive comfort.

The **Florien Fife** is a **manual, dynamic tilt-in-space chair** suitable for **semi and non ambulant users** who require **assistance** with maintaining their **seated position and posture**.

Contact us
for current prices
and see the full
specification on
p.56

FLORIEN FIFE with
Lateral Supports and
Flip-up Footrest in Wild
Heather Spey Vinyl



The **Waterfall backrest** provides an **immersive cushioned support** which has proved effective for those with a more **kyphotic posture**.

This highly configurable modular chair has a ramped seat angle enabling a good seated position to be achieved and maintained alongside a negative legrest angle to either accommodate those with contractures or indeed those that are able to standing transfer in or out of the chair. The **Florien Fife** offers dynamic movement for the user and is highly mobile for the carer with effective hoist access by the simple removal of the arm modules.

Features and Benefits

- Manual Tilt-in-Space action
- Manual legrest elevation
- 25° negative legrest angle - accommodates contractures at the knee or aids standing transfers
- Ramped Intelli-Gel® Seat Module
- Removable arms for easier hoist access and side transfers
- Three seat length options: 435mm, 460mm, 485mm (to be specified at time of manufacture)
- Three seat width options: 400mm, 450mm, 500mm (determined by width of arm)
- Multi-positioning, integral headrest
- Ergonomically positioned push handle
- Waterfall Backrest
- Height Adjustable Flip-Up Footrest
- Fixing Plates for Pelvic Positioners
- 100mm castors, braked at rear

Accessories

- Flat Seat Board with built-in 60mm ramp at front
- Infill Arm Blocks
- Lateral Supports
- Seat Infill Block
- Universal Headrest
- Lap-Belts and Pelvic Positioners
- Laminated Tray with Straps
- Universal Mobile Tray

Weight Limit

- 180kg / 28 stone



FLORIEN II

For the semi and non-ambulant user with mid to high posture and pressure care needs.

A **dynamic** and **highly mobile** seating system, the **Florien II** enables greater **independence** by promoting **correct posture** and facilitating a **supportive, comfortable seated position**.

FLORIEN II in Vinyl
Enduratex Prairie Raindance

For full
specification see
p.56 and contact
us for your **free**
assessment!

Watch the
Florien II in
action - scan
here or view the
video online!



The **removable arm** is a fantastic feature of this chair, making it suitable for those who are **hoisted** or able to **side transfer**.

The modularity of the **Florien II** enables simple replacement parts for reissue to new users making it a cost effective seating solution and valuable asset for care providers.

Features and Benefits

- Manual Tilt-in-Space action
- Manual legrest elevation
- 25° negative legrest angle - accommodates contractures at the knee or aids standing transfers
- Removable arms for easier hoist access and side transfers
- Three seat length options: 435mm, 460mm, 485mm (to be specified at time of manufacture)
- Three seat width options: 400mm, 450mm, 500mm (determined by width of arm)
- Multi-positioning, integral headrest
- Supremely comfortable moulded backrest
- Ergonomically positioned push handle
- Two wheel options: 100mm wheels, braked at rear and Large wheel version available with 315mm rear wheels with brake bar
- Directional lock castors available on request

Pressure Relieving Seat Options (integral)

- Intelli-Gel® Integral seat module
- Transflo Gel seat module
- Reflexion™ Foam seat module

Accessories

- Height adjustable flip up footrest
- Adjustable lateral supports
- Adjustable headrest
- Lap-Belt
- Pelvic Positioners
- Universal Tray
- Flat Seat Board

Weight Limit

- 180kg / 28 stone

FLORIEN II in Vinyl Enduratex
Prairie Raindance



FLORIEN II in Vinyl
Enduratex Prairie Raindance
with removable arm



Next Day Delivery...

Available on Florien II chair



OMEGA

For the semi and non-ambulant user with Huntington's Disease or other neurological conditions.

See the full specification on p.57 and call us for your free assessment!

OMEGA in Vinyl Brookland Espresso

Originally designed for those with **Huntington's disease** and other neurological conditions, the **comfortable** and **secure Omega** is a **robust, durable** chair able to withstand severe movements yet provide a **safe and relaxed seated position**.

A sculpted back and contoured seat assures optimum comfort and provides good positioning for activities such as eating and drinking, while relaxing and sleeping is encouraged by the uniquely linked back and seat recline.

Features and Benefits

- Manual back recline and variable seat angle – as the back reclines, the seat angle increases to enhance comfort and safety
- Pivoting seat at front edge to retain knee to floor height
- Robust, durable frame with modular back, arm and seat components
- Horizontally and vertically sculpted back aiding midline stability
- Two back height options: 740mm or 800mm*
- Three seat width options: 380mm, 430mm, 510mm*
- Three seat length options: 430mm, 470mm, 500mm*
- Supportive Reflexion™ foam legrest – (removable)
- Ergonomically positioned push handle

Pressure Relieving Seat Options (integral)

- Intelli-Gel® Integral seat module
- Transflo Gel seat module
- Reflexion™ Foam seat module

Accessories

- Sliding footrest for use when legrest is removed
- Weighted Universal headrest
- Cushioned headrest
- Washable loose covers
- Headcap
- Lap-Belt and Pelvic Positioners

Weight Limit

- 150kg / 23½ stone

*To be specified at time of manufacture



OMEGA with sliding footrest in Vinyl Berkshire Boxford

Watch the Omega in action - scan here or view the video online!



Back Angle Recline



Manual

REGENT

For the ambulant user with moderate to high posture and pressure care needs.

Sophisticated designs disguise the features that make the **Regent range** the ideal robust seating solution for the **ambulant users**.

Find the full specification on p.57

REGENT with Cushioned Back in Panvelle Stretch Plain Crimson

High, broad armrests offer a sense of **security** as well as facilitating **easy standing** while different contoured **back styles** provide **superior comfort** and **postural support**.

The adjustable seat length and angle enables the Regent to be tailored for each individual user.

Features and Benefits

- Two sizes: Major and Minor
- Four design styles to choose from
- Adjustable seat length and angle
- Modular configuration
- Pressure relieving Reflexion™ foam on backrest and arms of chair
- Reclining back version available

Pressure Relieving Seat Options (integral)

- Intelli-Gel® Integral seat module
- Transflo Gel seat module
- Reflexion™ Foam seat module

Accessories

- Sliding footrest
- Push handle
- Laminated tray with straps
- Loose covers for armrests and seat
- Weighted Universal Headrest
- Lap-Belt and Pelvic Positioners

Weight Limit

- 102kg / 16 stone




REGENT CLASSIC



REGENT WITH WINGED BACK in Panvelle Stretch Plain Lilac



REGENT WITH MODERN BACK in Vinyl Brookland Espresso

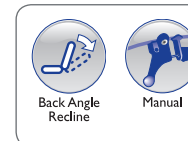

STIRLING

For the semi and non-ambulant user, the frail elderly and those with physical and learning disabilities.

Elegant simplicity combined with good proportions and attention to detail is at the heart of the Stirling. The degree of seat ramp to back angle recline combines with lateral support to ensure the correct posture and position of the user.



STIRLING in Vinyl Etch Peony

Find the full
specification
on p.58

Features and Benefits

- Elegant, simple design
- Correct positioning and support
- Fixed angle seat and back
- Lateral supports
- Built-in headrest support
- 75mm wheels, rear braked

Pressure Relieving Seat Options (integral)

- Intelli-Gel® Integral seat module
- Reflexion™ Foam seat module

Accessories

- Head supports
- Sliding footrest
- Legrests
- Laminated tray
- Wedges and cushions
- Lap-Belt and Pelvic Positioners

Weight Limit

- 102kg / 16 stone



SOVEREIGN

For the semi and non-ambulant user and those with neurological conditions.

Contact us
for your free
assessment

SOVEREIGN ADULT with
Loose Covers in Panaz
Trevira Spirit Evergreen

Simple, stylish and effective, the Sovereign provides excellent postural support and comfort.

Tailoring the Sovereign for each individual is made possible with the interchangeable back cushions and seat modules providing a bespoke solution in terms of seat length, height and angle. Strong construction, simple servicing and optional washable loose covers make the Sovereign a very practical chair.

Features and Benefits

- Two sizes: Adult and Adult Narrow
- Excellent posture and support
- Modular back and seat
- Choice of seat modules offering different seat lengths and angles
- Two back cushions: Foam or Dacron
- 75mm wheels, rear braked

Accessories

- Loose covers
- Sliding footrest
- Legrests

Weight Limit

- 102kg / 16 stone

Pressure Relieving Seat Options (integral)

- Intelli-Gel® Integral seat module
- Reflexion™ Foam seat module

Accessories

A wide range of accessories is available to complement our range of specialist seating and provide additional support.

For full details of all accessories and their specifications or prices please contact us or refer to the website www.kirton-healthcare.co.uk



Universal Tray



Pelvic Postioner



Fleecy-Faced Pelvic Positioner



Lap-Belt



Weighted Neck Cushion



Weighted Neck Support



Weighted Head Pillow

The Fitform Range of Riser Recliners

Kirton's Fitform Range of Riser Recliner chairs incorporates both the contemporary, European styled Fitform collection and the Hallingbury, a traditionally designed chair with truly British styling.

Much more than just a collection of chairs, the ergonomically acclaimed Fitform riser recliners combine an unrivalled understanding of postural seating, modern styling and an unwavering commitment to quality.

Personalising each chair for the user in terms of seat height, seat depth, seat width, back height and arm height creates a postural experience that alleviates pressure, promotes appropriate posture, encourages well being and avoids the negative impact of incorrect sitting and poor support.

Whether it is for seating in the home or within a healthcare environment the Fitform range can be tailored to meet specific needs in terms of functionality and the variety of options and accessories available. A range of pressure relief options, including the innovative Intelli-Gel® Integral cushion, is available with all riser recliners offering further comfort and support to those seated for long periods. Other options include swivel arms for side transfer and a selection of alternative back styles through to electrically heated back and seat for enhanced luxury and comfort.



FITFORM 570 VARIO

Stylish and contemporary, the **Fitform 570 Vario** is ideal for people who – either for medical or other reasons – want the most **comfortable** and **supportive** chair around.

The 570 Vario's triple motor action allows the tilt-in-space, back angle recline and elevating legrest to be adjusted independently in order to achieve a relaxed, comfortable position. The power lift also helps the user to an easy standing position. All movements are smooth and effected at the touch of a button.

One of the most unique features of our riser recliner range is the **Rotary Continuous Passive Motion (RCPM)** option. The seat of an RCPM Fitform chair rotates slightly three to five times per minute (0.8° both to the right and to the left), while the backrest remains immobile. Passive Motion means that the user does not have to do anything to set the rotary movement into action; it will start as soon as the user sits down on the chair. RCPM is the ultimate solution for people suffering from backache. Scientific research has shown that it can substantially reduce problems in the lower back and in some cases even prevent them.*

*Warning: If back troubles are caused by pelvic instability or a broken vertebra, the use of RCPM is not recommended.

Features and Benefits

- Triple Motor independently operates Raise Facility, Tilt-in-Space, Back Recline, Elevating Legrest
- Adjustable integral lumbar support
- Multi adjustable integral headrest
- Tailored seat height, arm height, seat depth and seat width

Pressure Relieving Seat Options (integral)

- Intelli-Gel® Integral seat module
- Transflo Gel
- Air Positioning & Pressure Relief System

Accessories

- Swivel armrests for side transfer
- Wheels
- Mobility Kit
- Electrically heated seat and backrest
- Electrically adjustable head support & positional memory function.
- RCPM


Weight Limit

- 150kg / 23½ stone
- Legrest weight limit 30kg / 5 stone

See page 59 for full specification



Extended Legrest

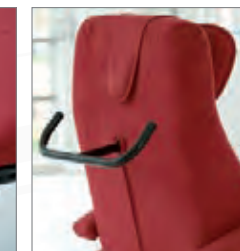

Mobility Kit



Multi adjustable integral headrest



FITFORM 570 Vario in Divine Velour Chocolate Brown



Tilt-in-Space



Back Angle Recline



Elevating Legrest



Raise Facility



Electric



Made to Measure

FITFORM 570 VARIO HERMES

Bariatric
Solution!

Proportioned to accommodate those up to **240 kg** or **40 st** in **weight**, the **Hermes** combines **modern styling** with **effortless performance**.



The three motor functionality, tilt-in-space, back recline and legrest elevation can be effected independently at just the click of a button, as can the remarkably simple standing action. A personal assessment ensures the chair is tailored to your individual needs in terms of seat height, seat depth, seat width and arm height.

Features and Benefits

- Triple Motor independently operates Raise Facility, Tilt-in-Space, Back Recline, Elevating Legrest
- Adjustable integral lumbar support
- Multi adjustable integral headrest
- Tailored seat height, arm height, seat depth and seat width

Accessories

- Swivel armrests for side transfer
- Mobility Kit
- Electrically heated seat and backrest
- Electrically adjustable head support & positional memory function.

Weight Limit

- 240kg / 40 stone
- Legrest weight limit 28kg / 4½ stone

See page 59 for full specification

Pressure Relieving Seat Options (integral)

- Intelli-Gel® Integral seat module
- Transflo Gel
- Air Positioning & Pressure Relief System



Tilt-in-Space



Back Angle Recline



Elevating Legrest



Raise Facility



Electric



Made to Measure

FITFORM 571 VARIO PETITE

The **smaller sister** to the Fitform 570 Vario, the **Fitform 571 Vario Petite** has identical features and adjustments in a chair that is perfectly proportioned for those **under 5ft 3"**.



Features and Benefits

- Triple Motor independently operates Raise Facility, Tilt-in-Space, Back Recline, Elevating Legrest
- Adjustable integral lumbar support
- Multi adjustable integral headrest
- Tailored seat height, arm height, seat depth and seat width

Accessories

- Swivel armrests for side transfer
- Mobility Kit
- Electrically heated seat and backrest
- Electrically adjustable head support & positional memory function.

Weight Limit

- 150kg / 23½ stone
- Legrest weight limit 30kg / 5 stone

See page 60 for full specification

Pressure Relieving Seat Options (integral)

- Intelli-Gel® Integral seat module
- Transflo Gel
- Air Positioning & Pressure Relief System

Underpinned by the knowledge that a correctly proportioned chair can have exceptionally positive effects on the well being of an individual, the Fitform Petite ensures that smaller individuals can achieve a healthy sitting posture too.

The Fitform Petite combines ease with functionality by delivering smooth tilt-in-space and a straightforward standing action while putting you in control of your sitting position with independent back angle and legrest actions.



Tilt-in-Space



Back Angle Recline



Elevating Legrest



Raise Facility



Electric



Made to Measure

HALLINGBURY RANGE

Underpinned by **Fitform Vario triple motor** technology this chair enables the user to operate the **tilt-in-space**, **legrest elevation** and **back recline** independently of each other. What sets the **Hallingbury Range** apart is its **traditional British Style**.

Contact us
for your **free**
assessment!

*Hallingbury Waterfall Back in
Enduratex Prairie Turtlecreek
and Graphite Dartex*

*Hallingbury Lateral Support
Waterfall Back in Panvelle Stretch
Monaco Mulberry*



Built in the UK and designed to emulate **traditional British aesthetics**, the **Hallingbury range** is the **perfect combination of comfort and style**.

While the seat width must be selected at time of order, seat depth and height can be adjusted to fit the user perfectly. With a wide range of backrest styles available too, tailoring the chair to the individual is easily achieved.

Features and Benefits

- Triple Motor independently operates Raise Facility, Tilt-in-Space, Back Recline and Legrest Elevation
- Five back options: T-Back, Firm Back, Soft Back, Lateral Support Waterfall Back, Waterfall Back
- Tailored seat height, arm height, seat depth and seat width
- Quality British design

Accessories

- Head Cushion
- Adjustable Lateral Supports

Weight Limit

- 110kg / 17 stone
- Legrest weight limit 60kg / 9½ stone

Pressure Relieving Seat Options

- Intelli-Gel® Integral seat module
- Reflexion™ Foam seat module

- 1 T-Back
- 2 Firm Back
- 3 Soft Back
- 4 Waterfall Back
- 5 Lateral Support Waterfall Back



DIRECT UPHOLSTERY FABRICS

Please note: Some fabric options shown in photography may no longer be available.
Colour shading may vary from time to time.

Vinyl: Venture Price Band **CH**



Hyacinth



Rose




Mulberry



Duck Egg

Vinyl: Cadet Trek Price Band **CH**

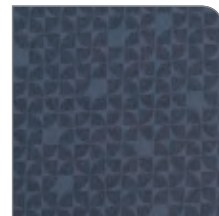

Indigo



Rose



Seafoam

Vinyl: Wild Heather



Spey Price Band **CH**

Vinyl: Etch Price Band **CH**




Wedgewood



Mauve




Peony

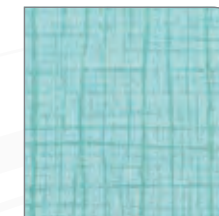

Ocean


Vinyl: Berkshire Price Band **CH**

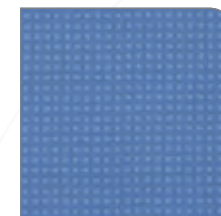

Boxford



Fawley

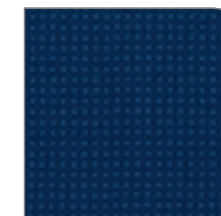


Caversham




Purley

Vinyl: Enduralex Prairie Price Band **A**

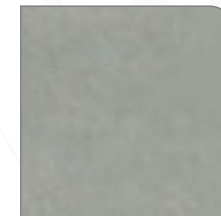

Sorrel

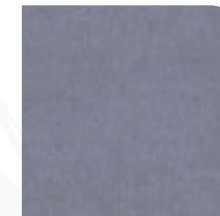

Raindance



Turtlecreek




Duststorm

**Need help in
choosing the right
fabric for you?**

Feel free to contact our
sales team who will be
more than happy to
talk you through the
available options.

Vinyl: Brookland

Price Band A

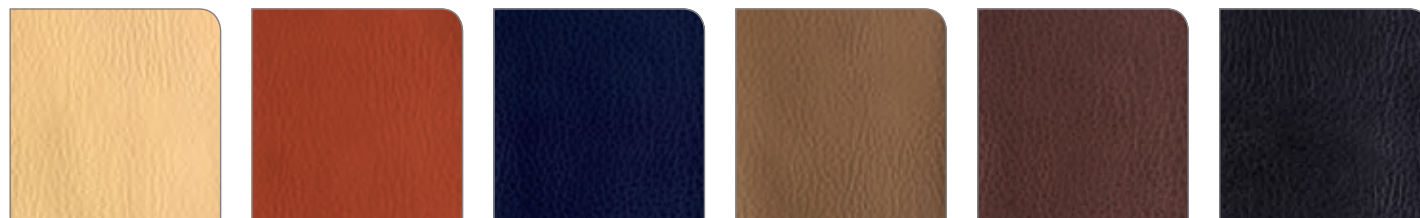

Chablis

Red

Navy

Mushroom

Espresso

Black

Vinyl: Vyflex

Price Band A




Jade

Navy

Rose

Panvelle Stretch: Plain

Price Band A

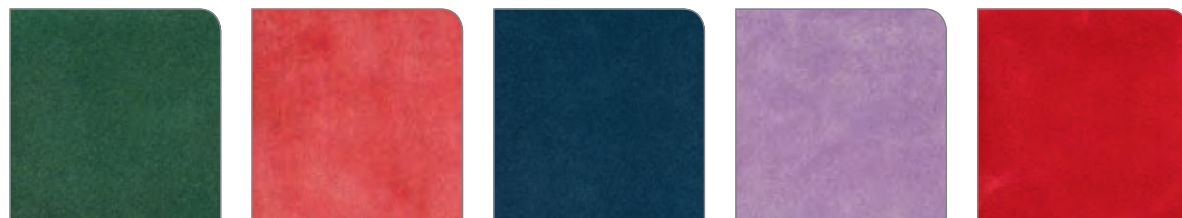

Spruce

Pink

Indigo

Lilac

Crimson

Panvelle Stretch: Spirit

Price Band A



Blue

Burgundy

Panvelle Stretch: Sonnet

Price Band A



Mauve

Beige

Mushroom

Damson

Panvelle Stretch: Monaco

Price Band A



Black/Chalk

Grey

Fern

Mulberry

Azure

Panvelle Stretch direct upholstery is a unique anti-bacterial fabric that is water-resistant, yet breathable and warm to the touch.

Vinyl is a water-resistant, anti-microbial, anti-fungal direct upholstery material suitable for demanding applications. Vinyl is highly recommended for hospital environments.

All Panvelle Stretch and Vinyl materials have been Fire tested to BS 5852 ignition source 0, I and 5. Please contact us for further details.


Cleaning for Panvelle Stretch and all Vinyl material: Routine soap and tepid water with a microfibre cloth is effective for ordinary soiling.

LOOSE COVER FABRICS

Panaz Trevira is a loose cover cotton material that can be easily cleaned.

Cleaning: Cool machine wash at 40°C.


Panaz Trevira: Eva

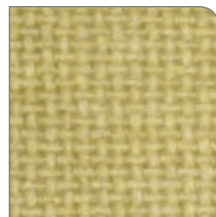


Fern



Azure

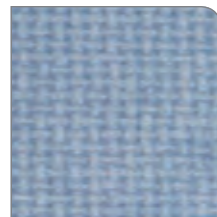

Hyacinth

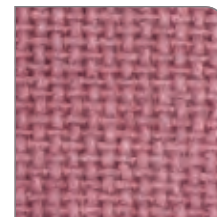

Mulberry

Panaz Trevira: Monaco



Black/Chalk



Grey



Fern




Mulberry



Azure

Panaz Trevira:
Spirit

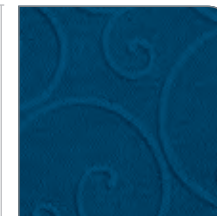

Blue



Burgundy

DARTEX

To ensure the pressure relieving properties of your chair are working to their full potential we highly recommend the use of Dartex on all user contact surfaces.

Dartex is a **strong, multi stretch, vapour permeable** fabric which is simple to **wipe clean** and truly complements the **pressure relief** incorporated into our chairs.

This waterproof material is available in **three** different colours.

Dartex:



Navy



Black



Graphite



Please note:

Some chairs are produced with Dartex as standard - contact us for details or refer to the current Price List or website.

FITFORM FABRICS

Fitform: Niroxx Meets BS 5852 CRIB 5



Navy Blue

Cream

Aubergine

Chocolate

For Fitform only

+ Standard Stocked Fabrics 8-10 weeks delivery

++ Non-Standard Stock Fabrics 12 weeks delivery

Fitform: Leatherette Meets BS 5852 CRIB 5



Green ++

Pink +

Brown +

Blue ++

Beige ++

Black ++

Fitform: Leather
A natural material for domestic use only.



Black



Forest Green

Fitform: Divine Velour Meets BS 5852 CRIB 5. 100% polyester for domestic and contract use.



Pebble +

Tobacco +

Denim +

Red +

Forest Green +

Chocolate Brown +



Burgundy



10 Year Frame Warranty for Dynamic Specialist Seating Products

All dynamic chair frames supplied by Kirton can now appreciate a warranty period of 10 years without the need to necessarily undertake an annual service. We do however, fully recommend that all dynamic Kirton chairs undergo an annual maintenance service in order to extend the lifespan of the chair and to ensure it is working at its optimum.

(subject to current Terms & Conditions of Sale and Warranty & Liability Guidelines).

The dynamic seating range comprises of the **GE-II, Encora, Delta, Duo, Omega, Florian II, Florian Fife** and **Florien Elite**.

Bespoke Seating and Upholstery Solutions

We are often requested to re-upholster an existing product or produce a bespoke item from a simple footstool to an architect's plan for banquette seating. We are happy to undertake individual projects, consider new ideas or investigate concepts.

These projects are undertaken on a case by case, individual-cost basis. We would be delighted to discuss any requirements you may have and we will do our utmost to find a solution that works for you and your budget.



REFURBISHMENT SERVICE - REVIEW, REFRESH, RE-USE



All Kirton chairs work hard for their clients however over time and through constant use they do become worn. In many instances the framework of the chair is in near perfect condition but worn, soiled and tired upholstery often means the chair has to be taken out of service.

With this in mind we offer a full refurbishment programme aiming to maximise the lifespan of a Kirton chair through replacement of the modular parts and components to ensure that it continually performs. This service complements our ISO 14001 Environmental Management System to minimise waste and assist in the recycling of product as well as providing a cost effective solution to our customers.

What is the Refurbishment Service?

- An initial consultation to assess the extent of the refurbishment required.
- A quotation will then be issued for replacement parts and any necessary work required.
- Upon receipt and manufacture of an order; a Service Technician will call to arrange a convenient time to visit and complete the work.
(Please note not all refurbishments can take place on-site).

An initial service call for a product assessment will cost **£130.00** per visit. All components and parts will be quoted and charged separately.

A quick turn around of product is intrinsic to the refurbishment programme; we will endeavour to minimise the time the chair is out of action.

Should you require any further information or to arrange a visit from one of our Service Technicians please contact our Customer Service Team

Are your chairs looking tired and worn out?
Does the upholstery look scuffed?
Have the seams worn and the covering split?
Have you lost a wheel?

CASE STUDY

Doncaster Community Equipment Stores providing an enhanced service at lower cost

At **Kirton** we pride ourselves on delivering **high quality, cost effective seating** to help make **life easier**.

Working closely with Community Equipment Stores and Health Authority Discharge Teams, we understand the challenges of providing a quality service within the constraints of strangled budgets - we want to help!

We can assist along with our distribution partners to ensure:

- Superior Seating Provision
- Cost Savings
- Improved Stock Control
- Better Time Management

Our work with Doncaster CES is just one example of how working together we can make a difference!

The Problem:

Doncaster Community Equipment Service (CES) had never provided specialist seating through a formal standardised stock offer.

The provision of riser recliner and tilt-in-space chairs had been through a 'new purchase specials' process.

"This led to a fragmented, costly and slow service".

The Solution:

- Doncaster CES selected Kirton Healthcare to provide three modular chairs to meet the majority of service user needs.
- Kirton provided free training programmes for prescribers to enhance their product knowledge and increase reuse and recycling rates.
- Only prescribers who attend training are able to order specialist seating.
- Kirton built a partnership with NRS Healthcare to effectively deliver this programme.

The Results:


Average time taken from order date to installation for stocked specialist seating reduced from **33.83 days** to **6.75 days**

Average number of people Doncaster could help by providing specialist seating **Grew by 100%**

Average cost per chair was reduced by over **£1,000**

Average number of special chair purchases reduced by **Over 85%**

DUO

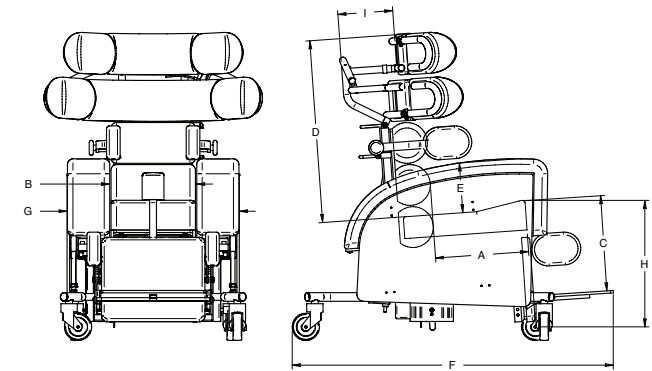

Duo	Major	Minor
A: Seat Length	380-500	320-460
B: Seat Width	350/400/450/500	300/350/400/450
C: Seat Height (from footboard)	350-500	300-450
D: Back Height	670-940	610-820
E: Arm Height	180-300	180-300
F: Overall Length	1180	1030
G: Overall Width	680-830	620-770
H: Seat Height (from floor)	510-630 (Manual version: 510)	500-620 (Manual version: 500)
I: Headrest Depth Adjustment	100	80


Tilt-in-Space:	Horizontal to 35° backwards (28° backwards on Manual version) Regulated Motion Tilt-in-Space range between 10° and 25°
----------------	--

Back Recline	90° to 130°	90° to 130°
Negative Legrest Angle:	-25°	-25°
Footrest Adjustable Angle:	60°, 75°, 90°, 105°, 120°	60°, 75°, 90°, 105°, 120°

Seat Modules:	All seat modules can be made to measure on request.
---------------	---

All measurements in mm


ENCORA

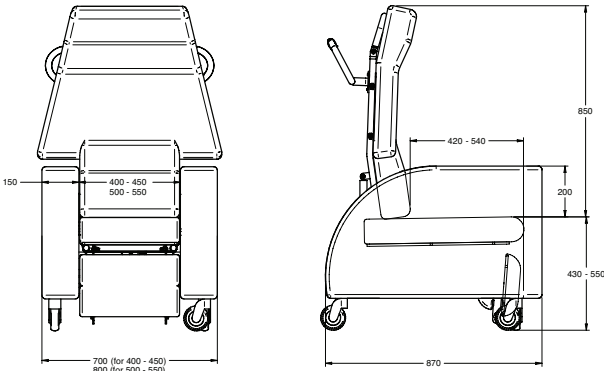


Need further assistance?

Don't forget we offer free assessments so if you need help choosing the right specification, just give us a call!

GE-II

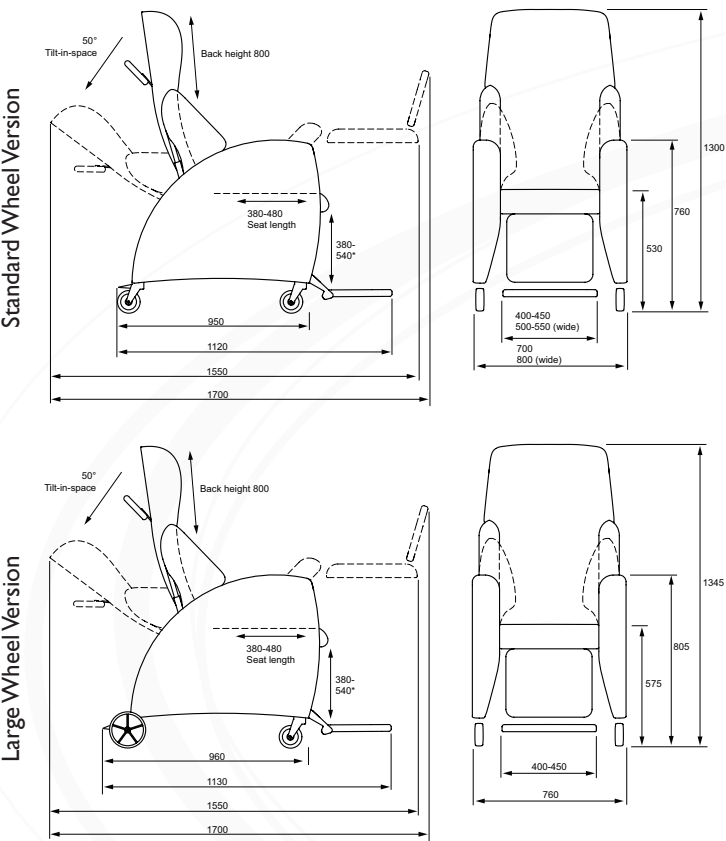


DELTA

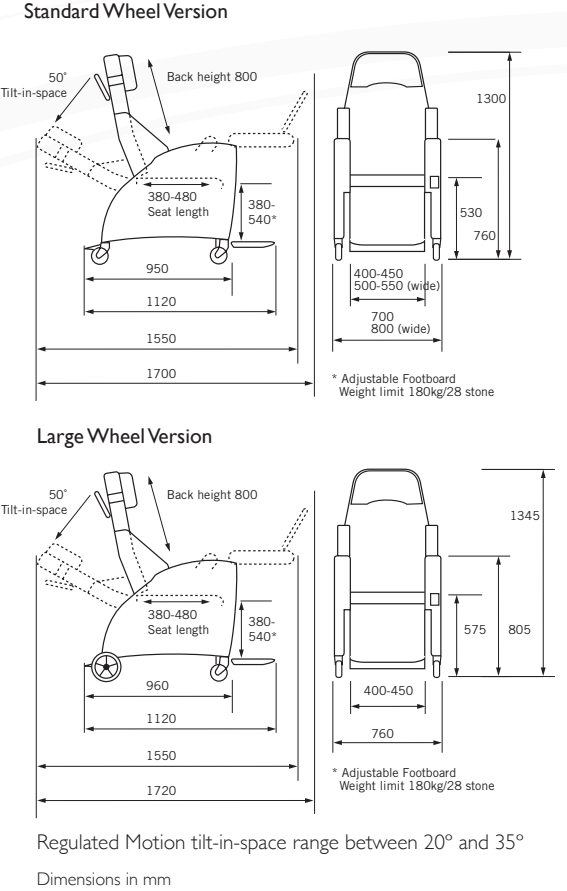


FLORIEN FIFE or FLORIEN II


Standard Wheel Version

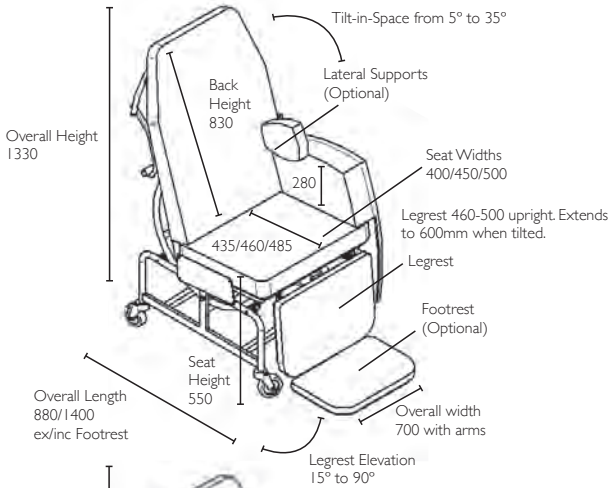



Large Wheel Version

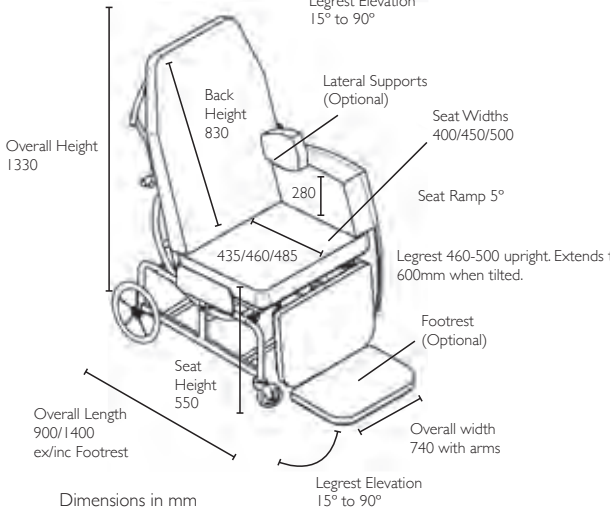


FLORIEN ELITE

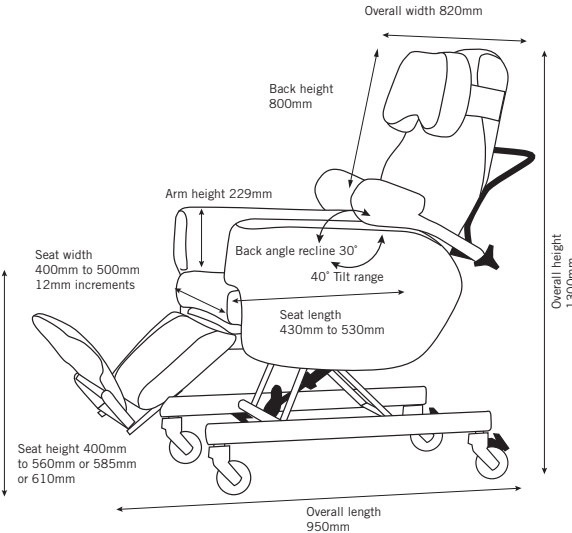

OMEGA

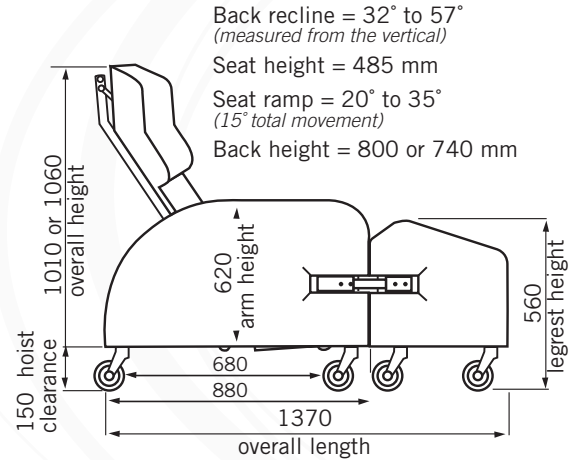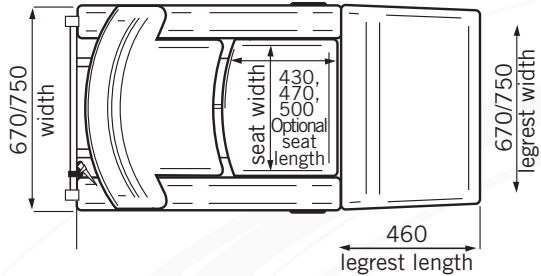

REGENT

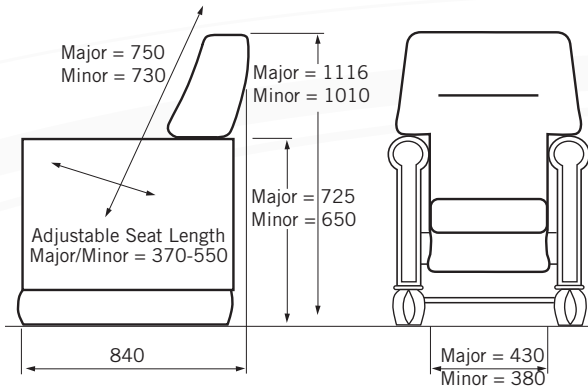


Dimensions in mm

SEAT HEIGHT & ANGLE OPTIONS



One option per chair
Chairs with reclining backs can only have
75° seat module

Seat Angle	Minor Seat Height	Major Seat Height
70°	480 mm	565 mm
75°	425 mm	525 mm
80°	380 mm	480 mm
87°	340 mm	425 mm

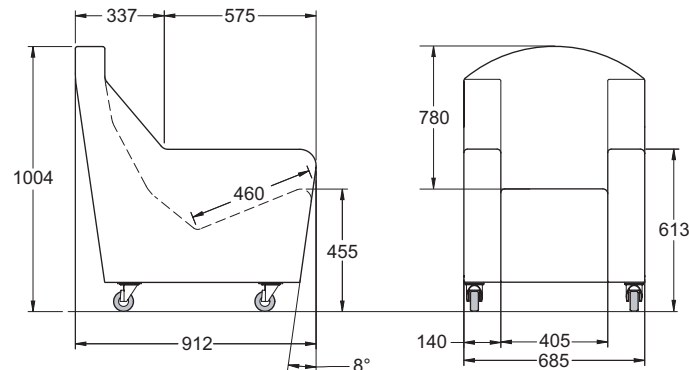

Measurements in mm

SOVEREIGN

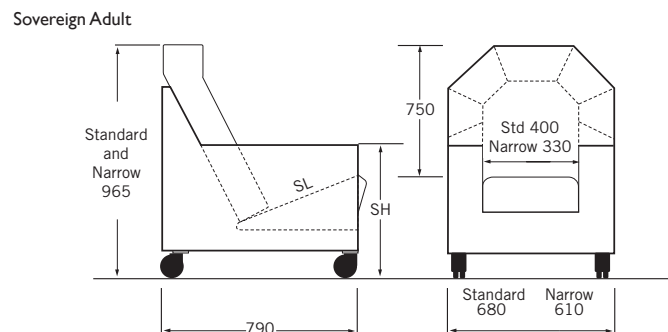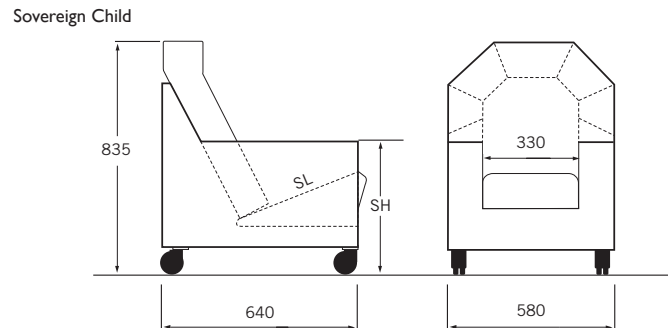

Seat Module:

Adult Std / Narrow	20°L	20°S	15°L	15°S
Seat Height	440	410	380	350
Seat Length	470	410	440	380
Child	15°S	20°L	25°L	-
Seat Height	280	345	375	-
Seat Length	360	410	440	-

FITFORM 570 VARIO

- | | |
|---|--|
| Triple Motor | • |
| Raise Facility | • |
| Tilt-in-Space | • |
| Back Recline | • |
| Elevating Legrest | • |
| Adjustable Integral Lumbar Support | • |
| Multi Adjustable Integral Headrest | • |
| Seat Height Options | between 40 cm-60 cm |
| Armrest Height Options | between 16 cm-29 cm |
| Seat Depth Options | 43 cm, 45 cm, 47 cm, 49 cm, 51 cm, 53 cm |
| Seat Width Options | 51 cm (std), 45 cm, 57 cm |
| Backrest Height | 84 cm |
| Side Pocket | • |
| Weight Limit | 150 kg / 23½ st |
| Dynamic Legrest Weight Limit | 30 kg / 5 st |
| Optional Accessories | |
| Intelli-Gel® Integral Cushion | • |
| Transflo Gel Seat Cushion | • |
| Air Positioning & Pressure Relief System | • |
| Electrically heated seat and backrest | • |
| Electrically inflatable lumbar support | • |
| Electrically adjustable head support & positional memory function | • |
| Chrome Feet | • |
| Wheels (for seat height less than 47 cm) | • |
| Wheels (for seat height greater than 47 cm) | • |
| Swivel Armrest for Side Transfer | • |
| Head Cushion | • |
| *RCPM System | • |
| Extended Leg Support (+8 cm) | • |
| Fabric: Leatherette | • |
| Fabric: Velour | • |
| Fabric: Leather | • |
| Fabric: Niroxx | • |
| Mobility Kit | • |
| Flexible Halogen Lamp | • |
| Adjustable Kyphosis Back | • |
| Bucket-Style Back | • |

FITFORM 570 VARIO HERMES

- | | |
|---|--|
| Triple Motor | • |
| Raise Facility | • |
| Tilt-in-Space | • |
| Back Recline | • |
| Elevating Legrest | • |
| Adjustable Integral Lumbar Support | • |
| Multi Adjustable Integral Headrest | • |
| Seat Height Options | between 40 cm-60 cm |
| Armrest Height Options | between 16 cm-29 cm |
| Seat Depth Options | 43 cm, 45 cm, 47 cm, 49 cm, 51 cm, 53 cm |
| Seat Width Options | 63 cm, 75 cm, 81 cm, 87 cm |
| Backrest Height | 84 cm |
| Side Pocket | • |
| Weight Limit | 240 kg / 40 st |
| Dynamic Legrest Weight Limit | 28 kg / 4½ st |
| Optional Accessories | |
| Intelli-Gel® Integral Cushion* | • |
| Transflo Gel Seat Cushion | • |
| Air Positioning & Pressure Relief System | • |
| Electrically heated seat and backrest | • |
| Electrically inflatable lumbar support | • |
| Electrically adjustable head support & positional memory function | • |
| Chrome Feet | • |
| Wheels (for seat height less than 47 cm) | - |
| Wheels (for seat height greater than 47 cm) | - |
| Swivel Armrest for Side Transfer | • |
| Head Cushion | • |
| *RCPM System | - |
| Extended Leg Support (+8 cm) | • |
| Fabric: Leatherette | • |
| Fabric: Velour | • |
| Fabric: Leather | • |
| Fabric: Niroxx | • |
| Mobility Kit | - |
| Flexible Halogen Lamp | • |
| Adjustable Kyphosis Back | - |
| Bucket-Style Back | • |

*When ordering this product please order two Intelli-Gel® Integral Cushions

FITFORM 570VARIO PETITE

Triple Motor	•
Raise Facility	•
Tilt-in-Space	•
Back Recline	•
Elevating Legrest	•
Adjustable Integral Lumbar Support	•
Multi Adjustable Integral Headrest	•
Seat Height Options	between 40 cm-60 cm
Armrest Height Options	between 16 cm-29 cm
Seat Depth Options	44 cm, 47 cm, 50 cm, 53 cm
Seat Width Options	45 cm only
Backrest Height	72 cm
Side Pocket	•
Weight Limit	150 kg / 23½ st
Dynamic Legrest Weight Limit	30 kg / 5 st
Optional Accessories	•
Intelli-Gel® Integral Cushion	•
Transflo Gel Seat Cushion	•
Air Positioning & Pressure Relief System	•
Electrically heated seat and backrest	•
Electrically inflatable lumbar support	•
Electrically adjustable head support & positional memory function	•
Chrome Feet	•
Wheels (for seat height less than 47 cm)	•
Wheels (for seat height greater than 47 cm)	•
Swivel Armrest for Side Transfer	•
Head Cushion	•
*RCPM System	-
Extended Leg Support (+8 cm)	-
Fabric: Leatherette	•
Fabric: Velour	•
Fabric: Leather	•
Fabric: Niroxx	•
Bucket-Style Backrest	•

HALLINGBURY

	Standard	Options
Seat height	500 mm	450 mm – 550 mm
Seat width	510 mm	450 mm or 570 mm
Seat depth	Firm or Soft Back	400 mm, 420 mm, 440 mm, 460 mm, 480 mm, 500 mm
	Waterfall, Lateral	
	Waterfall or T-Back	350 mm, 370 mm, 390 mm, 410 mm, 430 mm, 450 mm
Arm height	7"/180 mm	
Back height	29 ½"/750 mm	
Max user weight	17 st / 110 kg	
Dynamic Legrest Weight Limit	9 ½ st / 60 kg	



SUITES

Designed for high traffic communal areas and challenging or critical environments, Kirton Suites are built with a solid and robust frame at their core with a modern, stylish exterior.

Whether furnishing a communal lounge, reception or community home environment, this contemporary range of suites integrates effortlessly. Discover more at www.kirton-healthcare.co.uk



Jasper




Calgary



Toronto



Kelowna

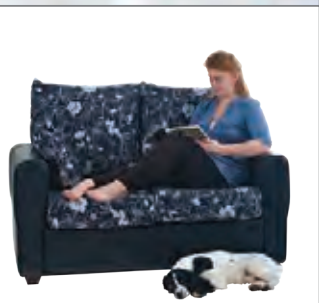

Banff

Terms & Conditions

Full terms and conditions can be found on our website: www.kirton-healthcare.co.uk along with a wealth of additional information regarding Kirton products and services. The full Kirton Price List can be viewed online too.

If you do not have access to our website, or would prefer to receive a hard copy of our current prices or terms and conditions, please call us on **0800 212709 / +44 (0) 1440 705352** and ask a member of our customer services team.



INNOVATION TEAM ACTION QUALITY CUSTOMER INTEGRITY FOCUSED

Our Core Values

At the Kirton Healthcare Group we understand that working together is key to achieving our collective goals.

Working together is much easier when we are all heading in the same direction; otherwise we lose our way, duplicate activities or, even worse, don't act at all.

But how can we be sure that we are all following the same path and pulling together to reach our full potential?

We need a culture. A way of working that everyone adheres to and no one is willing to compromise on. A set of standards and behaviours that each member of the Kirton Healthcare Group demonstrates with confidence and pride, or rather our Core Values





Making everyday
tasks a little easier.

The Kirton Healthcare Group Ltd
23 Rookwood Way
Haverhill
Suffolk CB9 8PB
UK

Telephone: +44(0)1440 705352
Facsimile: +44(0)1440 706199
E-mail: info@kirtonhealthcare.co.uk

Customer Helpline: 0800 212709
www.kirton-healthcare.co.uk
www.kirton-healthcare.com



Made in Britain

