

System**RoMedic**™

# MilkyWay

A flexible range of ceiling lift and overhead track solutions for all spaces

**Direct Healthcare**  
**Group**

Advancing Movement & Health®

# MilkyWay rail system

## – unlimited possibilities

A stationary lifting solution, consisting of a ceiling lift unit mounted on a permanently installed rail system or on a freestanding gantry/stand. Milkyway is an ergonomic favourite, an ideal concept that eliminates the need for manual lifting and allows the caregiver to work in a safe and comfortable moving and handling environment.

An installed lift system is always in place and available whilst the gantry choice requires a minimum of floor space and is very easy to manoeuvre to reposition. With a wide choice of track sizes, lengths and configurations both systems can cover a large lifting area, fit into small rooms, all with the potential of low reach and very high clearance lifting.

### A permanently installed rail system is the right solution when:

Each MilkyWay rail system is individually designed in close cooperation with the customer, to ensure adaptation to specific conditions and requirements.

Together with an optional choice of ceiling lift unit and the appropriate lifting accessories, MilkyWay offers unlimited possibilities for design and adaptation of the perfect stationary lift solution for all settings, rooms and lifting needs.

### Easy to install

MilkyWay offers great flexibility and can be adapted to most environments and settings. The system is very easy to install and there is a design and solution for most situations including new buildings or those being adapted. The MilkyWay rail system must always be installed by qualified and authorised personnel and in accordance with instructions issued by Direct Healthcare Group. MilkyWay has been developed to meet the highest safety standards of MDD and ISOEN 10535.



# A stylish and economic system

## Ceiling fixtures and pendants

MilkyWay offers several possibilities for installation in the ceiling. Depending on factors such as ceiling height, material and structure, you can choose between ceiling brackets and/or height adjustable pendants of different types. With the flexibility and range of its components and fixings including uni-strut and side supported pendants, MilkyWay can be installed into most roof spaces including high voids and into most building materials and designs.




## Wall fixtures

Similarly, MilkyWay also offers several alternatives for wall mounting of free bearing rail systems. Again, the choice of fixtures depends on the wall structure and material, as well as the load that the system will carry.



## Rails and trolleys

The MilkyWay range offers several possibilities of straight rails and side curves to build the overall solution that is required, as well as trolleys that can be used with the RiseAtlas range as well as ceiling lifts from other suppliers. All rails and side curves are prepared for in rail charging. Additionally and for that "stylish look", MilkyWay offers a "recessed ceiling track option", very popular in modern buildings.

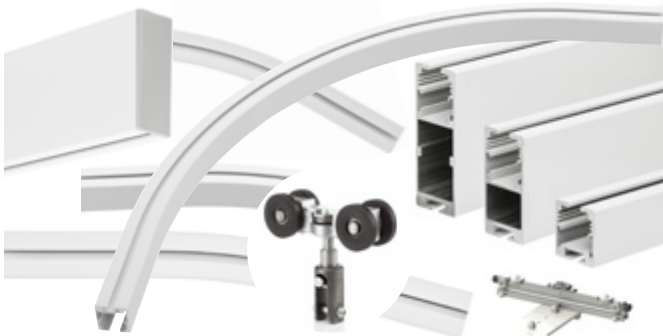

## Power supply


MilkyWay can be installed with in-rail charging, with a constant power supply (24 V DC) built into the rail. This means that the ceiling lift unit can be supplied with power all the time, everywhere in the rail system, eliminating the need for a charger station. Of course and if preferred, MilkyWay rail systems can also be installed with one or several charger stations for the lift unit at predetermined locations on the rail system.



## Switches

Switches are used to connect a single rail or rail system to another, allowing the ceiling lift unit to travel across the system for increased room coverage, flexibility and utility.

MilkyWay includes shunting rails, turntables and transition gates for all kinds of rail systems.

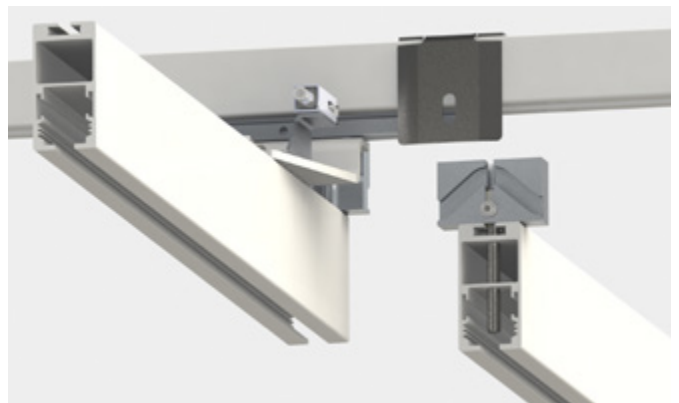

## Choose lift unit according to the lifting needs at hand

Depending on factors such as the required lifting capacity, the setting and situations in which the system is to be used and, of course, the users' needs, MilkyWay can be used in combination with many different ceiling lift units. For more information, please visit our website: [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com).



Art. No.	Product
50100039	RiseBasic300M, SWL 138kg/21 stone.
50100040	RiseBasic440M, SWL 200kg/31 stone.




Art. No.	Product
50100057	RiseAtlas450M, with QuickTrolley-System, SWL 205kg/32 stone.
50100051	RiseAtlas450M, with QuickTrolley-System, high humidity, SWL 205kg/32 stone.
50100058	RiseAtlas450T, SWL 205kg/32 stone.
50100052	RiseAtlas450T, with SmartPark-function, SWL 205kg/32 stone.
50100059	RiseAtlas625M, with QuickTrolley-System, SWL 285kg/44 stone.
50100060	RiseAtlas625T, SWL 285kg/44 stone.
50100053	RiseAtlas625T, with SmartPark-function, SWL 285kg/44 stone.

### More information

For more information about MilkyWay; available parts, instructions for design and installation and step-by-step examples of different types of installations, please consult the "MilkyWay Technical information" and/or "MilkyWay Handbook" documents or contact us or your local Direct Healthcare Group partner.

All contact details can be found on our website, [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com)

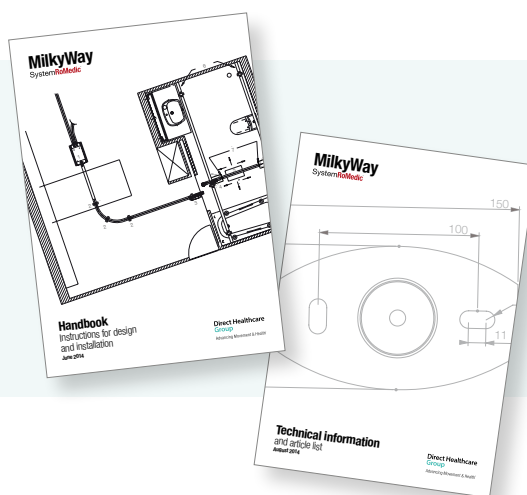

SystemRoMedic™ is a complete solution for easy transfers. The concept encompasses four product areas that meet most needs in different transfer situations.

**Transfer**, between two locations. **Positioning**, within the same location. **Support**, for mobility. **Lifting**, both manual and mechanical.

**Direct Healthcare Group**

Advancing Movement & Health®

Withey Court, Western Industrial Estate  
Caerphilly, United Kingdom CF83 1BF  
T: +44 (0) 800 043 0881  
F: +44 (0) 845 459 9832  
E: [info@directhealthcaregroup.com](mailto:info@directhealthcaregroup.com)  
[directhealthcaregroup.com](http://directhealthcaregroup.com)