ThoraxSling

System**RoMedic**

Instructions for use – English

Bruksanvisning – Svenska

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Návod na použitie – Slovensky

ThoraxSling

REF

45500003
45500004
45500006
45500007
45500008
45500009

ThoraxSling with
Seat Support

REF

45600004
45600006
45600007
45600008

ThoraxSling with Seat
Support Disposable

REF


45690004
45690006
45690007
45690008

SWL: 300 kg/ 660 lbs


SWL Disposable: 200 kg/ 440 lbs

Table of contents

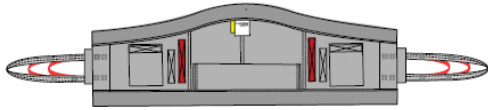
English	3
Svenska	7
Norsk	11
Dansk	15
Suomi	19
Deutsch	23
Français	27
Italiano	31
Český	35
Polski	39
Español	43
Slovensky	47
Symbols	50

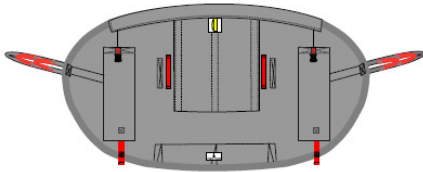ThoraxSling, polyester
SWL: 300 kg/660 lbs

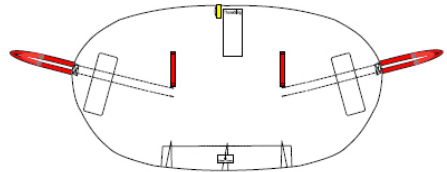ThoraxSling, with seat support, polyester
SWL: 300 kg/660 lbs

ThoraxSling, with seat support, disposable
SWL: 200 kg/440 lbs

ThoraxSling is a sit-to-stand sling for MiniLift that is easy for the caregiver to apply, while giving the patient a feeling of security. It is designed to meet the need for support and security during raising and to facilitate visits to the toilet. ThoraxSling is available in two models: one that provides support behind the back and one that also gives support under the seat. ThoraxSling with seat support is also manufactured in disposables.

SystemRoMedic includes a wide range of functional, comfortable, high-quality slings that can be adapted for different types of lifting and for patients with different needs. The choice of model and material depends on the patient's needs and the transfer situation.

Functional inspection

Visual and mechanical inspection

Check the condition and function of the sling regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. Apply load to the device and check to ensure that clasps, handles, etc. withstand heavy load. If there are signs of wear, the product must be discarded.

Always read the manual

Always read the manuals for all assistive devices used during a transfer.

Keep the manual where it is accessible to users of the product.

Do not leave the patient unattended during a transfer situation.

The lowest allowable safe working load always determines the safe working load of the assembled system.

Always check the safe working loads for the lift and accessories before use.

Contact your dealer if you have any questions.

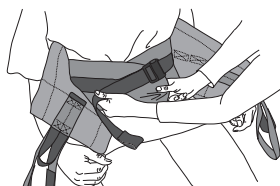
Description of the strap loops

Four different application alternatives are possible, since the strap loops have three different levels.

Choose the shortest strap loop that reaches the slingbar on the lift.


Description of waist belt

The waist belt provides support and a feeling of security during raising.

When it is connected, and as extra support is needed, it can be tightened by pulling the loose strap.

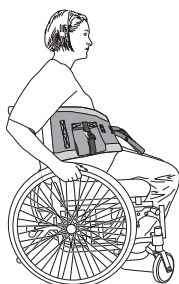Application of ThoraxSling

Place ThoraxSling behind the patient's back with the label facing out. The vest is placed so that it does not cause pressure in the underarms. The arms are placed outside. This is easier if the patient leans forward or can be helped to lean forward slightly. The mid-section of the vest should follow the patient's spine.

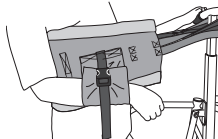
In combination with a seat support, ThoraxSling is applied in the same way, but with the seat support placed under the patient's seat. Grasp the reinforced seat support and place it under the patient's seat.


The waist belt is connected and adapted to the size of the patient.

For patients who need arm support, for example, patients suffering from shoulder arthrosis, there are arm support holders for both sides of the vest.

Application of ThoraxSling on MiniLift

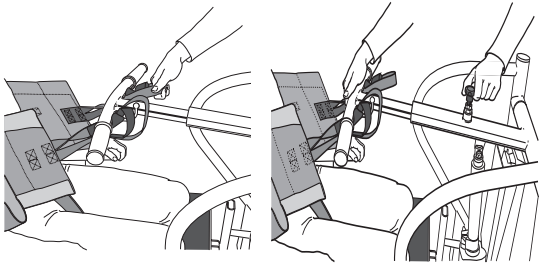
Place the individually adjusted sit-to-stand lift in front of the patient (see lift manual for instructions for trial adjustment) and place the patient's feet on the footrest. If a calf strap is to be used, it is applied first. Draw out the boom towards the patient for easier application of the strap loops.


Carefully place the lift as close as possible to the patient, so as to determine which strap loop is the most appropriate. Then, extend the straps before hooking them to the slingbar.

Application of ThoraxSling on MiniLift

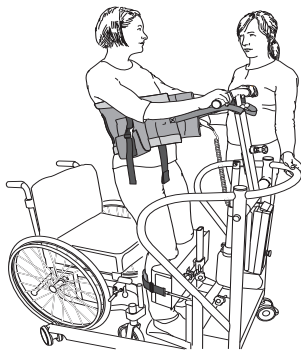Ask the patient to lean back against the vest before you start to raise the lift. Pay close attention to the patient. When everything is adjusted and the patient is comfortable, raise the lift to the desired height. Discontinue raising if the patient shows the least sign of discomfort.

ThoraxSling

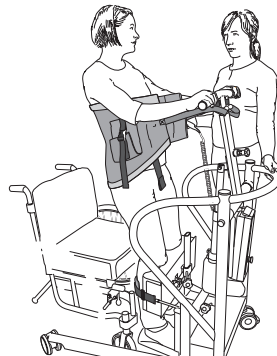ThoraxSling with seat support

Lifetime information & Periodic inspection

The product has an expected lifetime depending on how the product is used according to the manual instructions.

The service life of the product varies depending on usage frequency, washing, drying, etc.

Please be sure to read the product label and follow the care instructions for this product on www.directhealthcaregroup.com

The periodic inspection of the non-rigid body support unit should be performed at the timestamps stated by the manufacturer, but at least every 6 months to be sure. More frequently inspections may be required if a non-rigid body support unit is used or cleaned more frequently than advised by the manufacturer.

The periodic inspection instructions for this product are found on www.directhealthcaregroup.com.

Material:

Polyester: Durable material with low friction.

Disposable: Polypropylene, hydrophobic

Care of the product, polyester:

Read the product label.

Do not use rinsing agent. To ensure maximum material life, hang dry or tumble dry on low heat.

Care of the product, non-woven:

Read the product label.

Do not launder. Discard the product if it is soiled or if there is a need to use another product.

Size information


Use patient-specific slings to ensure safety and hygiene.

Size		XS/Orange	S/Red	M/Yellow	L/Green	XL/Purple	XXL/White
Rec. patient weight	kg	20-50	30-60	50-80	70-110	100-150	140-200
	lbs	44-110	66-132	110-176	154-242	220-331	309-441
Sling width	cm	85	94	104	124	146	170
	inch	33.5	37	41	49	57.5	67


Contact your local distributor if you have any questions about the product and its use.

See www.directhealthcaregroup.com for a complete list of distributors.

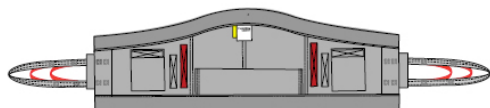
Always make sure that you have the right version of the manual. The most recent editions of manuals are available for downloading from our website, www.directhealthcaregroup.com.

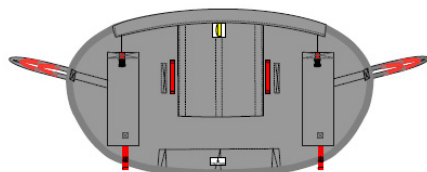ThoraxSling, polyester
SWL: 300 kg/660 lbs

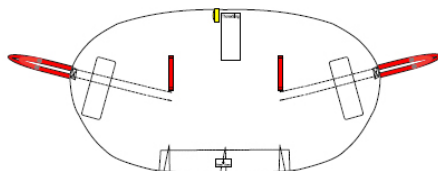ThoraxSling, with seat support, polyester
SWL: 300 kg/660 lbs

ThoraxSling, with seat support, disposable
SWL: 200 kg/440 lbs

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Symbols

	This product complies with the requirements of the Medical Device Regulation 2017/745		Visual Inspection
	Medical Device		Read the manual
	Caution		Product Code
	Manufacturer information		Batch Code

SystemRoMedic™

Simple solutions for great results

SystemRoMedic® is the name of Direct Healthcare Group's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic® is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic® is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic® offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website;
www.directhealthcaregroup.com

For questions about the product and its use

Please contact your local Direct Healthcare Group and SystemRoMedic® representative. A complete list of all our partners with their contact details can be found on our website; www.directhealthcaregroup.com.

Moving Health Forward

Direct Healthcare Group Sverige AB
Torshamnsgatan 35,
SE-164 40 Kista, Sweden

Tel: +46 (0)8-557 62 200
info@directhealthcaregroup.com
www.directhealthcaregroup.com

Medical Device Class I. The product complies with the requirements of the Medical Devices Regulation 2017/745.