# WendyLean

SystemRoMedic

Instructions for use – English

Bruksanvisning – Svenska

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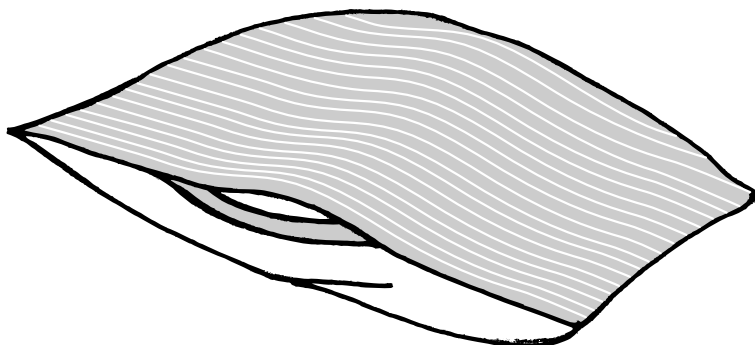
SWL: 200 kg/440 lbs

## Table of contents

English .....	3
Svenska .....	5
Norsk .....	7
Danske .....	9
Suomi .....	11
Deutsch .....	13
Nederlands .....	15
Français .....	17
Italiano .....	19
Español .....	21
Symbols .....	22
Accessories .....	23

WendyLean is a pillow case with an anti-friction surface on the underside and handles on both sides. The anti-friction surface facilitates positioning of the pillow. WendyLean is used for repositioning higher up in bed, since it glides easily against the underlying surface. With the help of the handles, it can also be used to help the patient into a seated position at the edge of the bed.

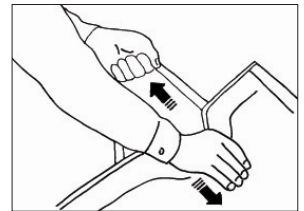
## Check Safety

### Visual inspection

Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.

### Mechanical load

Apply pressure to buckles and handles and verify that they will not open or break under heavy loads.

### Friction test

Low friction - Ensure that the surface is smooth and slides effortlessly. Compare with a new product.

## Always read the manual

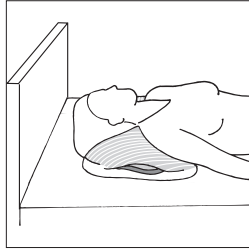
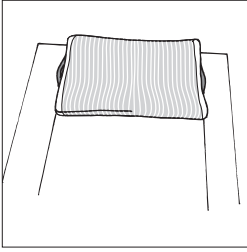
Always read the user manuals for all assistive devices used during a transfer.

Keep the user manual where it is accessible to users of the product.

Always make sure that you have the right version of the user manual.



The most recent editions of user manuals are available for downloading from our website, [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com).

## Placement

Place the pillow in the pillow case. The anti-friction surface (underside) faces downward. To use the handles, place the pillow in position under the shoulders.

## Using WendyLean

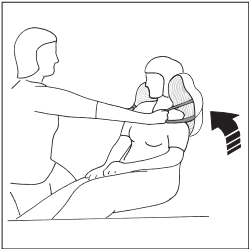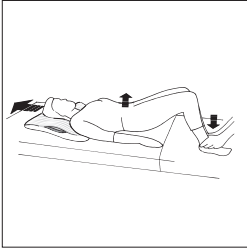To move higher up in bed, the patient bends his/her knees, lifts the hips slightly, and then pushes with the feet.

WendyLean can be used for support behind the head/shoulders, and to help the caregiver to assist the patient to rise from a horizontal to a seated position. The caregiver holds the handles.

## Material

Polyester, cotton, satin

## Care of the product

Read the product label.

Do not use rinsing agent. To ensure maximum material life, hang dry or tumble dry on low heat.

Contact your local distributor if you have any questions about the product and its use.

See [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com) for a complete list of distributors.

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## Vida útil estimada

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## Symbols

	This product complies with the requirements of the Medical Device Regulation 2017/745		Visual Inspection
	Medical Device		Read the manual
	Caution		Product Code
	Manufacturer information		Batch Code

## Accessories

Art. no.	Product	Size cm/inch	Colour
----------	---------	--------------	--------

### WendyLett - sliding sheet

(Can be combined with WendyLett2Way drawsheet or WendyLett4Way sliding drawsheet)

1609	WendyLett, 60 cm/24" sliding surface ( <i>bed 80/90 cm - 32"/36"</i> )	100 x 200/39 x 79	White
1614	WendyLett, 60 cm/24" sliding surface ( <i>bed 80/90 cm - 32"/36"</i> )	140 x 200/55 x 79	White
1629	WendyLett, 60 cm/24" sliding surface ( <i>bed 80/90 cm - 32"/36"</i> )	100 x 200/39 x 79	grey/white striped
1634	WendyLett, 60 cm/24" sliding surface ( <i>bed 80/90 cm - 32"/36"</i> )	140 x 200/55 x 79	grey/white striped
1639	WendyLett, 70 cm/28" sliding surface ( <i>bed 105 cm/42"</i> )	200 x 200/79 x 79	grey/white striped
1667	WendyLett, 100 cm/40" sliding surface ( <i>bed 120 cm/48"</i> )	100 x 220/39 x 88	grey/white striped
1668	WendyLett, 100 cm/40" sliding surface ( <i>bed 120 cm/48"</i> )	140 x 220/55 x 88	grey/white striped
1669	WendyLett, 100 cm/40" sliding surface ( <i>bed 120 cm/48"</i> )	200 x 220/79 x 88	grey/white striped
1641	WendyLett, 70 cm/28" sliding surface ( <i>bed 90 cm/36"</i> ), <i>tailored with elastic spring</i>	200 x 90/79 x 36	grey/white striped
1651	WendyLett, 80 cm/32" sliding surface ( <i>bed 105 cm/42"</i> ), <i>tailored with elastic spring</i>	200 x 105/79 x 42	grey/white striped
1661	WendyLett, 95 cm/37" sliding surface ( <i>bed 120 cm/48"</i> ), <i>tailored with elastic spring</i>	200 x 120/79 x 47	grey/white striped
1671	WendyLett, 110 cm/43" sliding surface ( <i>bed 140 cm/56"</i> ), <i>tailored with elastic spring</i>	200 x 140/79 x 55	grey/white striped

### WendyLett2Way - drawsheet

(To be combined with WendyLett sliding sheet)

1620	WendyLett2Way	90 x 180/35 x 70	white
1631	WendyLett2Way	100 x 200/39 x 79	white/grey striped
1636	WendyLett2Way	140 x 200/55 x 79	white/grey striped

### WendyLett4Way - drawsheet

(To be combined with WendyLett sliding sheet)

1637	WendyLett4Way	140 x 200/55 x 79	grey/white checked
1647	WendyLett4Way	200 x 200/79 x 79	grey/white checked
1648	WendyLett4Way, inco	75 x 200/30 x 79	grey (inco) checked
1649	WendyLett4Way, inco with handles	140 x 120/55 x 47	grey (inco)
1657	WendyLett4Way, with handles	140 x 190/55 x 75	grey/white checked
1652	WendyLett4Way	140 x 260/55 x 102	grey/white checked

### WendyLean - pillowcase with handles

1680	WendyLean	50 x 60/20 x 24	grey/white striped
1682	WendyLean	80x 80/32 x 32	grey/white striped

# SystemRoMedic®

## Simple solutions for great results

SystemRoMedic® is the name of Direct Healthcare Group's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic® is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

## Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic® is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic® offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

## Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website;  
[www.directhealthcaregroup.com](http://www.directhealthcaregroup.com)

## For questions about the product and its use

Please contact your local Direct Healthcare Group and SystemRoMedic® representative. A complete list of all our partners with their contact details can be found on our website; [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com).

Direct Healthcare Group Sverige AB  
Torshamnsgatan 35,  
SE-164 40 Kista, Sweden

Tel: +46 (0)8-557 62 200  
[info@directhealthcaregroup.com](mailto:info@directhealthcaregroup.com)  
[www.directhealthcaregroup.com](http://www.directhealthcaregroup.com)

 Medical Device Class I

Medical Device Class I. The product complies with the requirements of the Medical Devices Regulation 2017/745.