# WendyLett4Way, Inco

System**RoMedic**<sup>™</sup>

---

Instructions for use – English

Bruksanvisning – Svenska

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
Návod na použitie – Slovenský

WendyLett4Way, inco  
Art nr. 1648

WendyLett4Way, inco  
with handels  
Art nr. 1649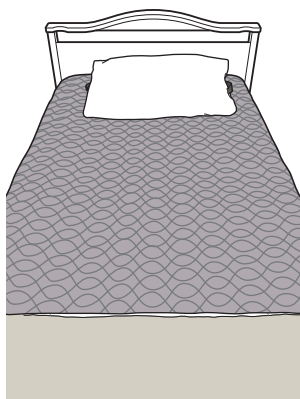

**REF**

1648

1649

SWL 1648: 300kg/660lbs

SWL 1649: 200kg/440lbs

## Table of contents

Product information .....	3
English .....	5
Svenska .....	7
Norsk .....	9
Dansk .....	11
Suomi .....	13
Deutsch .....	15
Nederlands .....	17
Français .....	19
Italiano .....	21
Español .....	23
Český .....	25
Slovensky .....	27
Symbols .....	29

**Product information/ Produktinformation/ Produktinformasjon/ Produktinformation/  
Tuotetiedot/ Produktinformation/ Productinformatie/ Information produit/  
Informazioni sul prodotto/ Información del Producto / Informace o produktu / Informácie  
o produkte**

Art. no.	Product/Produkt/ Tuote/ Produktname/ Product/ Produit/ Prodotto/ Producto	Size/ Storlek/ Størrelse/ Koko/ Größe/ Maat/ Taille/ Dimensione/ Tamaño cm/inch	Colour/Färg/Farge/ Farve/ Väri/ Farbe/ Kleur/ Couleur/ Colore/ Color
----------	--	--	---

**WendyLett - sliding sheet/ glidlakan/ glidelaken/ glidelagen/ liukuvaan telttakankaan/ piirtää arkki/ gleitlaken/ glijlaken/ feuille de glissement/ telone scorrevole/ sábana de deslizamiento / posuvný list / posuvný list**

(Can be combined with WendyLett2Way drawsheet or WendyLett4Way sliding drawsheet)

1609	WendyLett, 60 cm/24" sliding surface ( <i>bed 80/90 cm - 32"/36"</i> )	100 x 200/39 x 79	white
1614	WendyLett, 60 cm/24" sliding surface ( <i>bed 80/90 cm - 32"/36"</i> )	140 x 200/55 x 79	white
1629	WendyLett, 60 cm/24" sliding surface ( <i>bed 80/90 cm - 32"/36"</i> )	100 x 200/39 x 79	grey/white striped
1634	WendyLett, 60 cm/24" sliding surface ( <i>bed 80/90 cm - 32"/36"</i> )	140 x 200/55 x 79	grey/white striped
1639	WendyLett, 70 cm/28" sliding surface ( <i>bed 105 cm/42"</i> )	200 x 200/79 x 79	grey/white striped
1667	WendyLett, 100 cm/40" sliding surface ( <i>bed 120 cm/48"</i> )	100 x 220/39 x 88	grey/white striped
1668	WendyLett, 100 cm/40" sliding surface ( <i>bed 120 cm/48"</i> )	140 x 220/55 x 88	grey/white striped
1669	WendyLett, 100 cm/40" sliding surface ( <i>bed 120 cm/48"</i> )	200 x 220/79 x 88	grey/white striped
1641	WendyLett, 70 cm/28" sliding surface ( <i>bed 90 cm/36"</i> ), tailored with elastic string	200 x 90/79 x 36	grey/white striped
1651	WendyLett, 80 cm/32" sliding surface ( <i>bed 105 cm/42"</i> ), tailored with elastic string	200 x 105/79 x 42	grey/white striped
1661	WendyLett, 95 cm/37" sliding surface ( <i>bed 120 cm/48"</i> ), tailored with elastic string	200 x 120/79 x 47	grey/white striped
1671	WendyLett, 110 cm/43" sliding surface ( <i>bed 140 cm/56"</i> ), tailored with elastic string	200 x 140/79 x 55	grey/white striped
1681	WendyLett, 70 cm/28" sliding surface	200 x 365/79 x 144	grey/white striped

**WendyLett2Way - drawsheet/ draglakan/ stikklaenet/ træklagen/ piirtää arkki/ zuglaken/ treklaken/ dessiner feuille/ foglio di copertura/ sábana travesera / výkresový list / žrebovací list**

(To be combined with WendyLett sliding sheet)

1620	WendyLett2Way	90 x 180/35 x 70	white
1631	WendyLett2Way	100 x 200/39 x 79	white/grey striped
1636	WendyLett2Way	140 x 200/55 x 79	white/grey striped

**WendyLett4Way** - drawsheet/ draglakan/ stikklagenet/ træklagen/ piirtää arkki/ zuglaken/ treklaken/ dessiner feuille/ foglio di copertura/ sábana travesera / výkresový list / hárok

(To be combined with WendyLett sliding sheet)

1637	WendyLett4Way	140 x 200/55 x 79	grey/white checked
1647	WendyLett4Way	200 x 200/79 x 79	grey/white checked
1648	WendyLett4Way, inco	75 x 200/30 x 79	grey (inco) checked
1649	WendyLett4Way, inco with handles	140 x 120/55 x 47	grey (inco)
1657	WendyLett4Way, with handles	140 x 190/55 x 75	grey/white checked
1652	WendyLett4Way	140 x 260/55 x 102	grey/white checked

**WendyLean** - pillowcase with handles/ örngott med handtag/ putetreck med håntak/ hovedpudebetræk med håndtag/ tynnyliina kahvat/ Kissenbezug mit Zugschlaufen/ kussensloop met handgrepen/ Taie d'oreiller avec des poignées/ federa con maniglie/ unda de almohada con asas / povlak na polštář s uchy / obliečka na vankúš s rúčkami

1680	WendyLean	50 x 60/20 x 24	grey/white striped
------	-----------	-----------------	--------------------

Product combinations, Produktkombinationer, Produkt kombinasjon, Produkt kombination, Tuoteyhdistelmä, Produktkombination, Productcombinatie, Combinaison de produit, Combinazione di prodotti, Combinación del producto, Kombinace produkt / Kombinácie produktov

Art. no.	Combination
1610	1609+1620
1615	1614+1620
1630	1629+1631
1635	1634+1636
1640	1639+1636
1642	1641+1636
1638	1634+1637
1628	1634+1657
1658	1634+1649
1643	1639+1637
1644	1641+1637
1645	1641+1647
1646	1641+1657
1650	1669+1636
1656	1651+1657
1655	1651+1637
1658	1634+1649
1665	1669+1647
1699	1639+1649

WendyLett4Way with Inco is a sliding drawsheet that is used in combination with a WendyLett slide sheet. The sliding drawsheet, which is made from a highly absorbent material that can be laundered, is intended for users that are in need of incontinence protection. The sliding drawsheet also has an anti-friction material on the underside, which facilitates repositioning laterally as well as upwards and downwards on the bed.

### Check Safety

#### Visual inspection

Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.

#### Friction test

Low friction - Ensure that the surface is smooth and slides effortlessly. Compare with a new product.

### Always read the user manual


Always read the user manuals for all assistive devices used during a transfer.

Keep the user manual where it is accessible to users of the product. Always make sure that you have the right version of the user manual. The most recent editions of user manuals are available for downloading from our website, [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com).

### Placement


WendyLett4Way, inco  
Art nr. 1648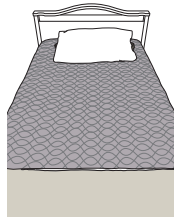

WendyLett4Way, inco  
with handles  
Art nr. 1649

The WendyLett slide sheet is placed with the shiny side in the middle of the bed and with the WendyLett4Way, Inco on top. WendyLett4Way, Inco is marked with a tag to show which side should be placed upwards on the bed.

### Materials

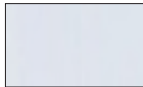The WendyLett4Way, Inco (1648) sliding drawsheet has a grey-checked pattern on the sides and an anti-friction, satin material on the underside. The pattern indicates the direction of glide. The topside of the sliding drawsheet is made of a soft, highly absorbent polyester material (Inco).

WendyLett4Way, Inco with handles (1649) has a topside that is fully covered with Inco material and an underside of anti-friction material. There are seven handles along the edges of the both sides. On this product there is no grey-checked pattern to indicate the direction of glide.





Be aware that there is a risk of the user sliding out of bed. Raise the bed rails before leaving the user.

Caution! Never attempt to lift a user with the handles on both sides of the sheet simultaneously.

## Turning

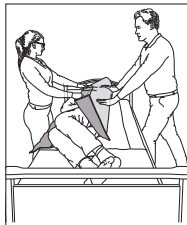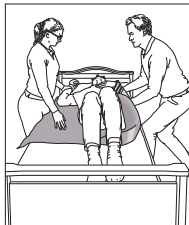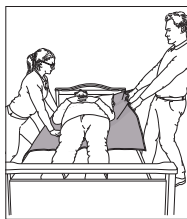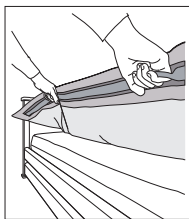
Turn the user by grasping the sliding drawsheet at the level of the hip and shoulder. When using WendyLett4Way, Inco a better grip at the centres of gravity can be achieved by bunching plenty of fabric together.

Turning using the sliding drawsheet can be done by 1 or 2 caregivers. When two caregivers are assisting, one can draw the sliding drawsheet (as above) and the other can press the sliding drawsheet down against the bed at the hip and shoulder while at the same time pushing (image 2), using the so-called “press down method”. Then, the user is turn on one side (images 3 and 4) and repositioned comfortably in the right position.

Decide, in each individual case, whether the user can be left without the bed side rails raised. Remember, there is a risk that the user may slide out of bed. When using the model 1649 the bed side rails should be raised, as the sheet cannot be tucked in under the mattress.

## Higher up in bed with

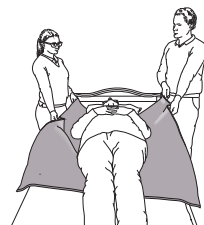
When repositioning the user higher up in bed the sliding drawsheet is grasped at the centres of gravity on either side of the user, either by grasping the fabric or the handles, depending on the model. Work in an upright stance and shift your weight while drawing the user higher up in bed.

## Sitting up in bed

The user can be prevented from sliding downward in bed by tucking in the lower 10 cm. By folding the sliding drawsheet double the friction side faces downward, against the underlying surface, thereby preventing the user from sliding downward in bed.

## Advice

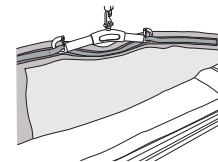If positioning is strenuous, the WendyLett4Way, Inco with handles (Art. No. 1649) can be used in combination with a user lift. Only connect the handles on one side of the sheet to the slingbar loops. See image. Positioning in combination with a lift reduces friction under half of the body and facilitates repositioning of the user in bed, e.g., lying on the side or higher up in bed.

Advice: Use AdjustmentLoops (Art. No. 49182006) to increase the distance between the slingbar and the sliding drawsheet.

## Material

Polyester, cotton, satin

## Care of the product

Read the product label.

Do not use rinsing agent. To ensure maximum material life, hang dry or tumble dry on low heat.

Contact your local distributor if you have any questions about the product and its use.

See [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com) for a complete list of distributors.

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# Symbols

	This product complies with the requirements of the Medical Device Regulation 2017/745		Visual Inspection
	Medical Device		Read the manual
	Caution		Product Code
	Manufacturer information		Batch Code

## Simple solutions for great results

SystemRoMedic® is the name of Direct Healthcare Group's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic® is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

## Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic® is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic® offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

## Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website;  
[www.directhealthcaregroup.com](http://www.directhealthcaregroup.com)

## For questions about the product and its use

Please contact your local Direct Healthcare Group and SystemRoMedic® representative. A complete list of all our partners with their contact details can be found on our website; [www.directhealthcaregroup.com](http://www.directhealthcaregroup.com).

Moving Health Forward

Direct Healthcare Group Sverige AB  
Torshamnsgatan 35,  
SE-164 40 Kista, Sweden

Tel: +46 (0)8-557 62 200  
[info@directhealthcaregroup.com](mailto:info@directhealthcaregroup.com)  
[www.directhealthcaregroup.com](http://www.directhealthcaregroup.com)

Medical Device Class I. The product complies with the requirements of the Medical Devices Regulation 2017/745.