Instructions for use – English

Bruksanvisning – Svenska

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Návod na použitie – Slovensky

**REF**

1601 1602 1603

1604 1605

1609 1614 1616

1629 1634

1639 1667

1668 1669

1641 1651

1660

1661 1671

1681 1683

1611 1612

1613 1621

1622 1691

1634CH

1641CH

1641CWS

1633

SWL: 300 kg/ 660 lbs

Table of contents

Product information	3
English	5
Svenska	7
Norsk	9
Dansk	11
Suomi	13
Deutsch	15
Nederlands	17
Français	19
Italiano	21
Español	23
Český	25
Slovensky	27
Symbols	29

**Product information/ Produktinformation/ Produktinformasjon/ Produktinformation/
Tuotetiedot/ Produktinformation/ Productinformatie/ Information produit/
Informazioni sul prodotto/ Información del Producto / Informace o výrobku / Informácie
o produkte**

Art. no.	Product/Produkt/ Tuote/ Produktname/ Product/ Produit/ Prodotto/ Producto / Výrobek	Size/ Storlek/ Størrelse/ Koko/ Größe/ Maat/ Taille/ /Dimensione/ Tamaño cm/ inch / Velikost	Colour/Färg/Farge/ Farve/ Väri/ Farbe/ Kleur/ Couleur/ Co- lore/ Color / Barva
----------	--	---	---

WendyLett - sliding sheet/ glidlakan/ glidelaken/ glidelagen/ liukuvan telttakankaan/ piirtää arkki/ gleitlaken/ glijlaken/ feuille de glissement/ telone scorrevole/ sábana de deslizamiento / prostěradlo pro posouvání / posuvný list

(Can be combined with WendyLett2Way drawsheet or WendyLett4Way sliding drawsheet)

1601	WendyLett, 60 x 140cm sliding surface (bed 80/90cm), elastic	140 x 250/55 x 98	grey/white striped
1602	WendyLett, 75 x 140cm sliding surface (bed 105cm), elastic	140 x 250/55 x 98	grey/white striped
1603	WendyLett, 90 x 140cm sliding surface (bed 120cm), elastic	140 x 250/55 x 98	grey/white striped
1604	WendyLett, 120 x 140cm sliding surface (bed 150cm), elastic	140 x 250/55 x 98	grey/white striped
1605	WendyLett, 150 x 140cm sliding surface (bed 180cm), elastic	140 x 250/55 x 98	grey/white striped
1609	WendyLett, 60 cm/24" sliding surface (bed 80/90 cm - 32"/36")	100 x 200/39 x 79	white
1611	WendyLett, 50 cm/20" sliding surface (bed 80/90 cm - 32"/36")	100 x 180/39 x 71	grey/white striped
1612	WendyLett, 50 cm/20" sliding surface (bed 80/90 cm - 32"/36")	140 x 180/55 x 71	grey/white striped
1613	WendyLett, 60 cm/24" sliding surface (bed 80/90 cm - 32"/36")	200 x 80/79 x 31	grey/white striped
1614	WendyLett, 60 cm/24" sliding surface (bed 80/90 cm - 32"/36")	140 x 200/55 x 79	white
1621	WendyLett, 75 cm/29" sliding surface (bed 80/90 cm - 32"/36")	140 x 205/55 x 81	grey/white striped
1622	WendyLett, 75 cm/29" sliding surface (bed 80/90 cm - 32"/36")	200 x 205/79 x 81	grey/white striped
1629	WendyLett, 60 cm/24" sliding surface (bed 80/90 cm - 32"/36")	100 x 200/39 x 79	grey/white striped
1634	WendyLett, 60 cm/24" sliding surface (bed 80/90 cm - 32"/36")	140 x 200/55 x 79	grey/white striped
1639	WendyLett, 70 cm/28" sliding surface (bed 105 cm/42")	200 x 200/79 x 79	grey/white striped
1667	WendyLett, 100 cm/40" sliding surface (bed 120 cm/48")	100 x 220/39 x 88	grey/white striped
1668	WendyLett, 100 cm/40" sliding surface (bed 120 cm/48")	140 x 220/55 x 88	grey/white striped
1669	WendyLett, 100 cm/40" sliding surface (bed 120 cm/48")	200 x 220/79 x 88	grey/white striped
1641	WendyLett, 70 cm/28" sliding surface (bed 90 cm/36"), tailored with elastic string	200 x 90/79 x 36	grey/white striped
1651	WendyLett, 80 cm/32" sliding surface (bed 105 cm/42"), tailored with elastic string	200 x 105/79 x 42	grey/white striped
1660	WendyLett, 70 cm/28" T-shape sliding surface (bed 90 cm/36"), tailored with elastic string	200 x 90/79 x 36	grey/white striped
1661	WendyLett, 95 cm/37" sliding surface (bed 120 cm/48"), tailored with elastic string	200 x 120/79 x 47	grey/white striped
1671	WendyLett, 110 cm/43" sliding surface (bed 140 cm/56"), tailored with elastic string	200 x 140/79 x 55	grey/white striped
1681	Wendylett, 70 cm/28" sliding surface	200 x 365/79 x 144	grey/white striped
1691	Wendylett, 70 cm/28" sliding surface	215 x 90/85 x 35	grey/white striped

WendyLett2Way - drawsheet/ draglakan/ stikkklakenet/ træklagen/ piirtää arkki/ zuglaken/ treklaken/ dessiner feuille/ foglio di copertura/ sábana travesera / napinací prostěradlo / žrebovací list

(To be combined with WendyLett sliding sheet)

1620	WendyLett2Way	90 x 180/35 x 70	white
1631	WendyLett2Way	100 x 200/39 x 79	white/grey striped
1636	WendyLett2Way	140 x 200/55 x 79	white/grey striped

WendyLett4Way - drawsheet/ draglakan/ stikkklakenet/ træklagen/ piirtää arkki/ zuglaken/ treklaken/ dessiner feuille/ foglio di copertura/ sábana travesera / napinací prostěradlo / žrebovací list

(To be combined with WendyLett sliding sheet)

1637	WendyLett4Way	140 x 200/55 x 79	grey/white checked
1647	WendyLett4Way	200 x 200/79 x 79	grey/white checked
1648	WendyLett4Way, inco	75 x 200/30 x 79	grey (inco) checked
1649	WendyLett4Way, inco with handles	140 x 120/55 x 47	grey (inco)
1657	WendyLett4Way, with handles	140 x 190/55 x 75	grey/white checked
1652	WendyLett4Way	140 x 260/55 x 102	grey/white checked
1687	WendyLett4Way, with handles	200 x 190/79 x 75	grey/white checked
1692	WendyLett4Way	215 x 200/85 x 79	grey/white checked

WendyLean - pillowcase with handles/ örngott med handtag/ putetrekk med håndtak/ hovedpudebetræk med håndtag/ tynnyliina kahvat/ Kissenbezug mit Zugschlaufen/ kussensloop met handgrepen/ Taie d'oreiller avec des poignées/ federa con maniglie/ unda de almohada con asas / povlak na polštář s úchyty / obliečka na vankúš s rúčkami

1680	WendyLean	50 x 60/20 x 24	grey/white striped
------	-----------	-----------------	--------------------

Product combinations, Produktkombinationer, Produkt kombinasjon, Produkt kombination, Tuoteyhdistelmä, Produktkombination, Productcombinatie, Combinaison de produit, Combinazione di prodotti, Combinación del producto, Kombinace výrobků, Kombinácie produktov,

Art. no. Combination

1610	1609+1620
1615	1614+1620
1630	1629+1631
1635	1634+1636
1640	1639+1636
1642	1641+1636
1638	1634+1637
1628	1634+1657
1658	1634+1649
1643	1639+1637
1644	1641+1637
1645	1641+1647
1646	1641+1657
1650	1669+1636
1656	1651+1657
1655	1651+1637
1658	1634+1649
1665	1669+1647
1699	1639+1649

WendyLett is a slide sheet that facilitates positioning and turning in bed. It is bedded in the bed like a normal sheet.

WendyLett with elastic spring is bedded in the bed like a normal sheet and is fastened around the mattress with the elastic spring.

WendyLett can be used in combination with drawsheet, WendyLett2Way, or sliding drawsheet, WendyLett4Way.

WendyLett is available in several sizes and models.

Check Safety

Visual inspection

Check the condition and function of the assistive device regularly.

Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.

Friction test

Low friction - Ensure that the surface is smooth and slides effortlessly. Compare with a new product.


Always read the user manual


Always read the user manuals for all assistive devices used during a transfer. Keep the user manual where it is accessible to users of the product. Always make sure that you have the right version of the user manual.

The most recent editions of user manuals are available for downloading from our website, www.directhealthcaregroup.com.


Placement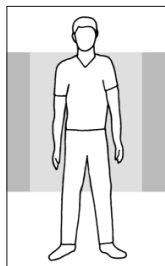


100x200
100x220

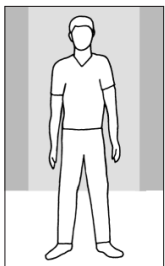140x200
140x220

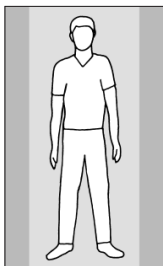200x200
200x220
200x365

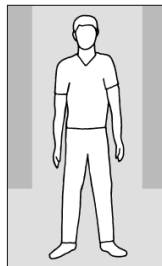
200x90, 200x105,
200x120, 200x140

Place WendyLett with the smooth surface in the middle of the bed. It is bedded in the bed like a normal sheet.

Ensure that the mattress cover's rough surfaces or evenly distributed on both sides of the bed. These rough surfaces prevent the user from sliding out of bed.

Material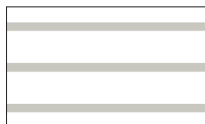


The WendyLett slide sheet has a pattern of white stripes on a grey background. Stripes indicate direction of glide.

The draw sheet, WendyLett2Way, has a white fabric with grey stripes. Stripes indicate the direction in which the sheet is drawn.

WendyLett4Way has a grey-checked pattern and has a sliding surface underneath. Stripes indicate the direction in which the sheet is drawn.


Using WendyLett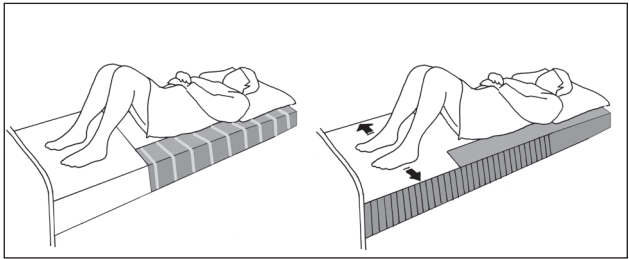

WendyLett can be bedded in cases where the user is able to turn himself/herself, but finds it easier with less friction.

The smooth surface of the sheet reaches over the foot of the bed, which makes it easier for the user to shift his/her feet when sitting up at the edge of the bed.

Accessories

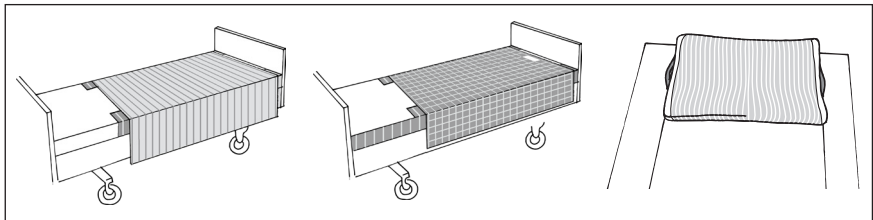With WendyLett2Way, the user can turn himself/herself without sliding down in the bed when the head end is raised. If the user needs some assistance or turning is heavy, WendyLett2Way can function as a draw sheet for the caregiver.

With WendyLett4Way, the user can be helped to a position higher up in bed or closer to one side.

WendyLean is a pillow case with an anti-friction surface on the underside and handles on both sides. The anti-friction surface facilitates positioning of the pillow.

Material

Polyester, cotton, satin

Care of the product

Read the product label.

Do not use rinsing agent. To ensure maximum material life, hang dry or tumble dry on low heat.

Contact your local distributor if you have any questions about the product and its use.

See www.directhealthcaregroup.com for a complete list of distributors.

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Úplný zoznam distribútorov nájdete na www.directhealthcaregroup.com.

Symbols

	This product complies with the requirements of the Medical Device Regulation 2017/745		Visual Inspection
	Medical Device		Read the manual
	Caution		Product Code
	Manufacturer information		Batch Code

Simple solutions for great results

SystemRoMedic® is the name of Direct Healthcare Group's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic® is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic® is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic® offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website;
www.directhealthcaregroup.com

For questions about the product and its use

Please contact your local Direct Healthcare Group and SystemRoMedic® representative. A complete list of all our partners with their contact details can be found on our website; www.directhealthcaregroup.com.

Direct Healthcare Group Sverige AB
Torshamnsgatan 35,
SE-164 40 Kista, Sweden

Tel: +46 (0)8-557 62 200
info@directhealthcaregroup.com
www.directhealthcaregroup.com

Medical Device Class I. The product complies with the requirements of the Medical Devices Regulation 2017/745.