Moving Health Forward

EN [User Manual](#)

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LEJRELET

Oval

EN

Application

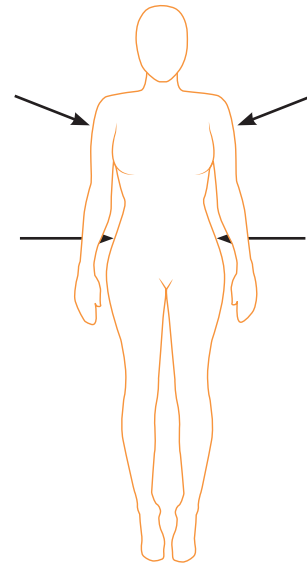
With the LEJRELET Oval it is possible to introduce the concept of "The Walking Cushion".

"The Walking Cushion" is a concept that works with frequent and discreet repositioning which ensures a calm night for the bedridden. This in turn supports an uninterrupted sleep for the bedridden, which is very important for the metabolic cycle.

LEJRELET Oval is made with a very flexible material and offers excellent pliability. Consequently, it is very easy to use and the user experiences good support and comfort.

Other examples for use of LEJRELET Oval.:

- Between the knees or feet.
- As lumbar support.
- Support of shoulder or elbow.
- Support of paralysed arm or hand.
- Between arm and thorax

"The Walking Cushion" – place the cushion as indicated and keep repositioning in this cycle.

Technical Information

Filling:

LEJRELET Oval is filled with viscoelastic foam granules to help prevent pressure ulcers.

Fabric:

- Waterproof
- Watervapour permeable
- Two way stretch
- Anti-bacteria treated
- Tight to house dust mite
- Biocompatible
- Oeko-TEX, class 1
- Fire-retardant to crib 5

Washing & Cleaning

Can be wiped with mild soap and disinfectant fluids or showered with boiling water.

Hand wash

Do not dry clean

Do not tumble dry

Do not bleach

Do not iron

Size: H 10 x W 25 x L 40

Item no.: 5000009

Weight: 350 g

Be aware that the patient is positioned such that airways are kept clear.

Other Symbols

Warning

Item no.

Year and month of manufacture

Name of manufacturer

CE marking:

The product complies with the Medical Device Regulation (EU) 2017/745

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Direct Healthcare Group Ltd
Direct Healthcare Group Sverige AB,
Direct Healthcare Group Pty Ltd.,