Moving Health Forward

**EN** [User Manual](#)

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# LEJRELET

## Tube 125

# EN

## Application


The LEJRELET Tube 125 has the same structure and positive qualities as the LEJRELET Tube 250. The great flexibility of the material affords many application options.

It is, however, only half the length, i.e. 125 cm long and therefore weighs less.


The LEJRELET Tube 125 is suitable for smaller adults and large children. It is also suitable for leg amputees who need good support for the shortened leg. The LEJRELET Tube 125 can be folded very tightly, thus providing great support for, inter alia, contractures.

Examples for use of LEJRELET Tube 125:


- To support the body in a lateral position
- For 30-degree positioning for leg amputees
- To support legs and arms while seated
- Stable lateral positioning for flexion contractures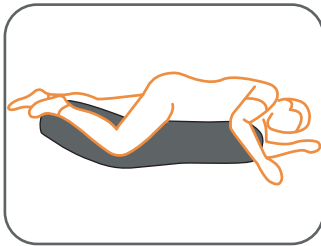


*Support of the entire body in lateral position*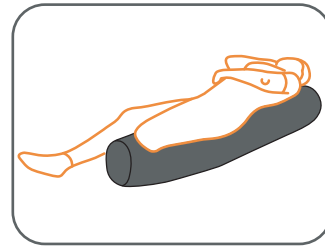

*For 30-degree positioning for leg amputees*

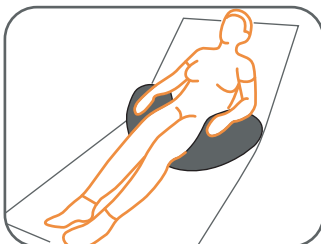*To support legs and arms while seated*

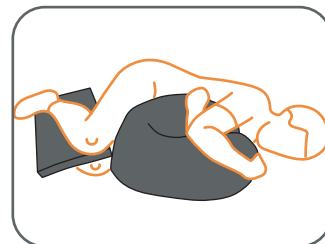*Stable lateral positioning for flexion contractures*

## Technical Information

### Filling:

LEJRELET Tube 125 is filled with viscoelastic foam granules preventing decubitus.

### Fabric:

- Waterproof
- Watervapour permeable
- Two way stretch
- Anti-bacteria treated
- Tight to house dust mite
- Biocompatible
- Oeko-TEX, class 1
- Fire-retardant to crib 5

## Washing & Cleaning

Can be wiped with mild soap and disinfectant fluids or showered with boiling water.

Hand wash

Do not dry clean

Do not tumble dry

Do not bleach

Do not iron

**Size:** Ø 25 x L 125

**Item no.:** 5000000

**Weight:** 2350 g

Be aware that the patient is positioned such that airways are kept clear.

## Other Symbols

Warning

Item no.

Year and month of manufacture

Name of manufacturer

CE marking:

The product complies with the Medical Device Regulation (EU) 2017/745

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**This Page is Left Intentionally Blank**

**This Page is Left Intentionally Blank**

Moving Health Forward

**PRESSURE ULCER PREVENTION**  
**SAFE MOVING & HANDLING**  
**SPECIALIST THERAPIES**  
**BATHROOM SAFETY**  
**RENTAL & SERVICE SOLUTIONS**

Direct Healthcare Group Ltd  
Withey Court, Western Industrial Estate  
Caerphilly, United Kingdom  
CF83 1BF

T: +44 (0) 800 043 0881  
F: +44 (0) 845 459 9832  
E: [info@directhealthcaregroup.com](mailto:info@directhealthcaregroup.com)

**EC** **REP**

**EU Authorised Representative, Importer, and Distributor:**

Direct Healthcare Group Sverige AB,  
Torshamnsgatan 35, SE-164 40, Kista  
Sweden  
T: +46 (0) 8 557 200

**Australian Sponsor:**

Direct Healthcare Group Pty Ltd.,  
68 Howe Street, Osborne Park,  
Western Australia 6017  
T: +61 (0) 423 852 810

Issue 3  
Date: June 2022

[DHG-HEALTHCARE.COM](http://DHG-HEALTHCARE.COM)