


QUICK REFERENCE GUIDE



Dyna-Form® Low Air Loss

This is a quick reference guide for the **Dyna-Form Low Air Loss System** - (refer to the User Manual for additional information)

Product Code MAT/LOW/AIR

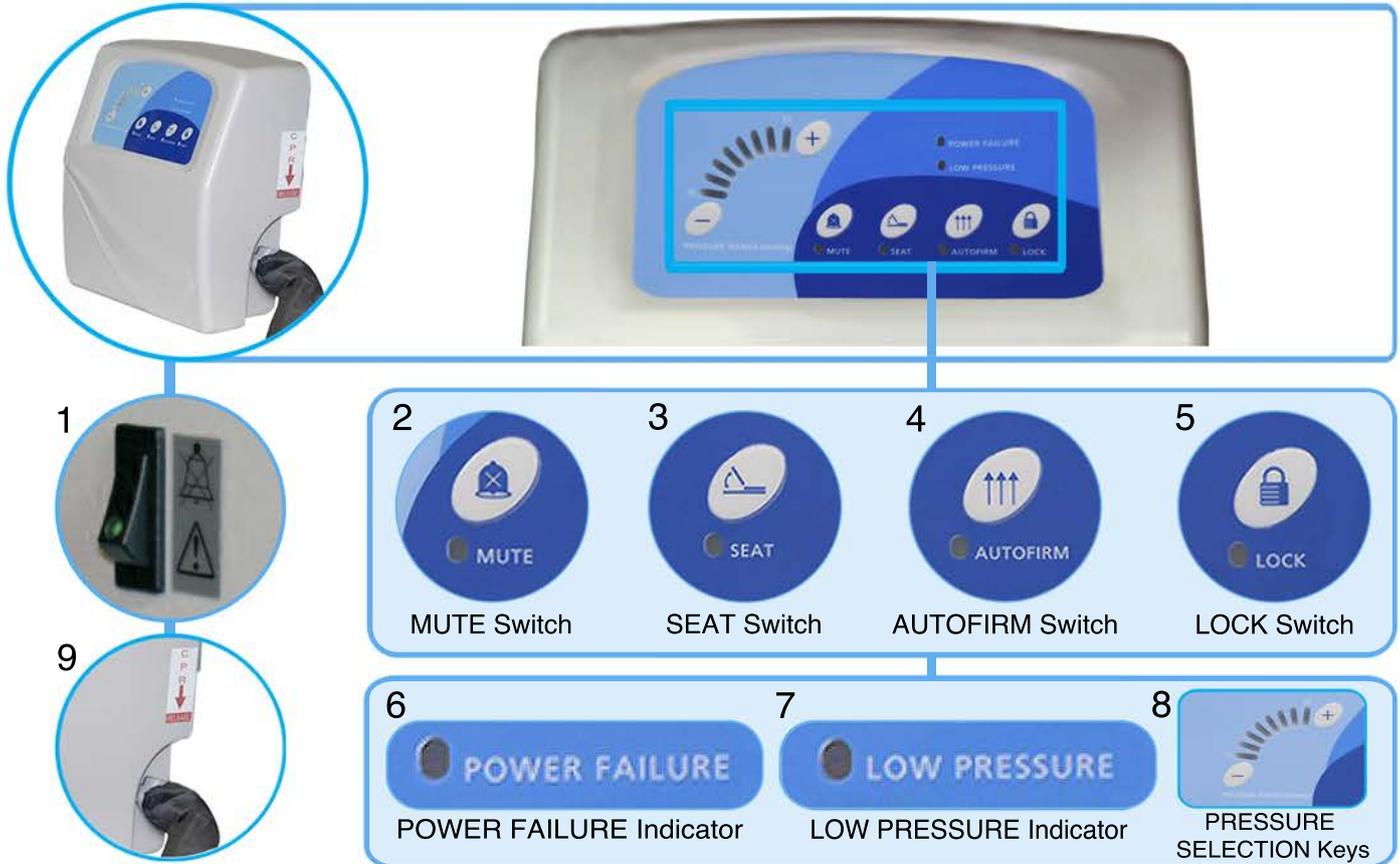

1) The POWER switch is at the lower right side of the pump. Turning the power switch ON/OFF will start/stop the pump. The switch lights up when the power is ON, and extinguishes when the power is OFF.

2) The audible/visible alarm turns on either when the pressure is low or the system fails. Press the Mute switch to mute the audible alarm, and the MUTE indicator (yellow) will light up. The alarm indicator will continue flickering. Re-press the Mute switch to re-activate the audible alarm and to extinguish the MUTE indicator.

3) Press the SEAT switch to increase the pressure in each cell by 5mmHg. The SEAT indicator (yellow) lights up when the system is in SEAT mode. To cancel SEAT mode, press the SEAT switch.

4) Press the AUTOFIRM switch to set AUTOFIRM mode to quickly inflate the air mattress to the maximum pressure which facilitates nursing care and bed mobility. The AUTOFIRM indicator (yellow) lights up when the system is in AUTOFIRM mode. The system will automatically return to the previous mode 30 minutes after the AUTOFIRM mode is in operation. It is also possible to cancel AUTOFIRM mode by pressing the Pressure Selection keys or Seat switch.

5) Press and hold the LOCK switch for 2 seconds to lock the panel. Press and hold the LOCK switch for 2 seconds again to unlock the panel. The LOCK indicator (green) lights up when the panel is locked. The panel will automatically enter locked mode if no control panel operations are entered for over 30 seconds.

6) This indicator (red) flickers when there is no power input to pump.

7) This indicator (red) flickers when the pressure is below the pre-defined level.

8) The PRESSURE SELECTION Keys can be used to determine internal cell pressure in the mattress. Eight pressure levels are indicated by the number of indicators (green) that light up as the pressure is increased to support your patient.

9) Pull the CPR Strap to release the air immediately from the mattress. The CPR Strap is located at the front right hand side of the mattress.