

SWL: 300kg/660lbs

ClassicSling enables a comfortable and secure seated posture and gives the patient a feeling of security. It is designed to be left in the chair after the transfer is completed, which means that the patient sits on the sling. ClassicSling is available in two models, with or without commode opening. It provides support all the way from the hollow of the knees up to the shoulders. ClassicSling is designed for use in most common lifting situations such as transfers from bed to wheelchair, etc.

SystemRoMedic includes a wide range of functional, comfortable, high-quality slings that can be adapted for different types of lifting and for patients with different needs. The slings are available in several different materials and in sizes from XXS to XXL. All models are safe and very easy to use and are rated for lifting patients weighing up to 300 kg. The choice of model and material depends on the patient's needs and the transfer situation.



Functional inspection

Visual and mechanical inspection

Check the condition and function of the sling regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. Apply load to the device and check to ensure that clasps, handles, etc. withstand heavy load. If there are signs of wear, the product must be discarded.



Always read the manual

Always read the manuals for all assistive devices used during a transfer.

Keep the manual where it is accessible to users of the product.



Do not leave the patient unattended during a transfer situation.

To prevent discomfort and the risk of the patient sliding out of the sling, trial fit the sling carefully. First, apply the sling's back loops to the slingbar, and then apply the leg support loops. Use a slingbar that has been tested.

Note that there may be a risk of the patient tipping forward when a sling with standard (not separate) leg supports is used.

The lowest allowable safe working load always determines the safe working load of the assembled system. Always check the safe working loads for the lift and accessories before use. Contact your dealer if you have any questions.

ClassicSling can only be applied when the patient is fully recumbent

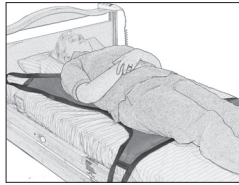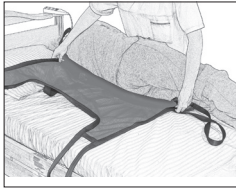

1. Turn the patient towards you. Place the sling so that the mid-section follows the spine and the lower edge is about 5 to 10 cm above the hollow of the patient's knees. Place the edge of the sling as far in under the patient as possible.
2. Turn the patient back over, ensuring that he or she is centred over the sling.
3. Preferably, raise the back support before lifting. Start raising the lift, when the straps become taut, stop and check that everything is ok before proceeding with the lift.
4. The sling is left in the chair.

ClassicSling can only be removed when the patient is lying on the bed/stretcher or floor

Follow the same steps as with application, but in the reverse order.

Material:

Polyester net: An airy material that can be left under the patient, if necessary. Withstands high laundry temperatures.

Care of the product:

Read the product label.



Do not use rinsing agent.
To prolong product life, avoid tumble-drying.

Size information

Slings can often be used for both 2 and 4-point suspension. It is essential to ensure that the width of the slingbar is appropriate for the sling size and the patient's status and capacity.

Use patient-specific slings to ensure safety and hygiene.

Size		XS/Orange	S/Röd	M/Yellow	L/Green	XL/Purple
Rec. user weight	lbs	44-88	66-132	121-187	176-265	243-309
	kg	20-40	30-60	55-85	80-120	110-140
Sling overall height	inch	31,5	35,5	37,5	38,5	39,5
	cm	80	90	95	98	100
Sling seat measurement	inch	12	15	18	25	31,5
	cm	30	38	46	63	80

Contact your local distributor if you have any questions about the product and its use. See www.handicare.com for a complete list of distributors. Always make sure that you have the right version of the manual. The most recent editions of manuals are available for downloading from our website, www.handicare.com.

Handicare AB

Veddestav. 15. Box 640
SE-175 27 Järfälla
SWEDEN

Tel: +46 (0)8 -557 62 200
Fax: +46 (0)8-557 62 299

Internet: www.handicare.com
E-mail: info@handicare.se

