

Coral

Mobile commode/shower chair

Height adjustable

Stable support

Compact and light-weight

Ergonomic design

3002020050
Coral, commode/shower chair, mobile, complete with
toilet rails and swing-away footrests



A mobile and height-adjustable commode/shower chair for flexible comfort and convenience

The mobile Coral commode/shower chair is an assistive device allowing the user to sit down safely and conveniently during toileting or showering. It is supplied complete with toilet rails and can be used freestanding with a bedpan or bucket or standing over a toilet. The commode/shower chair provides a stable and height-adjustable support for less active users with weak legs, low stamina and/or poor balance who sit on the chair for longer periods of time and who require more support and assistance during sit-to-stand and seated transfers. Four twin castors make the commode/shower chair mobile and very easy to move between different locations as well as to manoeuvre, turn and adjust to different positions in various settings, also when the user is sitting in the chair. The seat height is easily adjusted to ensure the perfect fit for every user and situation.

Coral

Molded toilet seat provides a good sitting position and a comfortable seating angle

The rust-free frame is fitted with a practical and comfortable molded toilet seat made from durable plastic with a smooth surface. The seat provides a good sitting position and the comfortable seating angle of one degree creates a safe situation and a pleasant experience during toileting and showering as the user will not slide off the seat. The lower parts in the back and front of the seat make it easier for the user to reach the intimate body parts without standing up.

Backrest with folding armrests provides extra support and security

For users with reduced strength and balance, sit-to-stand and stand-to-sit procedures can be difficult. This model of Coral is fitted with a stable, comfortable backrest with folding armrests that provide the extra support needed during these movements. The back- and armrests also offer the user an extra sense of security while seated without hindering a more active and independent user.

The backrest frame is fitted with a comfortable and easy-to-care-for mesh textile cover offering soft and pliable support. It also dries very quickly. The depth of the backrest is adjustable with Velcro at the lower end. The armrests can be individually folded up and down, thereby both facilitating sideways transfers and providing more space for the user while also permitting easier access for the caregiver. The length of the armrests allows for maximum support during forward or backward transfers.

Twin castors with double brakes and kick-brake pedals

The mobile Coral commode/shower chair is equipped with four twin castors with double brakes and very convenient kick-brake pedals. The encapsulated design makes them very easy to keep clean. The chair is also fitted with swing-away footrests providing even more comfort and security for the user.

Light-weight and compact, easy to fit

The mobile chair is light-weight and compact and fits easily into any wet room.

For more information, please visit our website, www.handicare.com, or contact your local Handicare partner.

FEATURES AND BENEFITS

- Mobile, easy for the caregiver to move and maneuver, easy for the user to toddle
- Stable and secure, ergonomic design
- Safe and very easy to use
- Compact, hygienic and attractive design, fits into any wet room
- Very easy to assemble – no tools required
- Stainless steel and plastic, suitable for wet environments
- Easy maintenance
- Molded toilet seat provides a good sitting position and a comfortable seating angle
- Lower parts in the back and front facilitate intimate hygiene
- The design of the lower parts prevents the user from slipping through the seat
- Easily adjustable seat height; 420-570 mm, adjustable in 7 steps of 25 mm
- Comfortable and ergonomically shaped backrest provides soft and pliable support
- The backrest features an integrated, loop-shaped push bar with ergonomic height and design
- The textile cover for the backrest is easy to remove and washable at 60-85 degrees for optimal hygiene
- Folding armrests provide ergonomic support during transfers and security while sitting
- Back- and armrests are designed not to hinder a more active and independent user
- Castors with brakes and very convenient and easy-to-use kick-brake pedals
- Swing-away footrests provide even more comfort and security

PRODUCT SPECIFICATIONS

Art.no	Description
3002020050	Coral, mobile commode/shower chair, complete with toilet rails and swing-away footrests

TECHNICAL INFORMATION

Material, frame and legs:	Stainless steel
Material, seat and armrests:	GF 30 % PP plastic
Material, backrest, frame/cover:	GF 30 % PP plastic / textile, polyester mesh
Material, footrests:	GF 30 % PP plastic
Castors:	Twin castors, Ø 100 mm/3.9"
Seat height:	420 - 570 mm / 16.5" - 22.4", adjustable
Seat measurements, width x depth:	490 mm x 460 mm / 19.3" x 18.1"
Toilet opening, width x length:	210 mm x 280 mm / 8.3" x 11"
Backrest, height x, width:	520 mm x 490 mm / 20.5" x 19.3"
Height, armrests (measured from seat):	230 mm / 9"
Length, armrest:	400 mm / 15.7"
Total height:	940 - 1090 mm / 37" - 42.9", adjustable
Outer measurements, width x depth:	620 mm x 780 mm / 24.4" x 30.7"
Free height under seat (for placing over a toilet):	365 - 515 mm / 14.4" - 20.3"
Free width under seat (for placing over a toilet):	410 mm / 16.1"
Weight:	14 kg / 31 lbs
Max. load on armrest:	75 kg / 165.3 lbs
Max. user weight:	150 kg / 330 lbs

ACCESSORIES

Item No.	Product
LI02979.411	Bedpan, with lid and a handle, Ø 300 mm, polished stainless steel
LI02979.430	Bucket, with lid and a handle, Ø 300 mm, polished stainless steel
3002920400	Soft toilet seat, PUR, oval opening
3002920401	Soft toilet seat, PUR, keyhole opening
3002920500	Transportation seat, PUR
3002920200	Calf support, textile, with Velcro
3002921300	Safety belt, for hip or chest
3002920900	Front support, aluminium/PUR
3002921100	Armrest lock, prevents folding function



handicare

Maskinvägen 17,
972 54 Luleå, Sweden
Tel: +46 (0)8-557 62 200
Fax: +46 (0)8-557 62 299
www.handicare.com