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EN - Please read the instructions before using the product.

The instructions apply to lateral trunk supports for the right or left side.

Intended use

The lateral trunk support can be installed on all Combi chairs.

It is well suited to people who have a reduced sense of balance and can fall over to the side. The body support works by providing support to the upper body of the user. It is available in two models: a standard adjustable width, and a broad adjustable width. Both have a height adjustment.

Installation

The body support can be supplied for the right or left side.

First remove the PUR backrest by loosening the two screws (1).

The body support is installed on the back of the Combi chair at the desired height, using the two screws supplied (2).

See the drawing.

Reinstall the PUR backrest with the two screws (1).

Adjustment and use

When the trunk supports are installed on the Combi chair, they can be adjusted in width by loosening the thumbscrews (3) in the "arm", and adjusting them to the desired position.

The trunk supports can be moved away from the user by pulling up on the positioning bolts(4), and then swinging the supports out to the side. When the user is back in the chair, the supports can be swung in again so that they support the user's upper body.

The trunk support can be adjusted in height by loosening the screws (5), and positioning the "arm" with the PUR support at the desired height. Then the screws (5) are tightened again.

The broad adjustable width body support is adjusted in the same manner as the narrow support.

Maintenance

The trunk support can be cleaned with ordinary cleaning products that do not have polishing or etching properties. It is not recommended to use solvents or chlorine. Polyurethane can tolerate temperatures up to 85 °C / 3 minutes.

Warning!

- Must only be used as described under the section "Intended use".
- When the user is moved out of the chair, the trunk supports should be swung out to the side.
- Be certain that the positioning bolts go into the notches, when the supports are swung in to support the user.
- Check on a regular basis that the screws are tight.
- Be careful not to have hands or fingers on the pipe support of the back rest where the trunk support is installed, so as to prevent the risk of being pinched.

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Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website; www.handicare.com.

For questions about the product and its use

Please contact your local Handicare and LinidoSolutions™ representative. A complete list of all our partners with their contact details can be found on our website; www.handicare.com.

Handicare offers solutions and support to increase the independence of disabled or elderly people as well as to improve the convenience of those who are caring for them.

The Handicare Group is one of the leading healthcare companies in Europe with its own manufacturing organizations and sales companies in Norway, Sweden, Denmark, Germany, the Netherlands, Great Britain, France, China, Canada and the USA. Handicare's products are also distributed by partners in more than 40 countries worldwide. Our wide range of high-quality products includes a complete easy transfer system and other patient handling aids, stairlifts, car adaptations and bathing and toileting products.

Handicare AB
Torshamnsgatan 35
SE-164 40 Kista , SWEDEN
Tel: +46 (0)8-557 62 200
Fax: +46 (0)8-557 62 299
E-mail: info@handicare.se
www.handicare.com