FlexiMove

SystemRoMedic™

User manual – English

Bruksanvisning – Svenska

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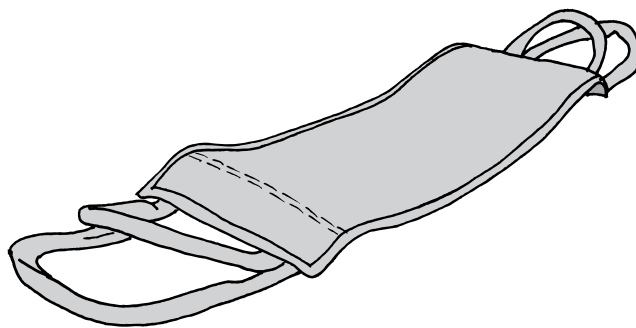
REF

3012

3020

3030

SWL: 150 kg/330 lbs

Table of contents

English	3-4
Svenska	5-6
Norsk	7-8
Dansk	9-10
Suomi	11-12
Deutsch	13-14
Nederlands	15-16
Français	17-18
Italiano	19-20
Español	21-22
Product information	23

FlexiMove is used to assist transfer and repositioning in various situations. Common situations in which FlexiMove can be used include transfers, raising and repositioning higher up in bed. FlexiMove features two different handle lengths to ensure a good working stance for the caregiver. Two materials on the inside: nylon or polyester. Nylon is an anti-slip material with a surface that is easy to disinfect. Polyester is soft and comfortable.

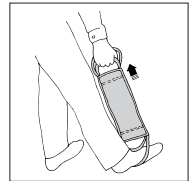
Check Safety

Visual inspection

Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded

Mechanical load

Apply pressure to buckles and handles and verify that they will not open or break under heavy loads.

Always read the user manual

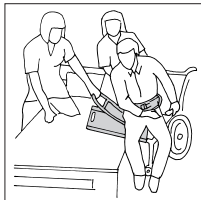
Always read the user manuals for all assistive devices used during a transfer.

Keep the user manual where it is accessible to users of the product.

Always make sure that you have the right version of the user manual.

The most recent editions of user manuals are available for downloading from our website, www.handicare.com.

Using FlexiMove


Transfer:

For transfers between e.g., wheelchair and bed, an EasyGlide should also be used, in some cases, even an EasyBelt/FlexiBelt.

Raising:

For raising, FlexiMove can be used to move the user's legs over the side of the bed and then as support behind the back/seat.

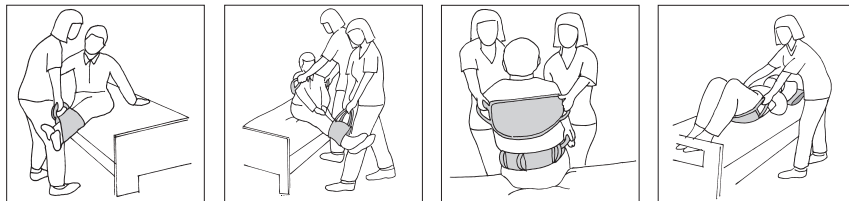
If the user requires the assistance of two caregivers, one should help with the legs and the other with the upper body. During the actual raising operation, the caregivers stand on either side of the user and choose the handles that enable the best working stance.

Higher up in bed:

To reposition the user higher up in bed, FlexiMove is placed under the seat.

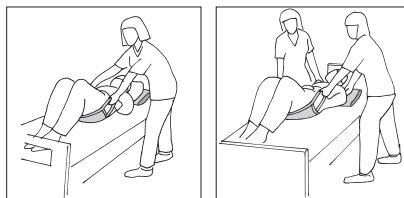It may also be advisable to use a friction-reducing device under the shoulders, for example, EasySlide.

Material

Polyamide, nylon/polyester

Care of the product

Read the product label.

Do not use rinse/ fabric softener and avoid tumble-drying for optimal service life.

CE Medical Device Class I

Medical Device Class I. The product complies with the requirements of the Medical Devices Directive 93/42/EEC.

Contact your local distributor if you have any questions about the product and its use. See www.handicare.com for a complete list of distributors.

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Simple solutions for great results

SystemRoMedic™ is the name of Handicare's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic™ is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic™ is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic™ offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website; www.handicare.com.

For questions about the product and its use

Please contact your local Handicare and SystemRoMedic™ representative. A complete list of all our partners with their contact details can be found on our website; www.handicare.com.

Handicare offers solutions and support to increase the independence of disabled or elderly people as well as to improve the convenience of those who are caring for them.

The Handicare Group is one of the leading healthcare companies in Europe with its own manufacturing organizations and sales companies in Norway, Sweden, Denmark, Germany, the Netherlands, Great Britain, France, China, Canada and the USA. Handicare's products are also distributed by partners in more than 40 countries worldwide. Our wide range of high-quality products includes various manual and power wheelchairs, seating systems, scooters, a complete easy transfer system and other patient handling aids, stairlifts, car adaptations, rise and recline chairs, and bathing and toileting products.

handicare

Handicare AB
Maskinvägen 17
SE-972 54 Luleå, SWEDEN
Tel: +46 (0)8-557 62 200
Fax: +46 (0)8-557 62 299
E-mail: info@handicare.se
www.handicare.com