**REF**

4003

4006

4007

**SWL: 150 kg/330 lbs**

## Table of contents

<b>English</b> .....	4-6
<b>Svenska</b> .....	7-9
<b>Norsk</b> .....	10-12
<b>Danske</b> .....	13-15
<b>Suomi</b> .....	16-18
<b>Deutsch</b> .....	19-21
<b>Nederlands</b> .....	22-24
<b>Français</b> .....	25-27
<b>Italiano</b> .....	28-30
<b>Español</b> .....	31-32
<b>Size</b> .....	33

LiftSeat is a sling that is adapted for manual lifting. For the caregiver, it is easy to apply, and it gives the user a feeling of security during manual lifting in a seated position. It features leg sections, which means that the user does not sit on the sling. Instead, the sling provides support from the tailbone up over the head and under the legs. LiftSeat is designed for use in most common situations where manual lifting in a seated position is required, such as transfers from wheelchair to aircraft seat, etc.


## Check Safety

### Visual inspection

Check the condition and function of the assistive device regularly. Always inspect the product after laundering. Check to ensure that seams and material are free from damage. Check to ensure that fabric is not worn or faded. If there are signs of wear, the product must be discarded.

### Mechanical load

Apply pressure to buckles and handles and verify that they will not open or break under heavy loads.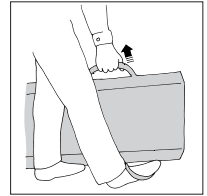

## Always read the user manual


Always read the user manuals for all assistive devices used during a transfer.

Keep the user manual where it is accessible to users of the product.

Always make sure that you have the right version of the user manual.


The most recent editions of user manuals are available for downloading from our website, [www.handicare.com](http://www.handicare.com).

## Description of LiftSeat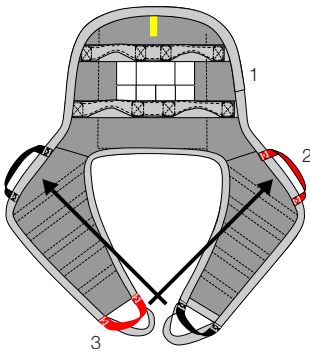

1. LiftSeat has handles in two sections on the back support; choose the handle that enables the best working stance.
2. One of the handles on the upper part of the leg supports is red; after application, it should meet the red handle on the lower part of the opposite leg support. On the opposite side, these handles are black, and should meet each other after application.
3. On the lower part of the leg supports are two handles. For a smaller size, choose the innermost (red or black); for a larger size, choose the outermost (grey).


## Application of LiftSeat when the user is sitting in a wheelchair/chair or in bed

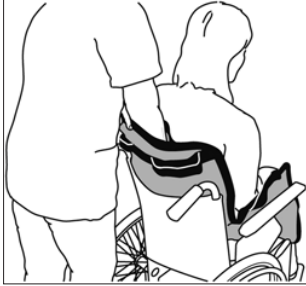Hold LiftSeat in one hand and grasp the lower edge of the sling with your other hand. This is easier if the user leans forward or can be helped to lean forward slightly. Guide the sling down behind the user's back, by sliding it along the back of the chair/wheelchair or along the bed, ideally, to the tailbone.

Take care to guide the sling down to the user's hips. Do this by pushing the sling into the space in the back corners of the wheelchair seat.

Pull the leg supports out along the outside of the thighs. Sit facing the user and pull the lower edge of the leg support, so that the sling slides into place around the hips. Light pressure against the user's knees prevents him/her from sliding forward in the chair.

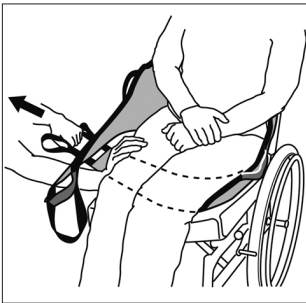
After checking to ensure that the leg supports are of equal length on both sides, place them under the user's thighs. Both leg supports should be under both of the user's legs.

## Transfer or lifting

Choose handles: On the leg sections are two alternative handles. Choose the most appropriate handles according to the width of the user.

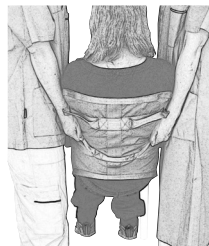At the back, there are handles at two levels; choose the handle that enables the best working stance.

**Plan the transfer so as to minimize lifting.**

## Different positions for lifting


One caregiver on each side.

One caregiver in front and one caregiver behind.

## Removal of the sling when the user is sitting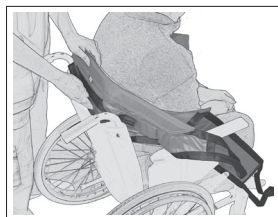


Fold the leg support and allow it to slide under itself when you pull it out at the outside of the thigh.

Then, pull the sling up behind the user's back.


## For extra support behind the back, LiftSeat can be applied with the help of EasyGlide

Place EasyGlide in the pocket on LiftSeat.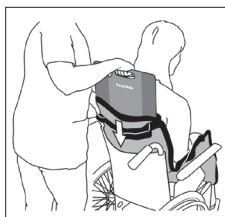

Press LiftSeat with EasyGlide behind the user's back.

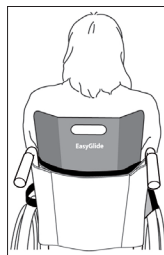Leave EasyGlide behind the user's back for extra support during the transfer.

## Material

Polyester

## Care of the product

Read the product label.

Do not use rinse/ fabric softener and avoid tumble-drying for optimal service life.

**CE** Medical Device Class I

Medical Device Class I. The product complies with the requirements of the Medical Devices Directive 93/42/EEC.

Contact your local distributor if you have any questions about the product and its use. See [www.handicare.com](http://www.handicare.com) for a complete list of distributors.

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## Size

---

<b>Art. no.</b>	<b>Productname</b>	<b>Size</b>	<b>Recommended user weight</b>
4003	LiftSeat	XXS/XS	SWL: 15-45 kg/33-99 lbs
4006	LiftSeat	S/M	SWL: 35-90 kg/80-190 lbs
4007	Liftseat	L/XL	SWL: 80-120 kg/180-260 lbs

## Simple solutions for great results

SystemRoMedic™ is the name of Handicare's unique easy transfer concept, the market's widest and most comprehensive range of clever, easy-to-use and safe transfer and lifting aids designed to make life easier, both for the user and for the caregiver. SystemRoMedic™ is a complete solution that provides for the majority of patient transfer or manual handling requirements. From the simplest to the most complex scenarios, from the lightest to the heaviest. The concept encompasses assistive devices for four different categories of transfers:

- Transfer, assistive devices for manual transfers of users between two locations.
- Positioning, assistive devices for manual repositioning of users within the same location.
- Support, assistive devices for mobility support e.g., during sit-to-stand or gait training.
- Lifting, assistive devices for manual and mechanical lifting of users.

## Improved work environment, improved quality of care and cost savings

The philosophy behind SystemRoMedic™ is focused on the prevention and reduction of occupational injuries while allowing users to experience a greater sense of independence and dignity. Through a unique combination of training and a complete range of efficient transfer aids, SystemRoMedic™ offers improvement of both work environment and quality of care and, at the same time, achieves significant cost savings.

## Always make sure that you have the correct version of the manual

The most recent version of all manuals are available for downloading at/from our website; [www.handicare.com](http://www.handicare.com).

## For questions about the product and its use

Please contact your local Handicare and SystemRoMedic™ representative. A complete list of all our partners with their contact details can be found on our website; [www.handicare.com](http://www.handicare.com).

### ***Handicare offers solutions and support to increase the independence of disabled or elderly people as well as to improve the convenience of those who are caring for them.***

*The Handicare Group is one of the leading healthcare companies in Europe with its own manufacturing organizations and sales companies in Norway, Sweden, Denmark, Germany, the Netherlands, Great Britain, France, China, Canada and the USA. Handicare's products are also distributed by partners in more than 40 countries worldwide. Our wide range of high-quality products includes various manual and power wheelchairs, seating systems, scooters, a complete easy transfer system and other patient handling aids, stairlifts, car adaptations, rise and recline chairs, and bathing and toileting products.*

Handicare AB  
Maskinvägen 17  
SE-972 54 Luleå, SWEDEN  
Tel: +46 (0)8-557 62 200  
Fax: +46 (0)8-557 62 299  
E-mail: [info@handicare.se](mailto:info@handicare.se)  
[www.handicare.com](http://www.handicare.com)