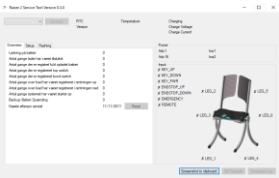
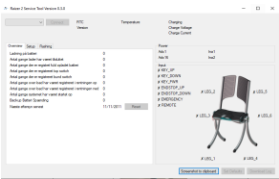



Raizer service report checklist procedures

	Check points	Remarks	pictures	links
1	Check all legs and back rest for defects	Clips and wheels There should be no gap between the Aluminium leg and plastic endcaps		
2	Legs and back rest easy to mount, without forcing	Mount legs and back rest and check it is mechanically okay. Force required for insertion/extraction should be 50N +/-10N		
3	Check all parts for any incipient fractures	Inspect all holes for the legs and backrest.	 	
4	Check the click log system on legs and back rest	You will hear an confirmation tone for each part. (6 pcs). Double confirmation tone when last part is correct fitted.		
5	Remote	Check the remote control using up and down. Inspect the rubber front for gaps between it and the back shell.		
6	Disassemble the Raizer II	See video on Support Site	Disassembly of motor unit: 	https://support.liftup.dk/technical-support/raizer/video-guides/raizer-ii-video-guides/

Raizer service report checklist procedures

	Check points	Remarks	pictures	links
7	Connect Raizer II Service Tool	Make sure you have newest version of Service Tool and Firmware downloaded to you Laptop		
8	Raizer II Service Tool	Verify Firmware version (and upgrade if required). Reset Service Timer. Save Screendump		
9	Battery on main board	Battery, must be at least 3V. If the battery is below 3V it will have a red mark and the battery must be replaced		
10	Assemble the Raizer II	See video on Support Site	Assembly of motor unit: 	https://support.liftup.dk/technical-support/raizer/video-guides/raizer-ii-video-guides/
11	Check lubrication on hinges	If the grease has been smeared off or is missing, it must be reapplied to ensure proper sealing against water ingress.		

Raizer service report checklist procedures

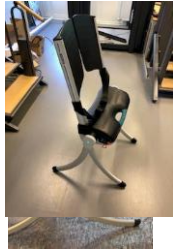	Check points	Remarks	pictures	links
12	Inspect the rubber seals	ensure there are no gaps between the two parts of the housing. Check that the rubber lip on the hinges are correctly seated		
13	Insert legs and back rest	Run complete function test.		
14	Run to top position and back to lower position	Test if end sensor is working correct (for top and lower position)		
15	Overload protection	Push up button (remote or on the Raizer) During transportation use body weight from a second person to check the Raizer stops when overloaded. Press down on the backrest.		
16	Operating buttons	Activate the leg switches on both sides. Test up and down and check the motor is moving. During up or down verify that the motor stops when the emergency cutoff cap is removed. You will also hear a bip.		

Raizer service report checklist procedures

	Check points	Remarks	pictures	links
17	user manual located at the Raizer	Check that the user manual and quick guide is available.		
18	Alarm for missing Remote	Check if Raizer II "is calling" for missing remote (audible alarm)		
19	Auto power off (shuts down after 10min)	Check that the Raizer shuts down when not used ,after max. 10 min.		
20	CE-mark mounted on the rear cover	check label is present		